


Titanium 80 Degree Extension Mount Hanger

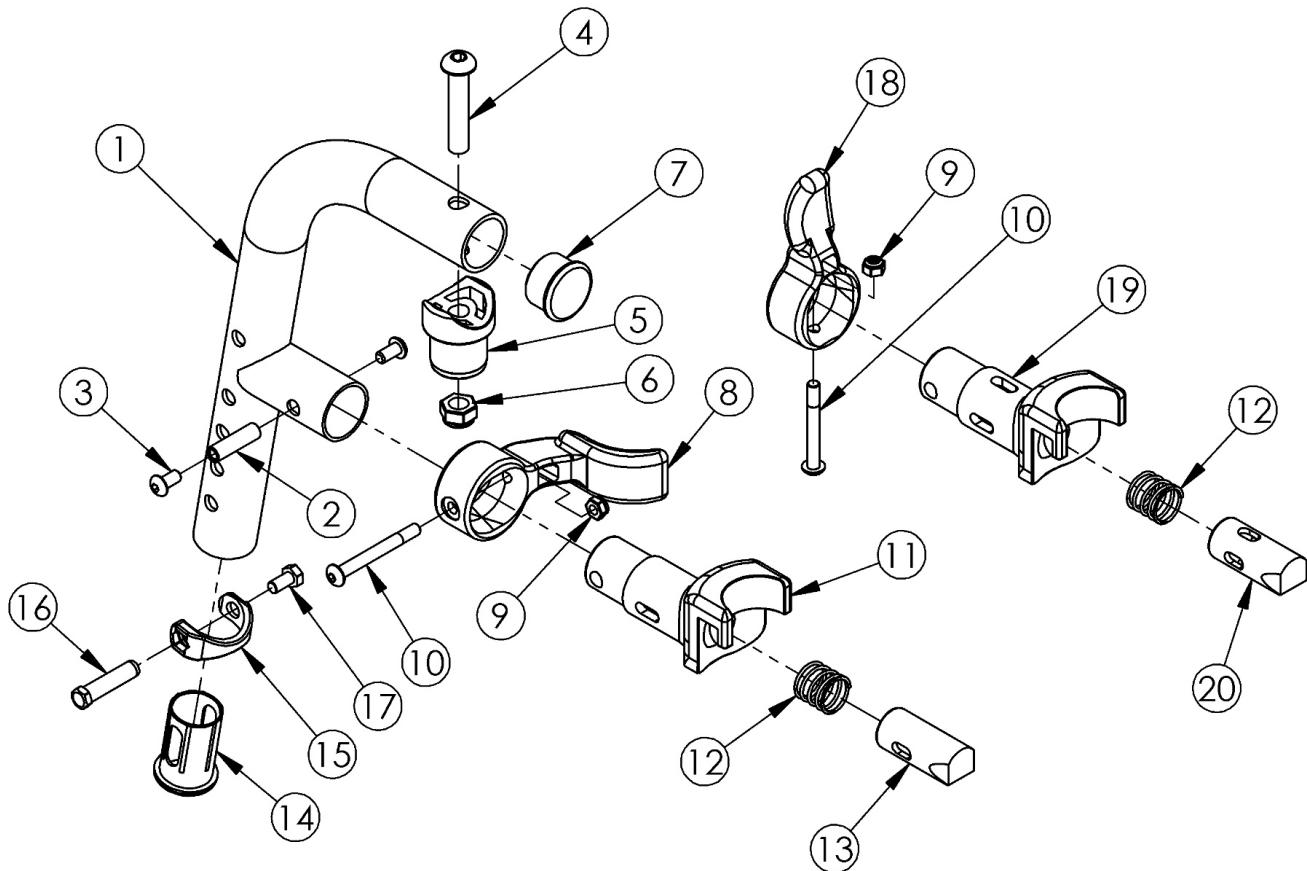

*Please Note: If chair was built with a fixed front frame or hangers were omitted, then latch blocks need to be ordered with hangers.
Please reference the side frame parts manual page.*

Item Number	Part Number	Description	Quantity	Retail
1 - 17	100407	Hanger Assembly Titanium 80 Extension Mount Right	1	\$235.84
1 - 17	100408	Hanger Assembly Titanium 80 Extension Mount Left	1	\$235.84
1-7, 9, 10, 12, 14-20	504744	Hanger Assm Titanium 80 Extension Mount 4-Way - In-Line	1	\$235.84
1-7, 9, 10, 12, 14-20	504745	Hanger Assm Titanium 80 Extension Mount 4-Way - Out Left	1	\$235.84
1-7, 9, 10, 12, 14-20	504746	Hanger Assm Titanium 80 Extension Mount 4-Way - Out Right	1	\$235.84
1, 7, 14	501020	Hanger Tube Assembly Titanium 80 Extension Mount	1	\$180.64
8 - 13	003417	Classic Hanger Latch Assembly, Left	1	\$63.78
8 - 13	003416	Classic Hanger Latch Assembly, Right	1	\$63.78

Item Number	Part Number	Description	Quantity	Retail
9, 10, 12, 18-20	003225	Assembled 4-Way In Line Hanger Release	1	\$36.77
9, 10, 12, 18-20	003225INST	Assembled 4-Way In Line Hanger Release with Instruction	1	\$36.77
9, 10, 12, 18-20	003643	Assembled 4-Way Out Hanger Release, Left	1	\$36.77
9, 10, 12, 18-20	003643INST	Assembled 4-Way Out Hanger Release, Left with Instruction	1	\$36.77
9, 10, 12, 18-20	003642	Assembled 4-Way Out Hanger Release, Right	1	\$36.77
9, 10, 12, 18-20	003642INST	Assembled 4-Way Out Hanger Release, Right with Instruction	1	\$36.77
2	000073	M5 x 24 Threaded Barrel	1	\$3.78
3	100669	M5 x 10 BTN SHCS BLKZC w/Patch	2	\$1.58
4	100673	M8 x 45 BTN SHCS BLKZC	1	\$1.58
5	000032	Footrest Hanger Pivot Saddle	1	\$5.25
6	100659	M8 Nylock Nut Blk Zn	1	\$1.05
7	100804	Tube End Plug 1"	1	\$1.58
8	001158	Lever Swing Away Short	1	\$7.31
9	100657	M5 Nylock Nut Blk Zn	1	\$0.53
10	001374	M5 x 40 BHCS P/THD BLKZC	1	\$2.14
11	000031	Hanger Bracket	1	\$23.73
12	000119	Hanger Spring	1	\$3.05
14	000034	Sleeve 3/4" x 1 1/2"	1	\$3.66
15	000988	Fastener Retainer	1	\$3.94
16	000987	Threaded Fastener	1	\$3.78
17	200427	M5 x 10 CLS8.8 HHCS ZC w/Patch DIN933	1	\$0.53
18	003228	Hanger Latch, 4-Way, Lever	1	\$3.15
18	003228INST	Hanger Latch, 4-Way, Lever with Instruction	1	\$3.15
19	003226	Hanger Latch, 4-Way, Bracket	1	\$5.25
19	003226INST	Hanger Latch, 4-Way, Bracket with Instruction	1	\$5.25
20	003227	Hanger Latch, 4-Way, Pin	1	\$19.96
20	003227INST	Hanger Latch, 4-Way, Pin with Instruction	1	\$19.96