




# **(Discontinued) Rogue Caster Forks and Stems**

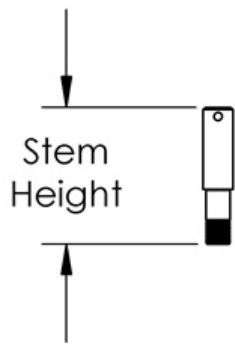

Std	2.0"
+ 3/4"	2.8"

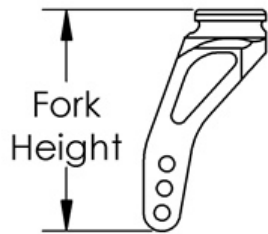

Short	4.5"
Tall	4.9"

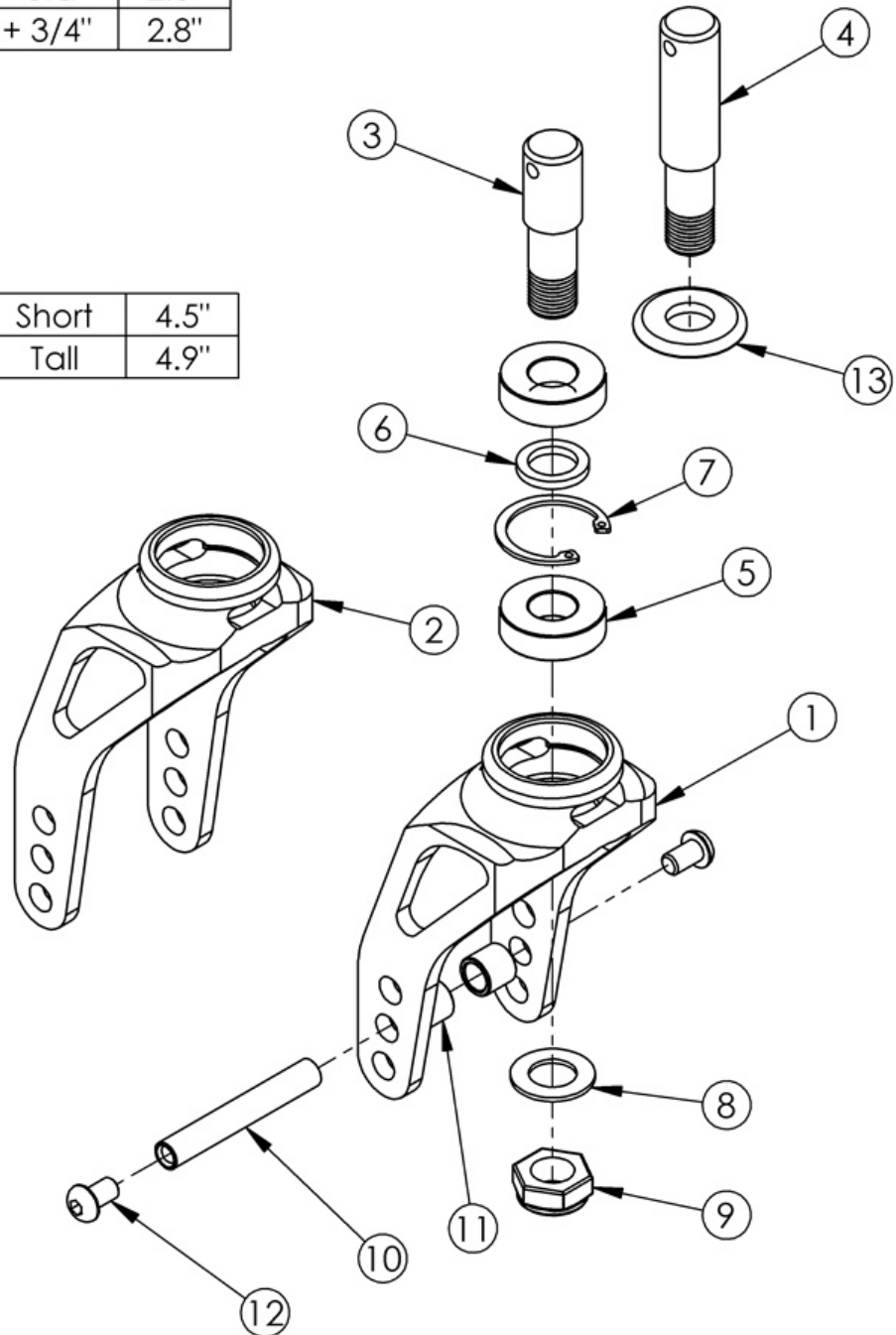

Item Number	Part Number	Description	Quantity
1b, 4-9, 13	500783_ _ _	Color Anodized Fork and Stem Assembly Short 3/4	1
2b, 4-9, 13	500785_ _ _	Color Anodized Fork and Stem Assembly Tall 3/4	1
2b, 5-7	000928_ _ _	Color Anodized Caster Fork Tall w/ Bearing Housing RG <i>Replaced with 004362_ _ _</i>	1
4	000930INST	Caster Stem In Line +3/4 Rogue/Clik <i>Replaced with 004371INST</i>	1
8	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	1
9	101459	1/2-20 NTE JAM NYL INS L/N BLK ZC	1
10	000422	M6 x 56 Threaded Barrel	1
11	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer	2
12	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2
13a	001355	Caster Bearing Cap Black Anodized <i>Used with 3/4" stem only</i>	1
13b	001355_ _ _	Color Anodized Caster Bearing Cap	1

# Color Anodizing Samples

