
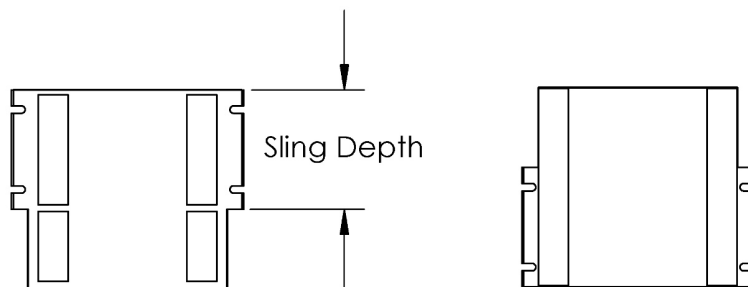


Clik Growing Seat Sling

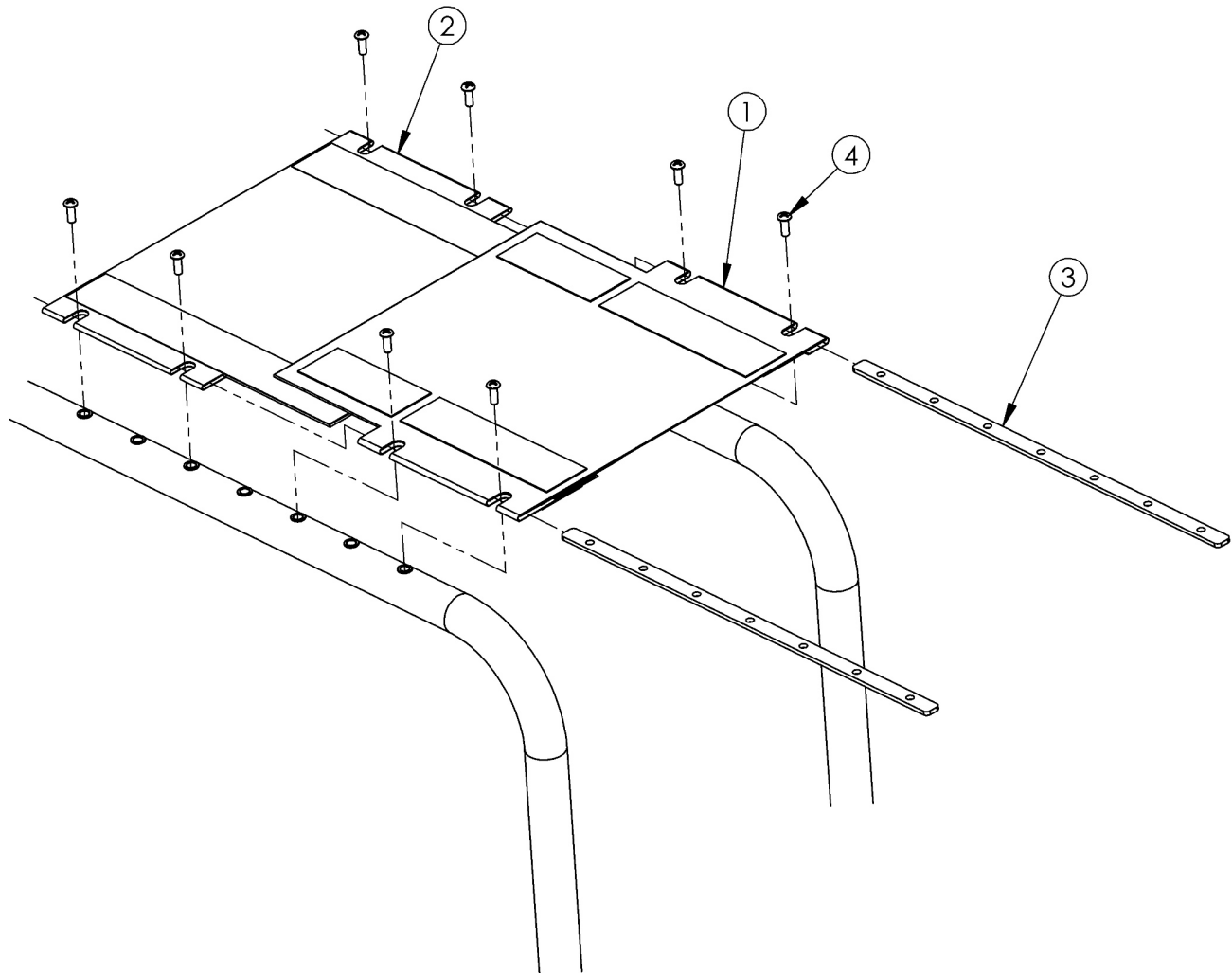

Front		
Part No's	Depth	For Chair Depths
002331 - 002339	4.0"	8"-11"
002347 - 002361	6.0"	12"-13"
002374 - 002383	8.0"	14"-18"

Rear		
Part No's	Depth	For Chair Depths
002340 - 002346	4.0"	8'-9"
002363 - 002373	6.0"	10"-15"
002384 - 002391	8.0"	16"-18"

Seat sling width measures approximately 0.75" less than the chair size. See image above for seat sling depth measurement.

Item Number	Part Number	Description	Quantity
3a, 4	502620	Seat Rail 7-10" Deep w/ Hardware PAIR Klik	1
3b, 4	502621	Seat Rail 9-12" Deep w/ Hardware PAIR Klik	1
3c, 4	502622	Seat Rail 11-14" Deep w/ Hardware PAIR Klik	1
3d, 4	502623	Seat Rail 13-16" Deep w/ Hardware PAIR Klik	1
3e, 4	502624	Seat Rail 15-18" Deep w/ Hardware PAIR Klik	1
1a	002331	8" Wide 4" Deep Front Seat Sling	1
1b	002332	9" Wide 4" Deep Front Seat Sling	1
1c	002333	10" Wide 4" Deep Front Seat Sling	1
1d	002334	11" Wide 4" Deep Front Seat Sling	1
1e	002335	12" Wide 4" Deep Front Seat Sling	1
1f	002336	13" Wide 4" Deep Front Seat Sling	1
1g	002337	14" Wide 4" Deep Front Seat Sling	1
1h	002338	15" Wide 4" Deep Front Seat Sling	1
1i	002339	16" Wide 4" Deep Front Seat Sling	1
1j	002347	8" Wide 6" Deep Front Seat Sling	1
1k	002348	9" Wide 6" Deep Front Seat Sling	1
1l	002353	10" Wide 6" Deep Front Seat Sling	1
1m	002354	11" Wide 6" Deep Front Seat Sling	1
1n	002355	12" Wide 6" Deep Front Seat Sling	1
1o	002356	13" Wide 6" Deep Front Seat Sling	1
1p	002357	14" Wide 6" Deep Front Seat Sling	1
1q	002358	15" Wide 6" Deep Front Seat Sling	1
1r	002359	16" Wide 6" Deep Front Seat Sling	1
1s	002360	17" Wide 6" Deep Front Seat Sling	1
1t	002361	18" Wide 6" Deep Front Seat Sling	1
1u	002374	9" Wide 8" Deep Front Seat Sling	1

Item Number	Part Number	Description	Quantity
1v	002375	10" Wide 8" Deep Front Seat Sling	1
1w	002376	11" Wide 8" Deep Front Seat Sling	1
1x	002377	12" Wide 8" Deep Front Seat Sling	1
1y	002378	13" Wide 8" Deep Front Seat Sling	1
1z	002379	14" Wide 8" Deep Front Seat Sling	1
1aa	002380	15" Wide 8" Deep Front Seat Sling	1
1ab	002381	16" Wide 8" Deep Front Seat Sling	1
1ac	002382	17" Wide 8" Deep Front Seat Sling	1
1ad	002383	18" Wide 8" Deep Front Seat Sling	1
2a	002340	8" Wide 4" Deep Rear Seat Sling	1
2b	002341	9" Wide 4" Deep Rear Seat Sling	1
2c	002342	10" Wide 4" Deep Rear Seat Sling	1
2d	002343	11" Wide 4" Deep Rear Seat Sling	1
2e	002344	12" Wide 4" Deep Rear Seat Sling	1
2f	002345	13" Wide 4" Deep Rear Seat Sling	1
2g	002346	14" Wide 4" Deep Rear Seat Sling	1
2h	002363	8" Wide 6" Deep Rear Seat Sling	1
2i	002364	9" Wide 6" Deep Rear Seat Sling	1
2j	002365	10" Wide 6" Deep Rear Seat Sling	1
2k	002366	11" Wide 6" Deep Rear Seat Sling	1
2l	002367	12" Wide 6" Deep Rear Seat Sling	1
2m	002368	13" Wide 6" Deep Rear Seat Sling	1
2n	002369	14" Wide 6" Deep Rear Seat Sling	1
2o	002370	15" Wide 6" Deep Rear Seat Sling	1
2p	002371	16" Wide 6" Deep Rear Seat Sling	1
2q	002372	17" Wide 6" Deep Rear Seat Sling	1
2r	002373	18" Wide 6" Deep Rear Seat Sling	1
2s	002384	11" Wide 8" Deep Rear Seat Sling	1

Item Number	Part Number	Description	Quantity
2t	002385	12" Wide 8" Deep Rear Seat Sling	1
2u	002386	13" Wide 8" Deep Rear Seat Sling	1
2v	002387	14" Wide 8" Deep Rear Seat Sling	1
2w	002388	15" Wide 8" Deep Rear Seat Sling	1
2x	002389	16" Wide 8" Deep Rear Seat Sling	1
2y	002390	17" Wide 8" Deep Rear Seat Sling	1
2z	002391	18" Wide 8" Deep Rear Seat Sling	1
3a	002064	Seat Rail 7-10 LW	2
3b	002065	Seat Rail 9-12 LW	2
3c	002066	Seat Rail 11-14 LW	2
3d	002067	Seat Rail 13-16 LW	2
3e	002068	Seat Rail 15-18 LW	2
4	100734	M5 x 16 PHMS ZC	8