


Ethos Transit

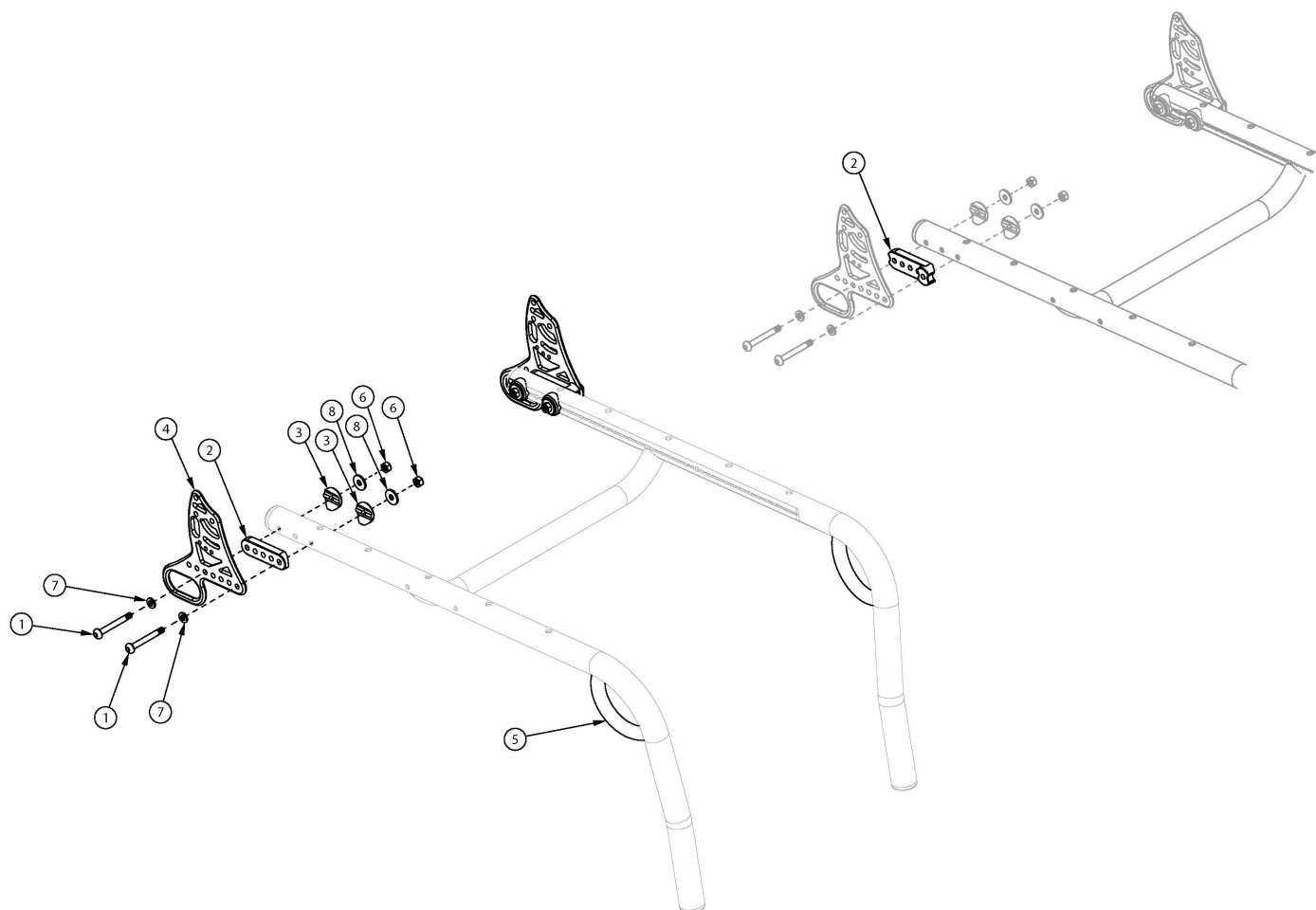

Ethos Transit cannot be retrofitted <p>

If needing to add transit, please contact Customer Service for a Replacement Seat Frame

Item Number	Part Number	Description	Quantity	Retail
1a	003846	M6 x 50 BHCS, Custom, Blk Zc <i>Used if Swing Away Arms are not present</i>	4	\$14.33
1b	003232	M6 x 55 BHCS Blk Zc <i>Used if Swing Away Arms are present</i>	4	\$8.76
2a	003848	Back Bracket Saddle 1 1/8" <i>Used if Swing Away Arms are not present</i>	2	\$1.06
2b	004116	Saddle, Swing Away Arm, 1 1/8" Tube <i>Used if Swing Away Arms are present</i>	2	\$1.11
3	003849	Saddle For Dimple 1 1/8"	4	\$0.43
4	109296	Back Bracket with Transit, Black, Rigid	2	\$26.20

Item Number	Part Number	Description	Quantity	Retail
6	100658	M6 Nylock Nut Blk Zn	4	\$1.03
7	100746	M6 Flat Washer Blk Zn	4	\$1.55
8	101853	M6 Washer Blk Zn	4	\$0.52