


Standard Elevating Leg Rest

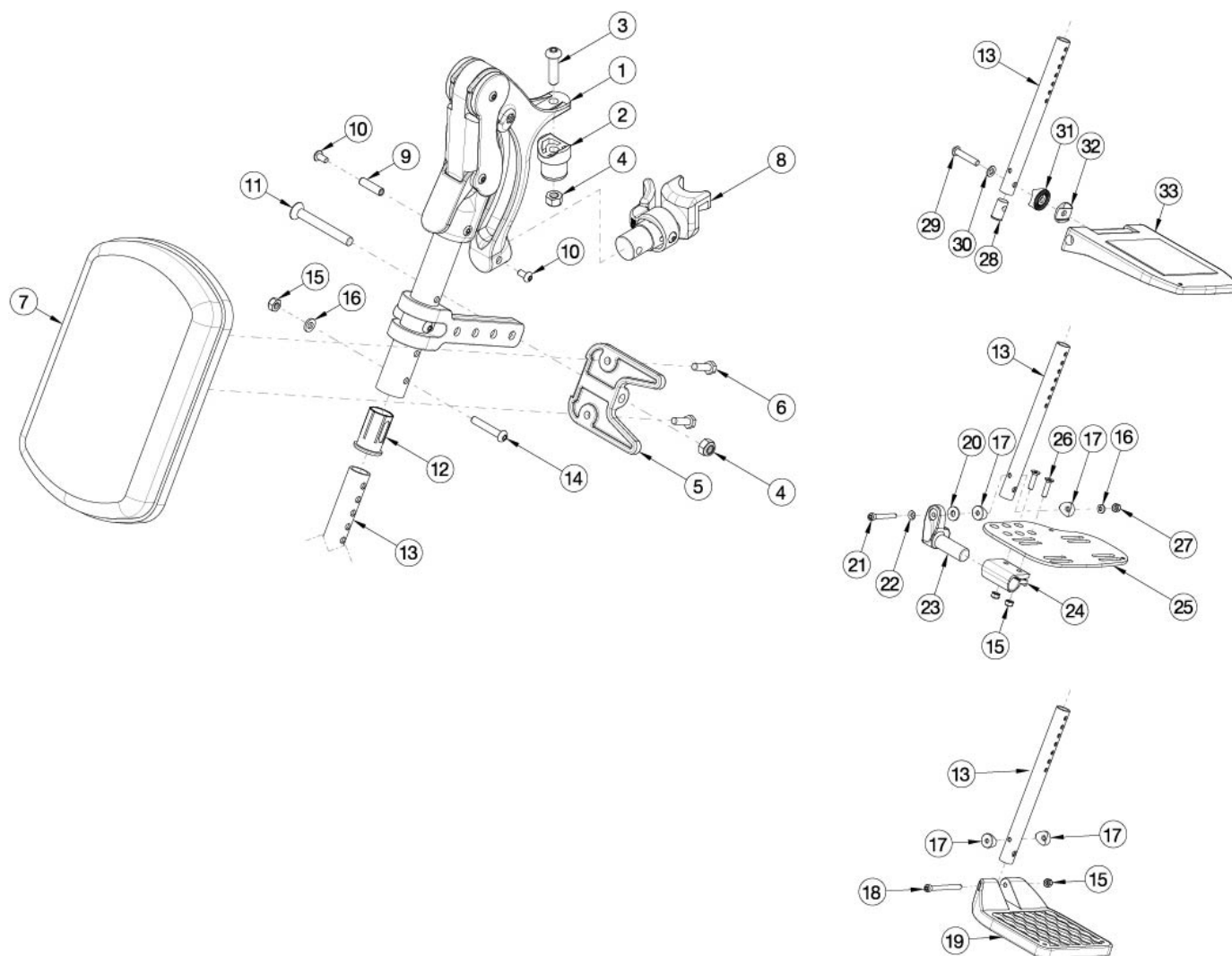

Sizes for assemblies are based on seat width with straight frame; if Tilt chair has Offset Frame, add 2" to seat width. If chair was built with a fixed front frame or hangers were omitted, then latch blocks need to be ordered with ELRs. Please reference the "Front Frame" parts manual page for Catalyst chairs and the "Seat Frame" parts manual page for Focus, Flip, and Liberty chairs. If switching from a contracture footrest to an ELR, front seat tubes and latch blocks with hardware will be needed.

Item Number	Part Number	Description	Quantity
1-6, 7a, 8a, 8b, 9-12, 13a, 14-18, 19a	122621	Standard ELR Assembly 14, Composite Footplate, Short Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-18, 19a	122622	Standard ELR Assembly 15, Composite Footplate, Short Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-18, 19b	122623	Standard ELR Assembly 16-17, Composite Footplate, Short Ext, Classic Release, Pair	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8a, 8b, 9-12, 13a, 14-18, 19c	122624	Standard ELR Assembly 18-24, Composite Footplate, Short Ext, Classic Release, Pair	1
1-6, 7b, 8a, 9-12, 13a, 14-18, 19a	122625	Standard ELR Assembly 14, Composite Footplate, Short Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13a, 14-18, 19a	122626	Standard ELR Assembly 15, Composite Footplate, Short Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13a, 14-18, 19b	122627	Standard ELR Assembly 16-17, Composite Footplate, Short Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13a, 14-18, 19c	122628	Standard ELR Assembly 18-24, Composite Footplate, Short Ext, Classic Release, Right	1
1-6, 7b, 8b, 9-12, 13a, 14-18, 19a	122629	Standard ELR Assembly 14, Composite Footplate, Short Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13a, 14-18, 19a	122630	Standard ELR Assembly 15, Composite Footplate, Short Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13a, 14-18, 19b	122631	Standard ELR Assembly 16-17, Composite Footplate, Short Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13a, 14-18, 19c	122632	Standard ELR Assembly 18-24, Composite Footplate, Short Ext, Classic Release, Left	1
1-6, 7b, 8a, 8b, 9-12, 13b, 14-18, 19a	122633	Standard ELR Assembly 14, Composite Footplate, Medium Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-18, 19a	122634	Standard ELR Assembly 15, Composite Footplate, Medium Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-18, 19b	122635	Standard ELR Assembly 16-17, Composite Footplate, Medium Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-18, 19c	122636	Standard ELR Assembly 18-24, Composite Footplate, Medium Ext, Classic Release, Pair	1
1-6, 7b, 8a, 9-12, 13b, 14-18, 19a	122637	Standard ELR Assembly 14, Composite Footplate, Medium Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13b, 14-18, 19a	122638	Standard ELR Assembly 15, Composite Footplate, Medium Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13b, 14-18, 19b	122639	Standard ELR Assembly 16-17, Composite Footplate, Medium Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13b, 14-18, 19c	122640	Standard ELR Assembly 18-24, Composite Footplate, Medium Ext, Classic Release, Right	1
1-6, 7b, 8b, 9-12, 13b, 14-18, 19a	122641	Standard ELR Assembly 14, Composite Footplate, Medium Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13b, 14-18, 19a	122642	Standard ELR Assembly 15, Composite Footplate, Medium Ext, Classic Release, Left	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8b, 9-12, 13b, 14-18, 19b	122643	Standard ELR Assembly 16-17, Composite Footplate, Medium Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13b, 14-18, 19c	122644	Standard ELR Assembly 18-24, Composite Footplate, Medium Ext, Classic Release, Left	1
1-6, 7b, 8a, 8b, 9-12, 13c, 14-18, 19a	122645	Standard ELR Assembly 14, Composite Footplate, Long Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-18, 19a	122646	Standard ELR Assembly 15, Composite Footplate, Long Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-18, 19b	122647	Standard ELR Assembly 16-17, Composite Footplate, Long Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-18, 19c	122648	Standard ELR Assembly 18-24, Composite Footplate, Long Ext, Classic Release, Pair	1
1-6, 7b, 8a, 9-12, 13c, 14-18, 19a	122649	Standard ELR Assembly 14, Composite Footplate, Long Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13c, 14-18, 19a	122650	Standard ELR Assembly 15, Composite Footplate, Long Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13c, 14-18, 19b	122651	Standard ELR Assembly 16-17, Composite Footplate, Long Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13c, 14-18, 19c	122652	Standard ELR Assembly 18-24, Composite Footplate, Long Ext, Classic Release, Right	1
1-6, 7b, 8b, 9-12, 13c, 14-18, 19a	122653	Standard ELR Assembly 14, Composite Footplate, Long Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13c, 14-18, 19a	122654	Standard ELR Assembly 15, Composite Footplate, Long Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13c, 14-18, 19b	122655	Standard ELR Assembly 16-17, Composite Footplate, Long Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13c, 14-18, 19c	122656	Standard ELR Assembly 18-24, Composite Footplate, Long Ext, Classic Release, Left	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-16, 28-32, 33a	122657	Standard ELR Assembly 15-16 Comp Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-16, 28-32, 33b	122658	Standard ELR Assembly 17-18 Comp Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-16, 28-32, 33c	122659	Standard ELR Assembly 19-22 Comp Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7a, 8a, 9-12, 13a, 14-16, 28-32, 33a	122660	Standard ELR Assembly 15-16 Comp Angle Adj FP Short Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-16, 28-32, 33b	122661	Standard ELR Assembly 17-18 Comp Angle Adj FP Short Ext Classic Release Right	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8a, 9-12, 13a, 14-16, 28-32, 33c	122662	Standard ELR Assembly 19-22 Comp Angle Adj FP Short Ext Classic Release Right	1
1-6, 7a, 8b, 9-12, 13a, 14-16, 28-32, 33a	122663	Standard ELR Assembly 15-16 Comp Angle Adj FP Short Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13a, 14-16, 28-32, 33b	122664	Standard ELR Assembly 17-18 Comp Angle Adj FP Short Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13a, 14-16, 28-32, 33c	122665	Standard ELR Assembly 19-22 Comp Angle Adj FP Short Ext Classic Release Left	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-16, 28-32, 33a	122666	Standard ELR Assembly 15-16 Comp Angle Adj FP Medium Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-16, 28-32, 33b	122667	Standard ELR Assembly 17-18 Comp Angle Adj FP Medium Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-16, 28-32, 33c	122668	Standard ELR Assembly 19-22 Comp Angle Adj FP Medium Ext Classic Release Pair	1
1-6, 7a, 8a, 9-12, 13b, 14-16, 28-32, 33a	122669	Standard ELR Assembly 15-16 Comp Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-16, 28-32, 33b	122670	Standard ELR Assembly 17-18 Comp Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-16, 28-32, 33c	122671	Standard ELR Assembly 19-22 Comp Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7a, 8b, 9-12, 13b, 14-16, 28-32, 33a	122672	Standard ELR Assembly 15-16 Comp Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-16, 28-32, 33b	122673	Standard ELR Assembly 17-18 Comp Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-16, 28-32, 33c	122674	Standard ELR Assembly 19-22 Comp Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-16, 28-32, 33a	122675	Standard ELR Assembly 15-16 Comp Angle Adj FP Long Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-16, 28-32, 33b	122676	Standard ELR Assembly 17-18 Comp Angle Adj FP Long Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-16, 28-32, 33c	122677	Standard ELR Assembly 19-22 Comp Angle Adj FP Long Ext Classic Release Pair	1
1-6, 7a, 8a, 9-12, 13c, 14-16, 28-32, 33a	122678	Standard ELR Assembly 15-16 Comp Angle Adj FP Long Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-16, 28-32, 33b	122679	Standard ELR Assembly 17-18 Comp Angle Adj FP Long Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-16, 28-32, 33c	122680	Standard ELR Assembly 19-22 Comp Angle Adj FP Long Ext Classic Release Right	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8b, 9-12, 13c, 14-16, 28-32, 33a	122681	Standard ELR Assembly 15-16 Comp Angle Adj FP Long Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13c, 14-16, 28-32, 33b	122682	Standard ELR Assembly 17-18 Comp Angle Adj FP Long Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13c, 14-16, 28-32, 33c	122683	Standard ELR Assembly 19-22 Comp Angle Adj FP Long Ext Classic Release Left	1
1-6, 7b, 8a, 8b, 9-12, 13a, 14-17, 20-24, 25a, 25b, 26, 27	122684	Standard ELR Assembly 14 Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-17, 20-24, 25a, 25b, 26, 27	122685	Standard ELR Assembly 15 Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-17, 20-24, 25c, 25d, 26, 27	122686	Standard ELR Assembly 16-17 Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-17, 20-24, 25e, 25f, 26, 27	122687	Standard ELR Assembly 18-24 Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7b, 8a, 9-12, 13a, 14-17, 20-24, 25b, 26, 27	122688	Standard ELR Assembly 14 Angle Adj FP Short Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-17, 20-24, 25b, 26, 27	122689	Standard ELR Assembly 15 Angle Adj FP Short Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-17, 20-24, 25d, 26, 27	122690	Standard ELR Assembly 16-17 Angle Adj FP Short Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-17, 20-24, 25f, 26, 27	122691	Standard ELR Assembly 18-24 Angle Adj FP Short Ext Classic Release Right	1
1-6, 7b, 8b, 9-12, 13a, 14-17, 20-24, 25a, 26, 27	122692	Standard ELR Assembly 14 Angle Adj FP Short Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13a, 14-17, 20-24, 25a, 26, 27	122693	Standard ELR Assembly 15 Angle Adj FP Short Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13a, 14-17, 20-24, 25c, 26, 27	122694	Standard ELR Assembly 16-17 Angle Adj FP Short Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13a, 14-17, 20-24, 25e, 26, 27	122695	Standard ELR Assembly 18-24 Angle Adj FP Short Ext Classic Release Left	1

Item Number	Part Number	Description	Quantity
1-6, 7b, 8a, 8b, 9-12, 13b, 14-17, 20-24, 25a, 25b, 26, 27	122696	Standard ELR Assembly 14 Angle Adj FP Medium Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-17, 20-24, 25a, 25b, 26, 27	122697	Standard ELR Assembly 15 Angle Adj FP Medium Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-17, 20-24, 25c, 25d, 26, 27	122698	Standard ELR Assembly 16-17 Angle Adj FP Medium Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-17, 20-24, 25e, 25f, 26, 27	122699	Standard ELR Assembly 18-24 Angle Adj FP Medium Ext Classic Release Pair	1
1-6, 7b, 8a, 9-12, 13b, 14-17, 20-24, 25b, 26, 27	122700	Standard ELR Assembly 14 Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-17, 20-24, 25b, 26, 27	122701	Standard ELR Assembly 15 Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-17, 20-24, 25d, 26, 27	122702	Standard ELR Assembly 16-17 Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-17, 20-24, 25f, 26, 27	122703	Standard ELR Assembly 18-24 Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7b, 8b, 9-12, 13b, 14-17, 20-24, 25a, 26, 27	122704	Standard ELR Assembly 14 Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-17, 20-24, 25a, 26, 27	122705	Standard ELR Assembly 15 Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-17, 20-24, 25c, 26, 27	122706	Standard ELR Assembly 16-17 Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-17, 20-24, 25e, 26, 27	122707	Standard ELR Assembly 18-24 Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7b, 8a, 8b, 9-12, 13c, 14-17, 20-24, 25a, 25b, 26, 27	122708	Standard ELR Assembly 14 Angle Adj FP Long Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-17, 20-24, 25a, 25b, 26, 27	122709	Standard ELR Assembly 15 Angle Adj FP Long Ext Classic Release Pair	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8a, 8b, 9-12, 13c, 14-17, 20-24, 25c, 25d, 26, 27	122710	Standard ELR Assembly 16-17 Angle Adj FP Long Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-17, 20-24, 25e, 25f, 26, 27	122711	Standard ELR Assembly 18-24 Angle Adj FP Long Ext Classic Release Pair	1
1-6, 7b, 8a, 9-12, 13c, 14-17, 20-24, 25b, 26, 27	122712	Standard ELR Assembly 14 Angle Adj FP Long Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-17, 20-24, 25b, 26, 27	122713	Standard ELR Assembly 15 Angle Adj FP Long Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-17, 20-24, 25d, 26, 27	122714	Standard ELR Assembly 16-17 Angle Adj FP Long Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-17, 20-24, 25f, 26, 27	122715	Standard ELR Assembly 18-24 Angle Adj FP Long Ext Classic Release Right	1
1-6, 7b, 8b, 9-12, 13c, 14-17, 20-24, 25a, 26, 27	122716	Standard ELR Assembly 14 Angle Adj FP Long Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13c, 14-17, 20-24, 25a, 26, 27	122717	Standard ELR Assembly 15 Angle Adj FP Long Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13c, 14-17, 20-24, 25c, 26, 27	122718	Standard ELR Assembly 16-17 Angle Adj FP Long Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13c, 14-17, 20-24, 25e, 26, 27	122719	Standard ELR Assembly 18-24 Angle Adj FP Long Ext Classic Release Left	1
1-6, 8a, 9-12, 14-16	122720	Standard ELR Upper Assembly with Classic Hanger Latch Release Right	1
1-6, 8b, 9-12, 14-17	122721	Standard ELR Upper Assembly with Classic Hanger Latch Release Left	1
2	000032	Footrest Hanger Pivot Saddle	2
3	103551	BHCS, M8 x 1.25 x 30 BLZ	2
4	100659	M8 Nylock Nut Blk Zn	2
5	003139	Swivel, Calf Pad	2
6	101823	M6 X 20 Hhcs Blk Zn	4
7a	101585	Calf Pad	2

Item Number	Part Number	Description	Quantity
7b	101470	Pediatric Calf Pad	2
8a	003416	Classic Hanger Latch Assembly, Right	1
8b	003417	Classic Hanger Latch Assembly, Left	1
9	000073	M5 x 24 Threaded Barrel	2
10	100669	M5 x 10 BTN SHCS BLKZC w/Patch	4
12	000034	Sleeve 3/4" x 1 1/2"	2
13a	118966	Standard ELR Extension Tube Short	2
13b	118967	Standard ELR Extension Tube, Medium	2
13c	118968	Standard ELR Extension Tube, Long	2
14	101811	M6 x 35 BHCS Blk Zn	2
15	100658	M6 Nylock Nut Blk Zn	6
16	100746	M6 Flat Washer Blk Zn	4
17	000053	Saddle 3/4"	2
18	101849	M6 x 65 SHCS Blk Zn	2
19a	100537	Footplate Composite 14-15" Wide CAT/TILT	2
19b	100540	Footplate Composite 16-17" Wide CAT/TILT	2
19c	100542	Footplate Composite 18-24" Wide CAT/TILT	2
20	000428	Wear Washer	2
21	101472	M6 x 45 SHCS BLK ZC	2
22	101455	M6 Belleville Washer (.252 x .551 x .050)	2
23	000047	Footplate Angle Adjustable Casting 3/4 Right	2
24	100502	Angle Adj Footplate Clamp 2"	2
25a	000105	Footplate Angle Adjustable 14-15" Wide CAT/TILT Left	1
25b	000106	Footplate Angle Adjustable 14-15" Wide CAT/TILT Right	1
25c	000107	Footplate Angle Adjustable 16-17" Wide CAT/TILT Left	1
25d	000108	Footplate Angle Adjustable 16-17" Wide CAT/TILT Right	1
25e	000109	Footplate Angle Adjustable 18-24" Wide CAT/TILT Left	1

Item Number	Part Number	Description	Quantity
25f	000110	Footplate Angle Adjustable 18-24" Wide CAT/TILT Right	1
26	100662	M6 x 25 FSHCS BLK ZC	2
27	120030	M6 Top Lock Nut	2
28	003161	Footplate Plug for Composite Angle Adjustable Footplate Extension Tube	2
29	003189	Bolt M8 x 45MM, Galvanized Steel	2
30	003188	Washer D8.4 x 1.6 MM	2
31	003187	Locking Wheel w/Radius, R12.5 & R10, D30 x 11.3, Cast Steel, Black	2
32	003186	Locking Wheel w/Hexagon, D30 x 6.8/6, Cast Steel, Black	2
33a	003173	Composite Angle Adjustable Footplate Small 15-16" Wide	2
33b	003174	Composite Angle Adjustable Footplate Medium 17-18" Wide	2
33c	003175	Composite Angle Adjustable Footplate Large 19-24" Wide	2