


# Flip Foot Tilt

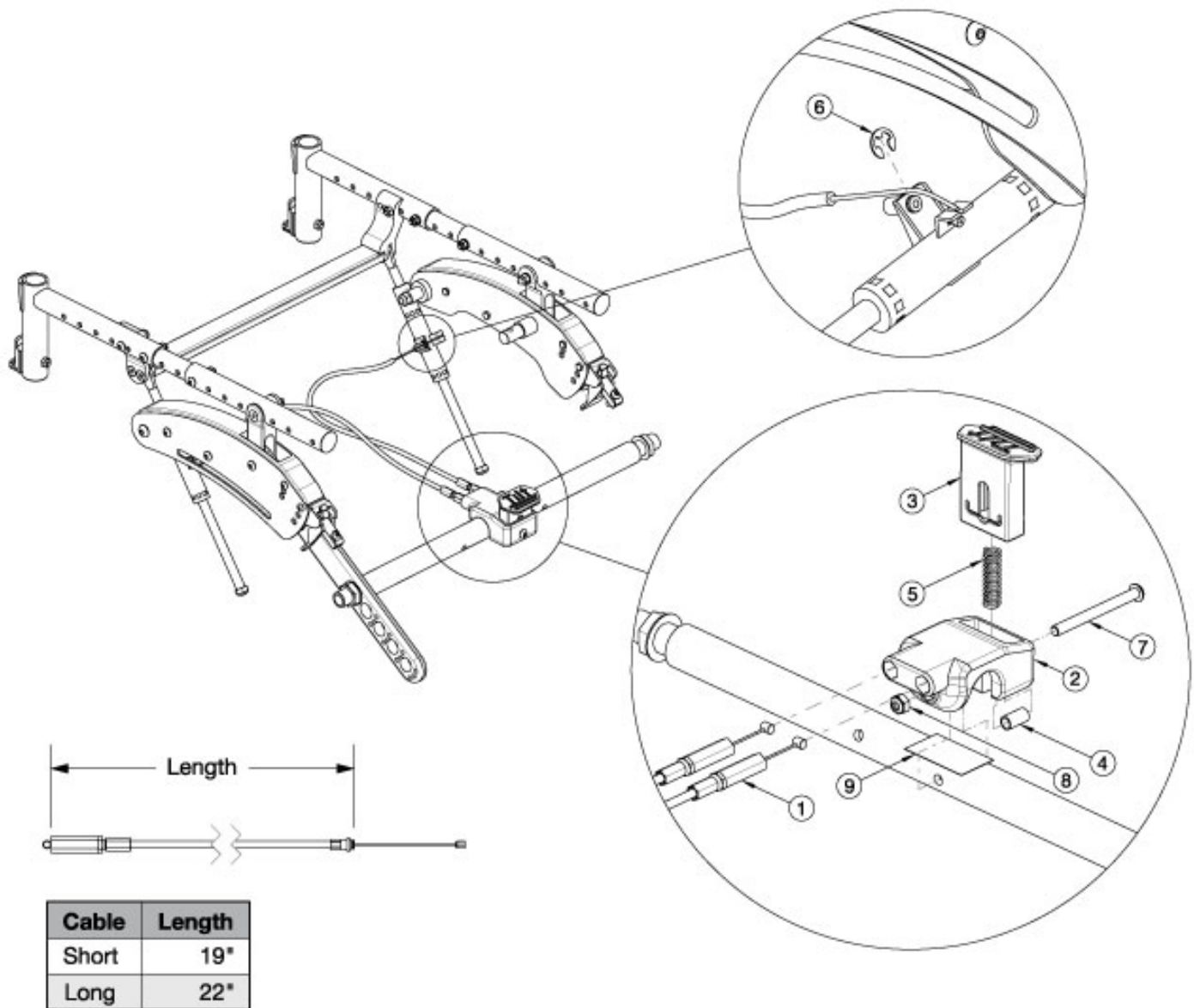

Item Number	Part Number	Description	Quantity
1a, 6, 9	502834	Cable, Foot Tilt, Short FLIP <i>Used with Self Propel Axle Plate or with Standard Axle Plate and 10-13" Widths. This kit contains a pair of cables.</i>	1
1b, 6, 9	502835	Cable, Foot Tilt, Long FLIP <i>Used with Standard Axle Plate and 14-18" Widths. This kit contains a pair of cables.</i>	1
2	001919	Foot Tilt Housing	1
3	001920	Button, Foot Tilt	1

Item Number	Part Number	Description	Quantity
4	001921	Housing Sleeve, Foot Tilt	1
5	001922	Spring, Foot Tilt	1
6	102822	E-Ring, M6, YZ	2
7	101127	M6 x 65 BTN SHCS BLKZC	1
8	100658	M6 Nylock Nut Blk Zn	1
1b, 2 - 9	502827	Foot Tilt Kit, Long, Flip <i>Used with Standard Axle Plate and 14-18" Widths.</i>	1
1a, 2 - 9	502828	Foot Tilt Kit, Short, Flip <i>Used with Self Propel Axle Plate or with Standard Axle Plate and 10-13" Widths.</i>	1
2-9	502829	Foot Tilt Pedal Kit FLIP	1