


## Catalyst 4 Armrests - Tall Height Adjustable Flip Back T-Arm

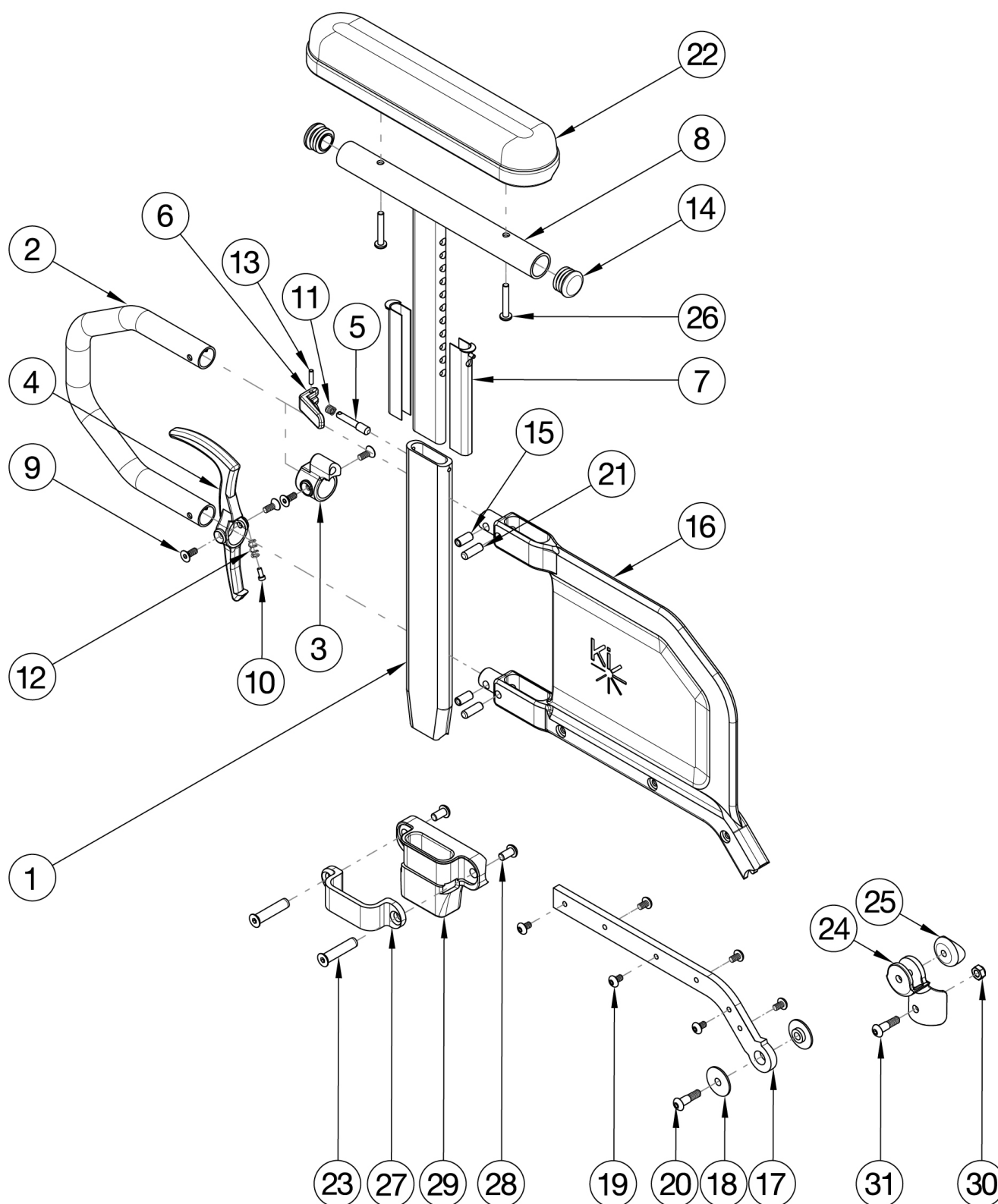

Receiver and Hardware update effective 11/25/24 Short Side Shield used with 14" - 17" Seat Depths. Long Side Shield used with 18" - 20" Seat Depths. If retrofiting, Receivers are needed.

| Item Number                                        | Part Number | Description                                                                                                                | Quantity |
|----------------------------------------------------|-------------|----------------------------------------------------------------------------------------------------------------------------|----------|
| 1-7, 8a, 9-15, 16a, 17a, 18-21, 22b, 23-31         | 116230      | Height Adjustable Tall Flip Back T-Arm Short Desk with Standard Pad and Receiver Assembly, Catalyst 4                      | 1        |
| 1-7, 8b, 9-15, 16a, 17a, 18-21, 22a, 23-31         | 116231      | Height Adjustable Tall Flip Back T-Arm Short Full with Standard Pad and Receiver Assembly, Catalyst 4                      | 1        |
| 1-7, 8a, 9-15, 16b, 17b, 18-21, 22b, 23-31         | 116232      | Height Adjustable Tall Flip Back T-Arm Long Desk with Standard Pad and Receiver Assembly, Catalyst 4                       | 1        |
| 1-7, 8b, 9-15, 16b, 17b, 18-21, 22a, 23-31         | 116233      | Height Adjustable Tall Flip Back T-Arm Long Full with Standard Pad and Receiver Assembly, Catalyst 4                       | 1        |
| 1-7, 8a, 9-15, 16a, 17a, 18-21, 22b, 24-26, 30, 31 | 116226      | Height Adjustable Tall Flip Back T-Arm Short Desk with Standard Pad Assembly, Catalyst 4                                   | 1        |
| 1-7, 8b, 9-15, 16a, 17a, 18-21, 22a, 24-26, 30, 31 | 116227      | Height Adjustable Tall Flip Back T-Arm Short Full with Standard Pad Assembly, Catalyst 4                                   | 1        |
| 1-7, 8a, 9-15, 16b, 17b, 18-21, 22b, 24-26, 30, 31 | 116228      | Height Adjustable Tall Flip Back T-Arm Long Desk with Standard Pad Assembly, Catalyst 4                                    | 1        |
| 1-7, 8b, 9-15, 16b, 17b, 18-21, 22a, 24-26, 30, 31 | 116229      | Height Adjustable Tall Flip Back T-Arm Long Full with Standard Pad Assembly, Catalyst 4                                    | 1        |
| 1-7, 9-13, 15, 16a, 17a, 19, 21                    | 114152      | Height Adjustable Flip Back T-Arm Short Lower Assy, Tall<br><i>Does not include t-arm upper, arm pad or arm pad screws</i> | 1        |
| 1-7, 9-13, 15, 16b, 17b, 19, 21                    | 114153      | Height Adjustable Flip Back T-Arm Long Lower Assy, Tall<br><i>Does not include t-arm upper, arm pad or arm pad screws</i>  | 1        |
| 1-7, 8a, 9-15, 16a, 17a, 19, 21                    | 114436      | Height Adjustable Flip Back T-Arm Desk, Short, Tall<br><i>Does not include arm pad or arm pad screws</i>                   | 1        |
| 1-7, 8b, 9-15, 16a, 17a, 19, 21                    | 114437      | Height Adjustable Flip Back T-Arm Short, Full, Tall<br><i>Does not include arm pad or arm pad screws</i>                   | 1        |
| 1-7, 8a, 9-15, 16b, 17b, 19, 21                    | 114438      | Height Adjustable Flip Back T-Arm Long, Desk, Tall<br><i>Does not include arm pad or arm pad screws</i>                    | 1        |
| 1-7, 8b, 9-15, 16b, 17b, 19, 21                    | 114439      | Height Adjustable Flip Back T-Arm Long, Full, Tall<br><i>Does not include arm pad or arm pad screws</i>                    | 1        |
| 1-7, 9-13, 15, 16a, 17a, 18-21, 24, 25             | 505617      | Height Adjustable Flip Back T-Arm Short Retrofit Assembly, Catalyst 4                                                      | 1        |

| Item Number                            | Part Number | Description                                                                                                                                                                                                                   | Quantity |
|----------------------------------------|-------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------|
|                                        |             | <i>This kit is used to retrofit from a Height Adjustable T-Arm to a Height Adjustable Flip Back Tall T-Arm. This kit is to retrofit one side.</i>                                                                             |          |
| 1-7, 9-13, 15, 16b, 17b, 18-21, 24, 25 | 505618      | Height Adjustable Flip Back T-Arm Long Retrofit Assembly, Catalyst 4<br><br><i>This kit is used to retrofit from a Height Adjustable T-Arm to a Height Adjustable Flip Back Tall T-Arm. This kit is to retrofit one side.</i> | 1        |
| 18, 20, 24, 25                         | 112957      | Armrest Flip Back Height Adj T-Arm Hinge Assembly, PAIR, Catalyst 4<br><br><i>This kit is for a pair of hinge assemblies. A pair is required as a replacement as of May 19th, 2021</i>                                        | 1        |
| 3, 5, 6, 9, 11, 13                     | 504560      | Height Adjustable Flip Back T-Arm Latch Assembly                                                                                                                                                                              | 1        |
| 4, 9, 10, 12                           | 504561      | Height Adjustable Flip Back T-Arm Release Lever Assembly                                                                                                                                                                      | 1        |
| 23, 27, 28, 29                         | 501730      | Height Adj T-Arm Receiver Assm 1" CAT                                                                                                                                                                                         | 1        |
| 30, 31                                 | 504577      | Height Adjustable Flip Back T-Arm Mounting Hardware Assembly<br><br><i>This kit contains various mounting hardware for multiple configurations</i>                                                                            | 1        |
| 1                                      | 115983      | Height Adjustable Flip Back T-Arm Post Tall                                                                                                                                                                                   | 1        |
| 2                                      | 113138      | Height Adjustable T-Arm Transfer Tube, Tall                                                                                                                                                                                   | 1        |
| 3                                      | 000096      | Height Adjustable T Armrest Pin Lock Housing                                                                                                                                                                                  | 1        |
| 4                                      | 003419      | Armrest Release Lever                                                                                                                                                                                                         | 1        |
| 5                                      | 000099      | Height Adjustable Armrest Lever Pin                                                                                                                                                                                           | 1        |
| 6                                      | 003418      | Armrest Height Adjustment Lever                                                                                                                                                                                               | 1        |
| 7                                      | 000090      | Height Adjustable T Armrest Shim                                                                                                                                                                                              | 2        |
| 8a                                     | 000091      | Height Adj T Armrest Upper Desk                                                                                                                                                                                               | 1        |
| 8b                                     | 000101      | Height Adj T Armrest Upper Full                                                                                                                                                                                               | 1        |
| 8a, 14                                 | 116379      | Height Adjustable Tall T Armrest Upper Desk w/ Tube End Plug                                                                                                                                                                  | 1        |
| 8b, 14                                 | 116380      | Height Adjustable Tall T Armrest Upper Full w/ Tube End Plug                                                                                                                                                                  | 1        |
| 9                                      | 003458      | M5 X 12mm FHCS BLK ZN W/ Patch                                                                                                                                                                                                | 2        |
| 10                                     | 000152      | Height Adjustable T Armrest Release Spring Button                                                                                                                                                                             | 1        |
| 11                                     | 000134      | Height Adjustable Armrest Lever Spring                                                                                                                                                                                        | 1        |
| 12                                     | 000135      | Height Adjustable T Armrest Release Lever Spring                                                                                                                                                                              | 1        |
| 13                                     | 200257      | Dowel Pin 1/8 x 1/2                                                                                                                                                                                                           | 1        |
| 14                                     | 100803      | Tube End Plug 7/8"                                                                                                                                                                                                            | 2        |

| Item Number | Part Number | Description                                              | Quantity |
|-------------|-------------|----------------------------------------------------------|----------|
| 15, 16a, 21 | 113143      | Height Adjustable Flip Back T-Arm Short Shield, Tall     | 1        |
| 15, 16b, 21 | 113142      | Height Adjustable Flip Back T-Arm Long Side Shield, Tall | 1        |
| 17a, 19     | 504564      | Height Adjustable Flip Back T-Arm Metal Insert Short     | 1        |
| 17b, 19     | 504565      | Height Adjustable Flip Back T-Arm Metal Insert Long      | 1        |
| 22a         | 100500      | Arm Pad Full                                             | 1        |
| 22b         | 100501      | Arm Pad Desk                                             | 1        |
| 23          | 122055      | Binding Post, Flat Head, M6, .38"                        | 2        |
| 26          | 100696      | M5 x 30mm PPH Capscrew                                   | 2        |
| 27          | 122222      | Height Adjustable T Armrest Bracket, CAT/SPARK           | 1        |
| 28          | 100667      | M6 x 12 BTN SHCS BLKZC w/Patch                           | 2        |
| 29          | 122221      | T-Arm Receiver 1", Cat                                   | 1        |
| 30          | 100658      | M6 Nylock Nut Blk Zn                                     | 2        |
| 31          | 100544      | M6 x 75 CLS 8.8 DIN931 HHCS ZC                           | 1        |