


Spark Height Adjustable T-Arm

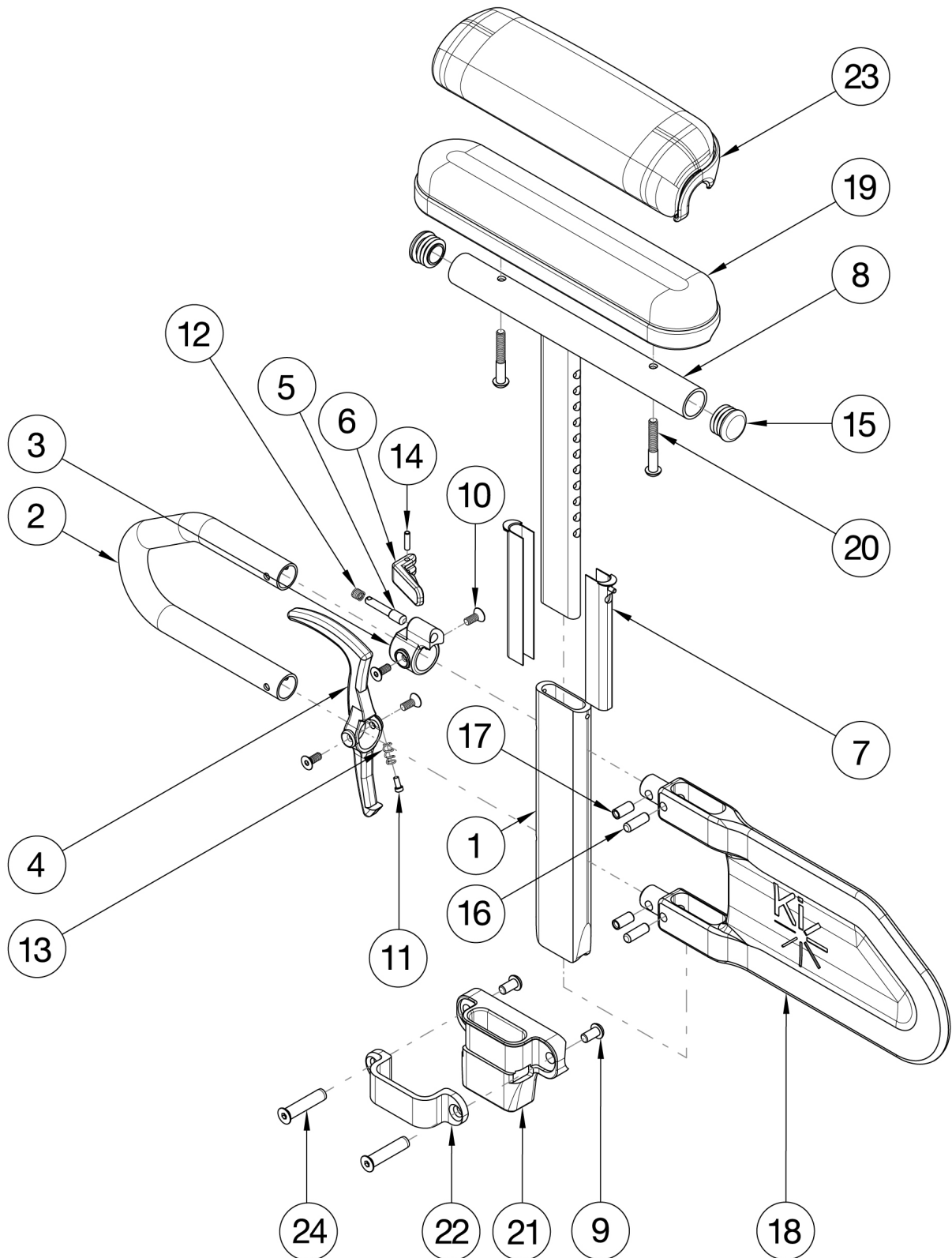

Short assemblies are used with 14-17 Seat Depths. Long assemblies are used with 18"-19" Seat Depths This design effective November 10, 2017 for long panels and December 6, 2017 for short panels If retrofitting to Height Adjustable T-Arms, please note the following: * New rear frames are required * Receivers are needed * If One Arm Drive is present, a new drive shaft may be required. In addition to a new drive shaft, order the corresponding hardware kit: 109439 for Blade Style One Arm Drive, 503433 for Hex Style One Arm Drive or 112898 for Quick Release Style One Arm Drive

Item Number	Part Number	Description	Quantity
1	000089	Height Adjustable T Armrest Post	1
2	000095	Height Adjustable T Armrest Transfer Tube	1
3, 5, 6, 10, 12, 14	505490	Height Adjustable T-Arm Upper Latch Assembly	1
7	000090	Height Adjustable T Armrest Shim	2
8a, 15	112906	Height Adjustable T Armrest Upper Desk w/ Tube End Plug	1
8b, 15	112907	Height Adjustable T Armrest Upper Full w/ Tube End Plug	1
9	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2
10	003458	M5 X 12mm FHCS BLK ZN W/ Patch	4
11	000152	Height Adjustable T Armrest Release Spring Button	1
13	000135	Height Adjustable T Armrest Release Lever Spring	1
15	100803	Tube End Plug 7/8"	2
16	003641	.25" x .75" Dowel Pin	2
17	003457	M5 x 14mm Barrel Nut	2
19a	100500	Arm Pad Full	1
19b	100501	Arm Pad Desk	1
20a	100696	M5 x 30mm PPH Capscrew	2
20b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC	2
22	122222	Height Adjustable T Armrest Bracket, CAT/SPARK	1
23a	102452	Waterfall Arm Pad Full	1
23b	102521	Waterfall Arm Pad Desk	1
24	122055	Binding Post, Flat Head, M6, .38"	2
4, 10, 11, 13	505507	Height Adjustable T-Arm Release Lever Assembly	1
16, 17, 18a	505508	Height Adj T-Arm Long Side Shield Assembly	1
16, 17, 18b	506082	Height Adjustable T-Arm Short Shield Assembly	1

Item Number	Part Number	Description	Quantity
9, 21, 22, 24, 25	505276	Height Adjustable T-Arm Receiver Assembly 1" Spark	1
1-7, 10-14, 16, 17, 18b	004160	Height Adjustable T-Arm Lower Assembly Short	1
1-7, 10-14, 16, 17, 18a	004161	Height Adjustable T-Arm Lower Assembly Long	1
1-7, 8a, 10-18a	003465	Height Adjustable T-Arm Long Desk	1
1-7, 8b, 10-18a	003466	Height Adjustable T-Arm Long Full	1
1-7, 8a, 10-18b	003463	Height Adjustable T-Arm Short Desk	1
1-7, 8b, 10-18b	003464	Height Adjustable T-Arm Short Full	1
1-7, 8a, 10-18a, 19a, 20a	505491	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full	1
1-7, 8a, 10-18a, 19a, 20a	505492	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full, PAIR	1
1-7, 8a, 10-18b, 19a, 20a	505644	Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Full	1
1-7, 8a, 10-18b, 19a, 20a	505645	Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Full, Pair	1
1-7, 8a, 10-18a, 19b, 20a	505493	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk	1
1-7, 8a, 10-18a, 19b, 20a	505494	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk, PAIR	1
1-7, 8a, 10-18b, 19b, 20a	505646	Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Desk	1
1-7, 8a, 10-18b, 19b, 20a	505647	Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Desk, Pair	1
1-7, 8a, 10-18a, 20b, 23a	505495	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full	1
1-7, 8a, 10-18a, 20b, 23a	505496	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full, PAIR	1
1-7, 8a, 10-18b, 20b, 23a	505648	Height Adjustable T-Arm Assembly w/ Short Side Shield and Waterfall Pad Full	1
1-7, 8a, 10-18b, 20b, 23a	505649	Height Adjustable T-Arm Assembly w/ Short Side Shield and Waterfall Pad Full, Pair	1
1-7, 8a, 10-18a, 20b, 23b	505497	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk	1
1-7, 8a, 10-18a, 20b, 23b	505498	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk, PAIR	1
1-7, 8a, 10-18b, 20b, 23b	505650	Height Adjustable T-Arm Assembly w/ Short Side Shield and Waterfall Pad Desk	1
1-7, 8a, 10-18b, 20b, 23b	505651	Height Adjustable T-Arm Assembly w/ Short Side Shield and Waterfall Pad Desk, Pair	1
1-7, 8a, 9-18a, 19a, 20a, 21, 22, 24	505509	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full, 1" Receiver Spark	1
1-7, 8a, 9-18a, 19a, 20a, 21, 22, 24	505510	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full, 1" Receiver, PAIR Spark	1

Item Number	Part Number	Description	Quantity
1-7, 8a, 9-18b, 19a, 20a, 21, 22, 24	506057	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, 1" Receiver Spark	1
1-7, 8a, 9-18b, 19a, 20a, 21, 22, 24	506058	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, 1" Receiver, PAIR Spark	1
1-7, 8a, 9-18a, 19b, 20a, 21, 22, 24	505511	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk, 1" Receiver Spark	1
1-7, 8a, 9-18a, 19b, 20a, 21, 22, 24	505512	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk, 1" Receiver, PAIR Spark	1
1-7, 8a, 9-18b, 19b, 20a, 21, 22, 24	506059	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, 1" Receiver Spark	1
1-7, 8a, 9-18b, 19b, 20a, 21, 22, 24	506060	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, 1" Receiver, PAIR Spark	1
1-7, 8a, 9-18a, 20b, 21, 22, 23a, 24	505513	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full, 1" Receiver Spark	1
1-7, 8a, 9-18a, 20b, 21, 22, 23a, 24	505514	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full, 1" Receiver, PAIR Spark	1
1-7, 8a, 9-18b, 20b, 21, 22, 23a, 24	506061	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, 1" Receiver Spark	1
1-7, 8a, 9-18b, 20b, 21, 22, 23a, 24	506062	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, 1" Receiver, PAIR Spark	1
1-7, 8a, 9-18a, 20b, 21, 22, 23b, 24	505515	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk, 1" Receiver Spark	1
1-7, 8a, 9-18a, 20b, 21, 22, 23b, 24	505516	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk, 1" Receiver, PAIR Spark	1
1-7, 8a, 9-18b, 20b, 21, 22, 23b, 24	506063	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, 1" Receiver Spark	1
1-7, 8a, 9-18b, 20b, 21, 22, 23b, 24	506064	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, 1" Receiver, PAIR Spark	1