


(Discontinued) Catalyst 5 Standard Caster Housing for Fixed Front

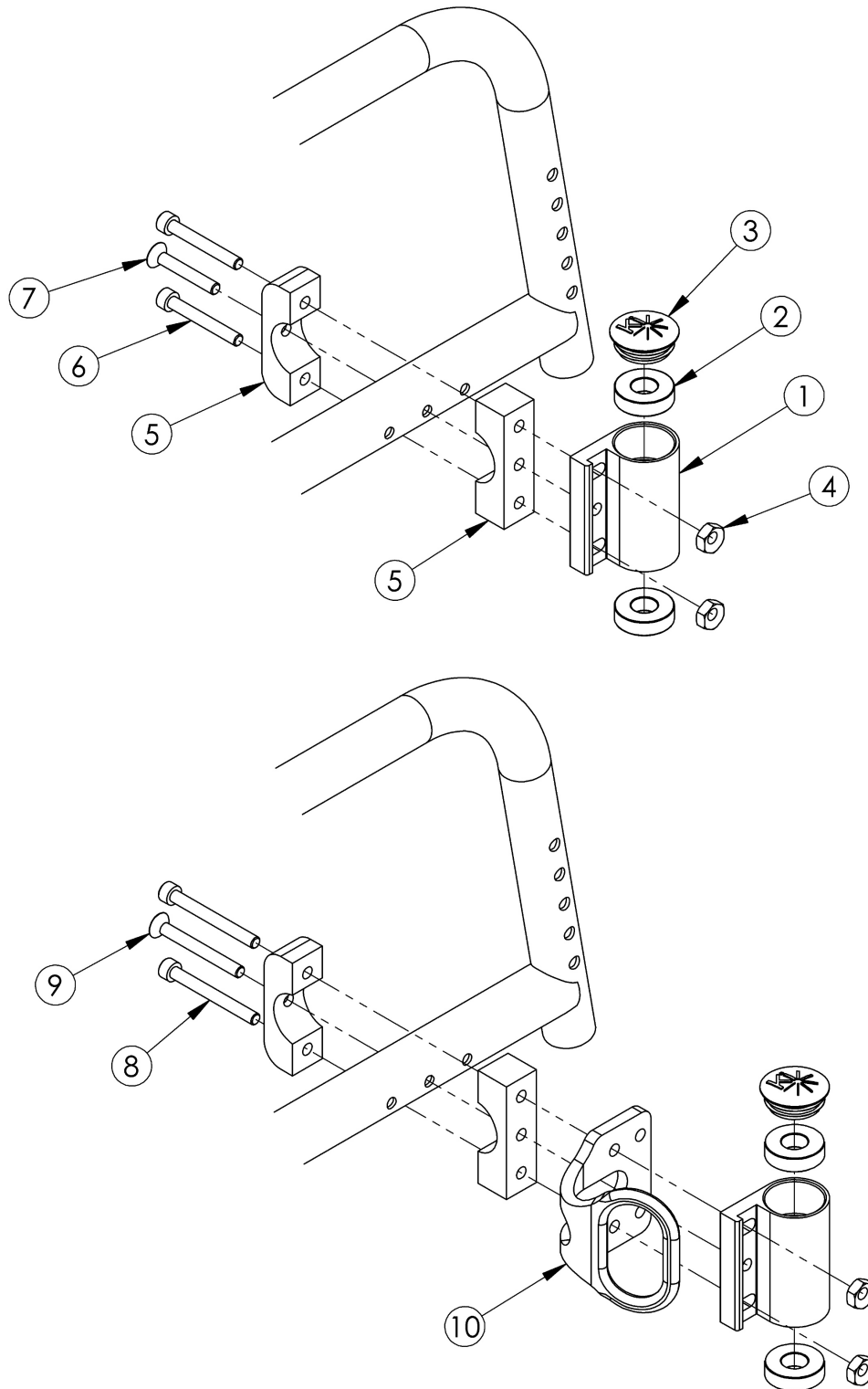

<p>This design discontinued July 12th, 2021<p>

Top configuration pictured used on chairs without transit option; bottom configuration used on chairs with transit option.

Item Number	Part Number	Description	Quantity	Retail
1a, 2-7	502636	Caster Housing Assembly, Black, Fixed Front, Left, Catalyst	1	\$93.06
1b, 2-7	502636_ _ _	Color Anodized Caster Housing Assembly, Fixed Front, Left, Catalyst	1	\$161.35
1c, 2-7	502637	Caster Housing Assembly, Black, Fixed Front, Right, Catalyst	1	\$93.06
1d, 2-7	502637_ _ _	Color Anodized Caster Housing Assembly, Fixed Front, Right, Catalyst	1	\$161.35
1a, 1c, 2-7	502638	Caster Housing Assembly, Black, Fixed Front, PAIR, Catalyst	1	\$186.12
1b, 1d, 2-7	502638_ _ _	Color Anodized Caster Housing Assembly, Fixed Front, PAIR, Catalyst	1	\$322.70
1a, 2-5, 8, 9	502639	Caster Housing Assembly, Black, Fixed Front, with Transit, Left, Catalyst <i>Transit Bracket not included in assembly</i>	1	\$94.32
1b, 2-5, 8, 9	502639_ _ _	Color Anodized Caster Housing Assembly, Fixed Front, with Transit, Left, Catalyst <i>Transit Bracket not included in assembly</i>	1	\$162.61
1c, 2-5, 8, 9	502640	Caster Housing Assembly, Black, Fixed Front, with Transit, Right, Catalyst <i>Transit Bracket not included in assembly</i>	1	\$94.32
1d, 2-5, 8, 9	502640_ _ _	Color Anodized Caster Housing Assembly, Fixed Front, with Transit, Right, Catalyst <i>Transit Bracket not included in assembly</i>	1	\$162.61
1a, 1c, 2-5, 8, 9	502641	Caster Housing Assembly, Black, Fixed Front, with Transit, PAIR, Catalyst <i>Transit Bracket not included in assembly</i>	1	\$188.65
1b, 1d, 2-5, 8, 9	502641_ _ _	Color Anodized Caster Housing Assembly, Fixed Front, with Transit, PAIR, Catalyst <i>Transit Bracket not included in assembly</i>	1	\$325.23
1a, 2	001429	Caster Housing Fixed Front Left, Black, Catalyst	1	\$37.66
1b, 2	001429_ _ _	Color Anodized Caster Housing Fixed Front, Left, Catalyst	1	\$105.95
1c, 2	001430	Caster Housing Fixed Front Right, Black, Catalyst	1	\$37.66
1d, 2	001430_ _ _	Color Anodized Caster Housing Fixed Front, Right, Catalyst	1	\$105.95
2	100530	Bearing M28 x 1/2" 6001RS	2	\$12.02
3	102005	Ki Caster Housing Cap	1	\$1.17
4	100745	Eccentric nut (threaded)	2	\$1.33
5	100528	Caster Housing Clamp	1	\$23.47
6	101472	M6 x 45 SHCS BLK ZC	2	\$1.58
7	101129	M6 x 50 FSHCS P/THD BLKZC	1	\$2.10

Item Number	Part Number	Description	Quantity	Retail
8	100739	M6 X 55 SHCS CLS 12.96 DIN912 BLKZC	2	\$2.10
9	101130	M6 x 55 FSHCS P/THD BLKZC	1	\$2.10

Color Anodizing Samples

