


Liberty Seat Upholstery

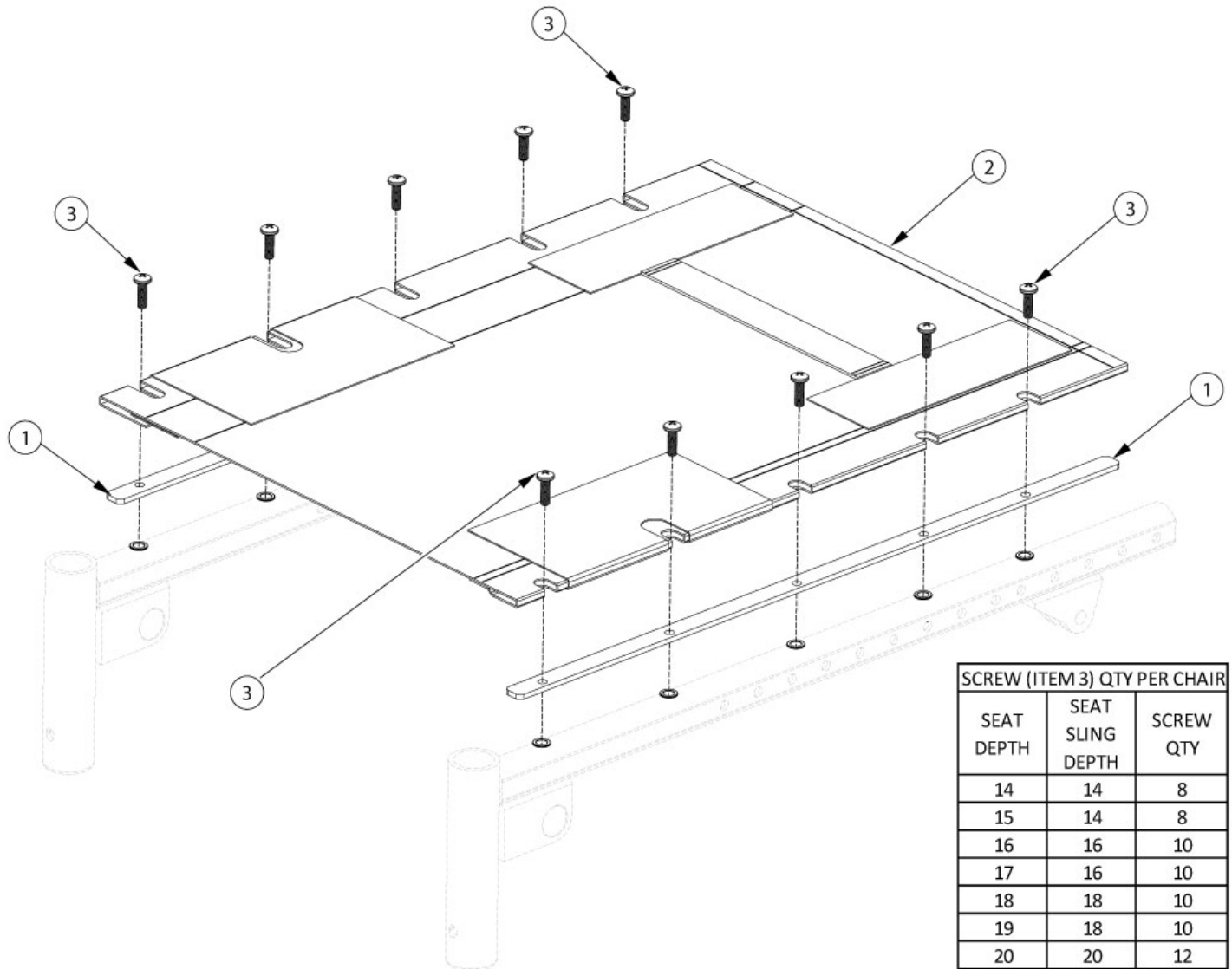

SCREW (ITEM 3) QTY PER CHAIR		
SEAT DEPTH	SEAT SLING DEPTH	SCREW QTY
14	14	8
15	14	8
16	16	10
17	16	10
18	18	10
19	18	10
20	20	12

Seat upholstery width measures approximately 1" less than the chair size. Seat upholstery depth measures true to the chair size.

Item Number	Part Number	Description	Quantity
1a	000241	Seat Rail 14" Deep Rigid/Liberty <i>Used on 14" and 15" Seat Depths</i>	2
1b	000243	Seat Rail 16" Deep Rigid/Liberty <i>Used on 16" and 17" Seat Depths</i>	2
1c	000245	Seat Rail 18" Deep Rigid/Liberty <i>Used on 18" and 19" Seat Depths</i>	2
1d	000247	Seat Rail 20" Deep Rigid/Liberty <i>Used on 20" Seat Depth</i>	2

Item Number	Part Number	Description	Quantity
2a	101000	Seat Upholstery Rigid/Liberty 1414 <i>Used on 14" and 15" Seat Depths</i>	1
2b	101012	Seat Upholstery Rigid/Liberty 1416 <i>Used on 16" and 17" Seat Depths</i>	1
2c	101026	Seat Upholstery Rigid/Liberty 1418 <i>Used on 18" and 19" Seat Depths</i>	1
2d	101074	Seat Upholstery Rigid/Liberty 1420 <i>Used on 20" Seat Depth</i>	1
2e	101001	Seat Upholstery Rigid/Liberty 1514 <i>Used on 14" and 15" Seat Depths</i>	1
2f	101013	Seat Upholstery Rigid/Liberty 1516 <i>Used on 16" and 17" Seat Depths</i>	1
2g	101027	Seat Upholstery Rigid/Liberty 1518 <i>Used on 18" and 19" Seat Depths</i>	1
2h	101040	Seat Upholstery Rigid/Liberty 1520 <i>Used on 20" Seat Depth</i>	1
2i	101002	Seat Upholstery Rigid/Liberty 1614 <i>Used on 14" and 15" Seat Depths</i>	1
2j	101014	Seat Upholstery Rigid/Liberty 1616 <i>Used on 16" and 17" Seat Depths</i>	1
2k	101028	Seat Upholstery Rigid/Liberty 1618 <i>Used on 18" and 19" Seat Depths</i>	1
2l	101041	Seat Upholstery Rigid/Liberty 1620 <i>Used on 20" Seat Depth</i>	1
2m	101003	Seat Upholstery Rigid/Liberty 1714 <i>Used on 14" and 15" Seat Depths</i>	1
2n	101015	Seat Upholstery Rigid/Liberty 1716 <i>Used on 16" and 17" Seat Depths</i>	1
2o	101029	Seat Upholstery Rigid/Liberty 1718 <i>Used on 18" and 19" Seat Depths</i>	1
2p	101042	Seat Upholstery Rigid/Liberty 1720 <i>Used on 20" Seat Depth</i>	1
2q	101004	Seat Upholstery Rigid/Liberty 1814 <i>Used on 14" and 15" Seat Depths</i>	1
2r	101016	Seat Upholstery Rigid/Liberty 1816 <i>Used on 16" and 17" Seat Depths</i>	1

Item Number	Part Number	Description	Quantity
2s	101030	Seat Upholstery Rigid/Liberty 1818 <i>Used on 18" and 19" Seat Depths</i>	1
2t	101043	Seat Upholstery Rigid/Liberty 1820 <i>Used on 20" Seat Depth</i>	1
2u	102534	Seat Upholstery Rigid/Liberty 1914 <i>Used on 14" and 15" Seat Depths</i>	1
2v	101017	Seat Upholstery Rigid/Liberty 1916 <i>Used on 16" and 17" Seat Depths</i>	1
2w	101031	Seat Upholstery Rigid/Liberty 1918 <i>Used on 18" and 19" Seat Depths</i>	1
2x	101044	Seat Upholstery Rigid/Liberty 1920 <i>Used on 20" Seat Depth</i>	1
2y	101101	Seat Upholstery Rigid/Liberty 2014 <i>Used on 14" and 15" Seat Depths</i>	1
2z	101018	Seat Upholstery Rigid/Liberty 2016 <i>Used on 16" and 17" Seat Depths</i>	1
2aa	101032	Seat Upholstery Rigid/Liberty 2018 <i>Used on 18" and 19" Seat Depths</i>	1
2ab	101045	Seat Upholstery Rigid/Liberty 2020 <i>Used on 20" Seat Depth</i>	1
2ac	102535	Seat Upholstery Rigid/Liberty 2114 <i>Used on 14" and 15" Seat Depths</i>	1
2ad	101528	Seat Upholstery Rigid/Liberty 2116 <i>Used on 16" and 17" Seat Depths</i>	1
2ae	101530	Seat Upholstery Rigid/Liberty 2118 <i>Used on 18" and 19" Seat Depths</i>	1
2af	101532	Seat Upholstery Rigid/Liberty 2120 <i>Used on 20" Seat Depth</i>	1
2ag	101723	Seat Upholstery Rigid/Liberty 2214 <i>Used on 14" and 15" Seat Depths</i>	1
2ah	101535	Seat Upholstery Rigid/Liberty 2216 <i>Used on 16" and 17" Seat Depths</i>	1
2ai	101537	Seat Upholstery Rigid/Liberty 2218 <i>Used on 18" and 19" Seat Depths</i>	1
2aj	101539	Seat Upholstery Rigid/Liberty 2220 <i>Used on 20" Seat Depth</i>	1

Item Number	Part Number	Description	Quantity
3	100734	M5 x 16 PHMS ZC	12