


Rogue Style Transit Backrest Option on Tsunami

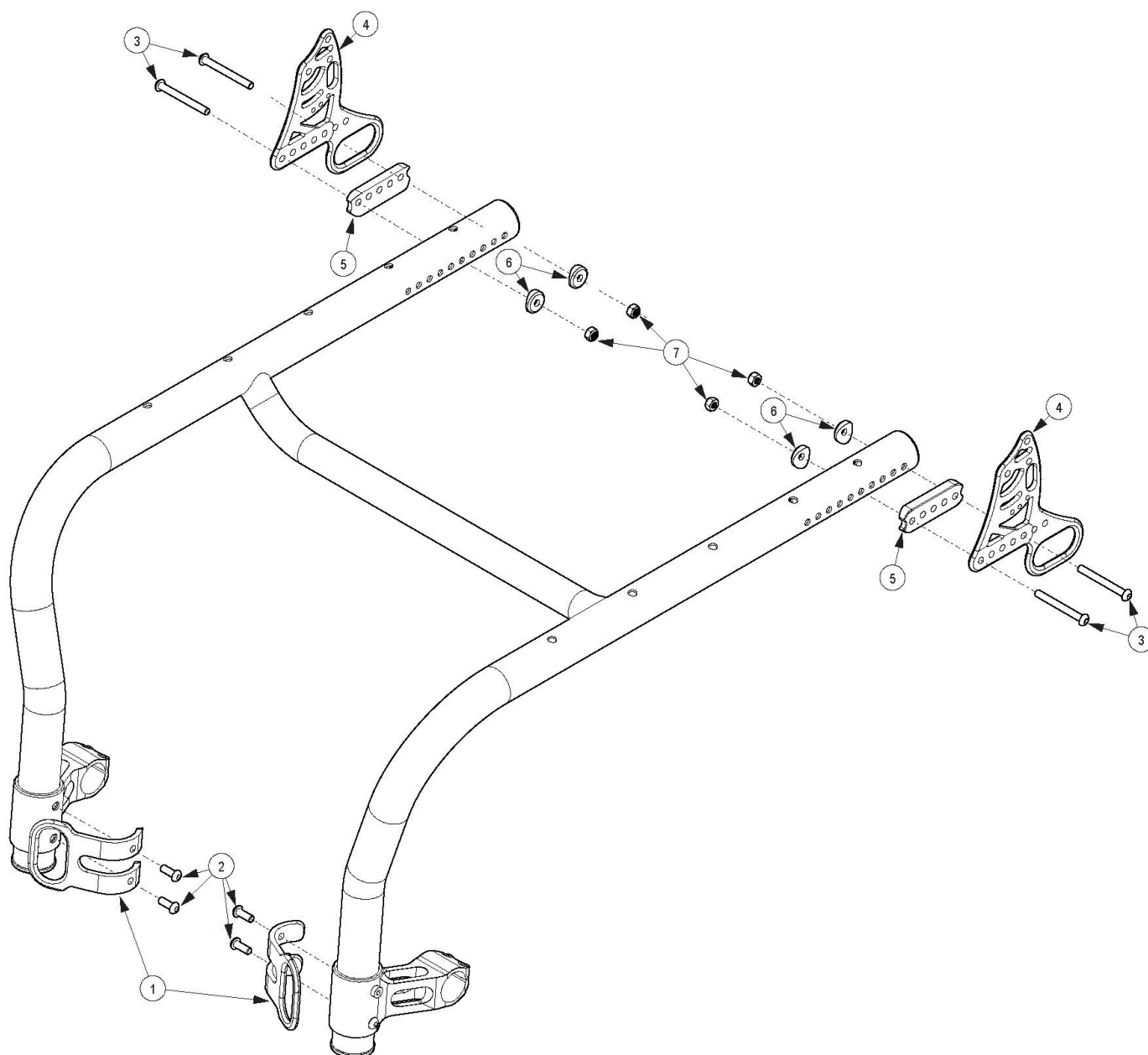

This Transit setup is used with the Rogue Style Locking Backrest <p>

For all backrest hardware options, please see Rogue Style Locking Backrest Mount and Hardware on Tsunami page <p>

Not available with Heavy Duty Option

Item Number	Part Number	Description	Quantity	Retail
1 - 7	109635	Tsunami Transit Bracket Retrofit Kit for Rogue Style Backrest with Standard Caster Mounts <i>Not available with Heavy Duty Option. This kit is a complete retrofit, containing a pair of front and rear transit brackets and hardware.</i>	1	\$371.58
2	101708	M6 x 14 BTN SHCS BLKZC w/Patch	4	\$1.58

Item Number	Part Number	Description	Quantity	Retail
3a	003232	M6 x 55 BHCS Blk Zc <i>Used when bolt goes through the tower and frame only. Also used when bolt goes through the frame, back bracket and back saddles.</i>	2	\$8.94
3b	107138	M6 x 65, BHCS, P/THD BLZ <i>Used when bolt goes through the tower, frame, back bracket and back saddle</i>	4	\$2.04
3c	101781	M6 x 50mm Partial Thread BHCS Blk ZC <i>Used when bolt goes through the frame, back bracket and back saddle</i>	4	\$1.58
4	109564	Back Bracket with Transit and Hook Label, Black, Rigid	2	\$26.72
5	003715	Back Saddle 1 3/8" Tube	2	\$2.43
6	003716	Saddle 1 3/8" Tube	4	\$1.05
7	100658	M6 Nylock Nut Blk Zn	6	\$1.05