


Tyke Seat Frame

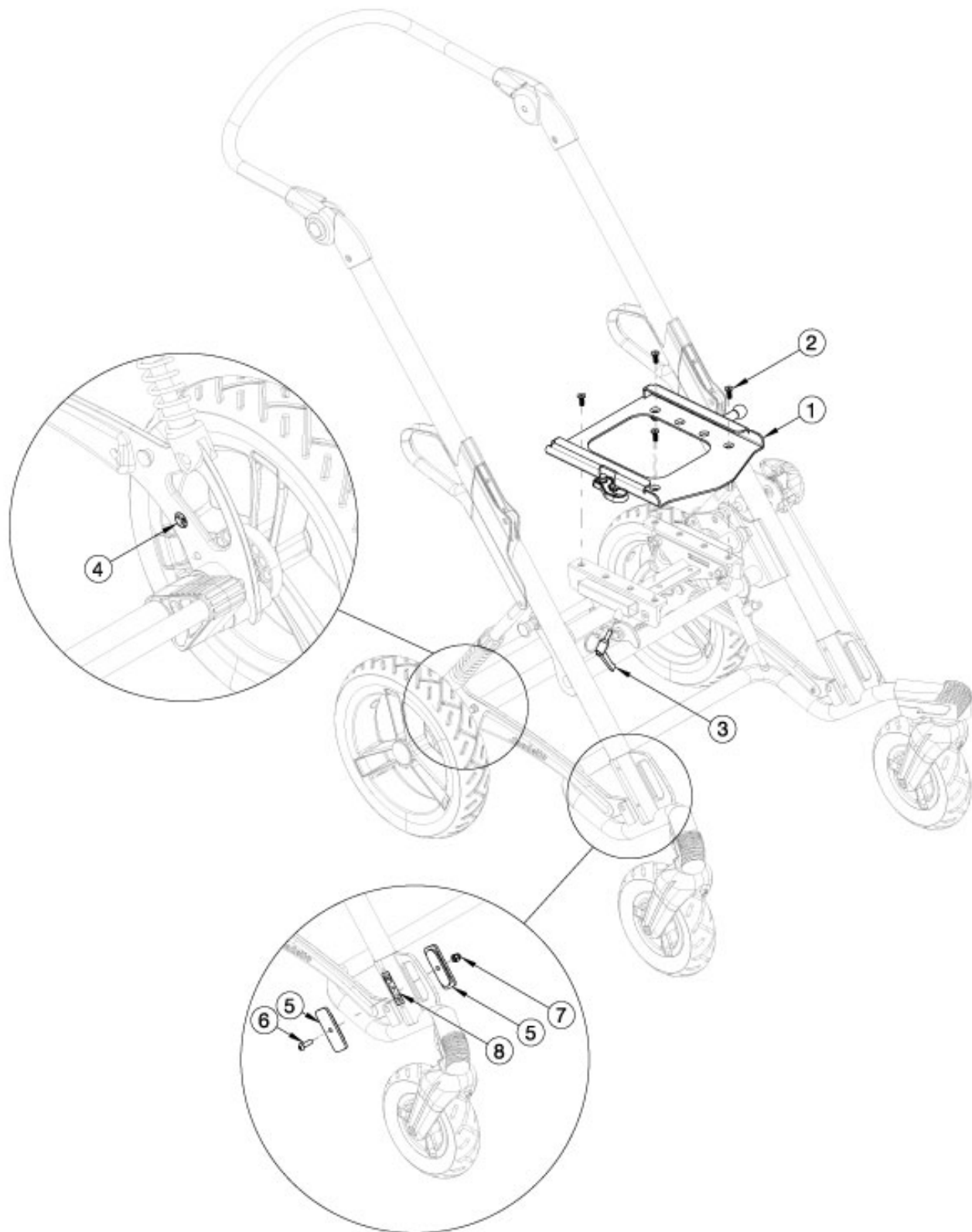

Reverse Configuration Only

Item Number	Part Number	Description	Quantity
	120801	Reverse to Forward Seating, Tyke for x:panda	1
	120802	Forward to Reverse Seating, Tyke for x:panda	1
1	116699	Interface Plate, x:panda, Tyke/Flip	1
2	101594	M6 x 14 CLS10.9 DIN7991 FSHCS BLKZC w/Patch	4
3	102734	Stroller Handle Clamp Handle (16mm) Rigid/Flip/Tyke	1
5	120643	Cover, Transit, x:panda	2
6	102623	M6 x 20 BHCS BLK ZC	1
7	100558	M6 NYL INS Domed Cap L/N ZC	1