


Spark Side Frame Assembly

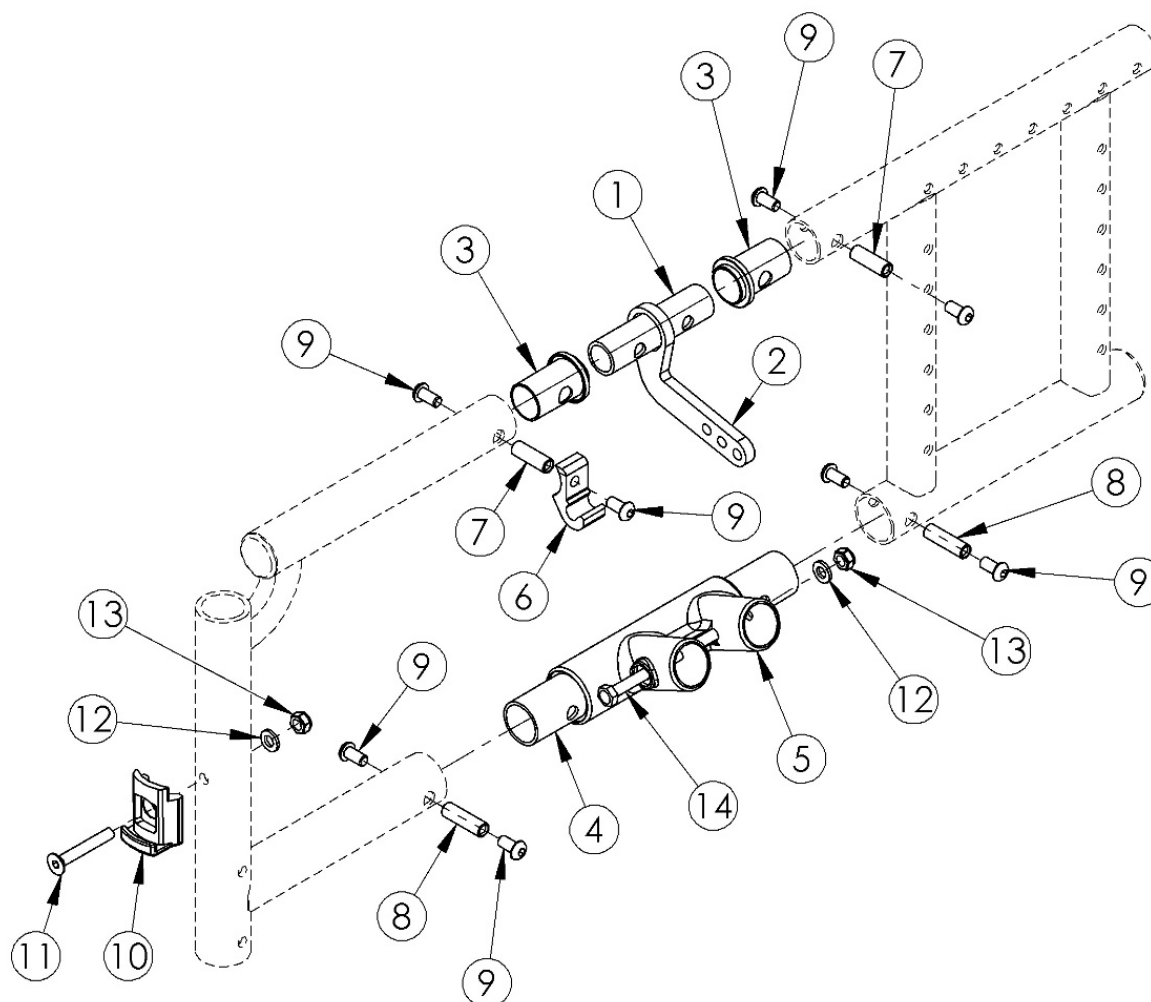

Item Number	Part Number	Description	Quantity	Retail
1	000628	Upper Pivot Tube SPARK	1	\$13.91
2a	000641	Cross Tube Hinge 10-11" Wide SPARK	1	\$44.68
2b	000642	Cross Tube Hinge 11-13" Wide SPARK	1	\$44.68
2c	000643	Cross Tube Hinge 12-14" Wide SPARK	1	\$44.68
2d	000644	Cross Tube Hinge 13-15" Wide SPARK	1	\$44.68
2e	000645	Cross Tube Hinge 14-16" Wide SPARK	1	\$44.68
3	000627	Cross Tube Hinge Sleeve SPARK	2	\$4.02
4	000009	Frame Connector Lower CAT	1	\$4.58

Item Number	Part Number	Description	Quantity	Retail
5	000001	Lower Frame Pivot CAT	1	\$27.71
6	000626	Spark X-Tube Hook	1	\$28.43
7	000065	M6 x 23.68 Threaded Barrel	2	\$4.13
8	000066	M6 x 27 Threaded Barrel	2	\$4.46
9	100667	M6 x 12 BTN SHCS BLKZC w/Patch	8	\$1.58
10	000336	Latch Block	1	\$37.52
11	101819	M6 x 40 FHCS Blk Zn	1	\$1.05
12	100746	M6 Flat Washer Blk Zn	2	\$1.58
13	100658	M6 Nylock Nut Blk Zn	2	\$1.05
14	100704	M6 x 90 CLS8.8 DIN931 HHCS ZC	1	\$1.58