

INSTRUCTION SHEET

JAY 3 VANITY FLAP INSTALLATION

(Please read and understand instructions before beginning installation)

Tools Required:

None

Install:

1. Lift upper flap of J3 Back Cover (Fig. 1)

2. Match hook to loop and attach Vanity Flap to the back of the J3 Back Cover (Fig. 1). Reposition upper flap of cover to sandwich the Vanity Flap between the flaps the upper and lower flaps (Fig. 3).

Fig. 1



Fig. 2




Fig. 3



3. Remove Seat Cushion from wheelchair and sandwich “tail” of Vanity Flap between wheelchair support surface and Seat Cushion (Fig 4).

4. Adjust Vanity Flap as necessary by moving the tail in or out from under the Seat Cushion. If necessary the upper part of the vanity flap can be moved up or down on the Seat Back also.

5. Insert Hook & Loop Wrap ties through the appropriate grommets on the Vanity Flap, then wrap around the wheelchair frame and insert the leading end through the slot end of the Wrap (Fig. 5). Snug up and close outside gaps in Vanity Flap (Fig.6).

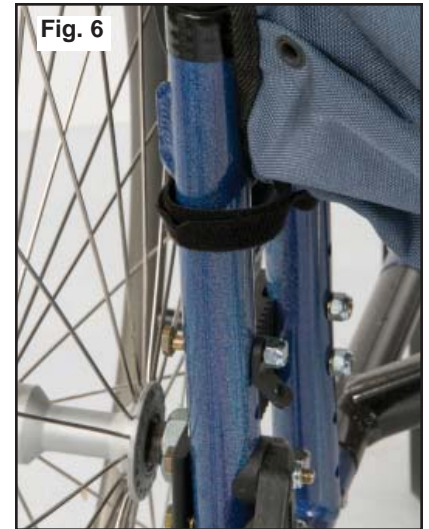

Finished J3 Vanity Flap Installation.

