


Focus CR / Liberty Removable Stroller Handle

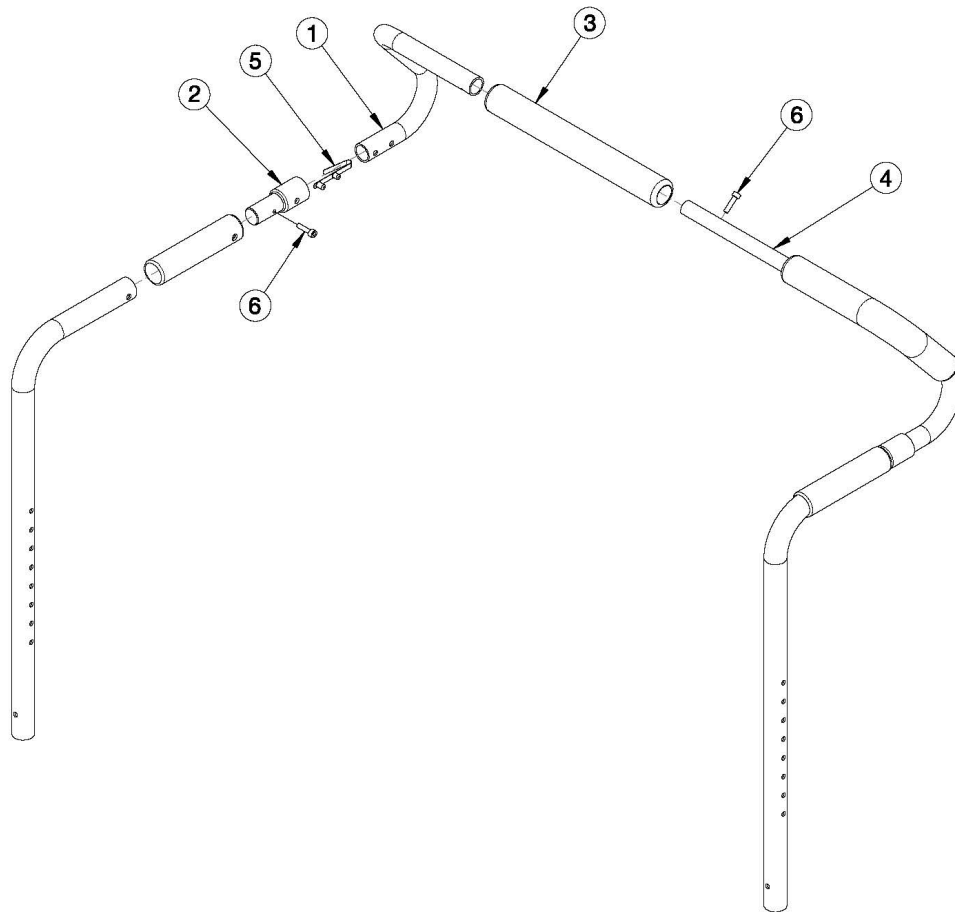

Part No.	Growth Range	Used on Widths	Length
002578	14"-20"	14"-17"	8.75"
002579	16"-22"	18"-22"	10.75"
112003	18"-24"	23"-24"	12.75"

Assemblies and Growth Strut based on Seat Width. If retrofitting, new back canes will be required.

Item Number	Part Number	Description	Quantity
1a, 1b, 2, 3, 4a, 5, 6	503104	Removable Stroller Handle Assembly 14-18W Focus/CR45/CR45/Flip/Liberty	1
1a, 1b, 2, 3, 4b, 5, 6	503105	Removable Stroller Handle Assembly 18-22W Focus/CR45/CR45/Liberty	1
1a, 1b, 2, 3, 4c, 5, 6	114288	Removable Stroller Handle Assm 23-24W Focus	1
1a, 3	501090	Stroller Handle with Foam Grip, Right	1
1b, 3	501091	Stroller Handle with Foam Grip, Left	1
2	001243	Stroller Handle Adaptor	2
4a	002578	Growth Strut, Stroller, Short	1
4b	002579	Growth Strut, Stroller, Long	1
4c	112003	Growth Strut, Stroller, Extra Long	1

Item Number	Part Number	Description	Quantity
5	100533	Detent Button	2
6	101843	M5 x 20 SHCS Blk Zn	4