


Rogue2 / Rogue XP Rear Anti-Tip

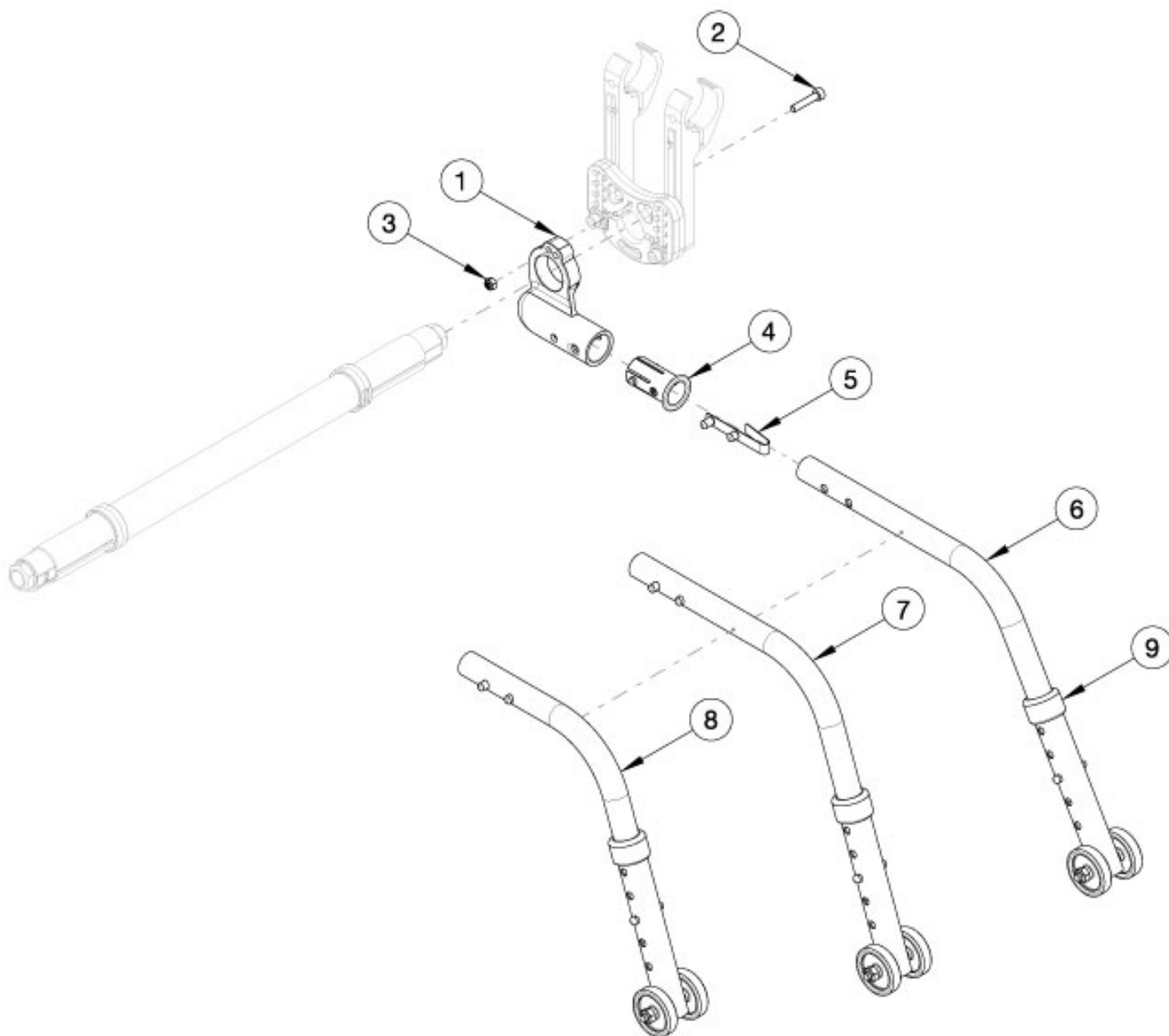

If retrofitting, Anti-Tip Receivers will be needed. Used on Rogue XP chairs invoiced starting on 3/12/25 IF the Alber Emotion/Twion Systems is present on a Roue XP, visit the Alber - Emotion / Twion Anti-Tips branch

Item Number	Part Number	Description	Quantity
1a, 4	117465	Standard Anti Tip Receiver with Sleeve, Right Side, Rogue2 / Rogue XP	1
1b, 4	117466	Standard Anti Tip Receiver with Sleeve, Left Side, Rogue2 / Rogue XP	1
5, 8, 9	119669	Anti Tip Medium 22/24CAT 18/20 Rogue-Clík-LW 22/24/25ETHOS <i>Replaces 100226 and Kit 117467</i>	2

Item Number	Part Number	Description	Quantity
5, 7, 9	117468	Standard Anti Tip Assembly, 22" Wheels, Rogue2 / Rogue XP	2
5, 6, 9	117469	Standard Anti Tip Assembly, 24-26" Wheels, Rogue2 / Rogue XP	2
1a, 2-5, 8, 9	117470	Standard Anti Tip Receiver Complete Assembly, 18-20" Wheels, Right Side, Rogue2 / Rogue XP	1
1b, 2-5, 8, 9	117471	Standard Anti Tip Receiver Complete Assembly, 18-20" Wheels, Left Side, Rogue2 / Rogue XP	1
1a, 2-5, 7, 9	117472	Standard Anti Tip Receiver Complete Assembly, 22" Wheels, Right Side, Rogue2 / Rogue XP	1
1b, 2-5, 7, 9	117473	Standard Anti Tip Receiver Complete Assembly, 22" Wheels, Left Side, Rogue2 / Rogue XP	1
1a, 2-6, 9	117474	Standard Anti Tip Receiver Complete Assembly, 24-26" Wheels, Right Side, Rogue2 / Rogue XP	1
1b, 2-6, 9	117475	Standard Anti Tip Receiver Complete Assembly, 24-26" Wheels, Left Side, Rogue2 / Rogue XP	1
1a	113997	Rogue Standard Tipper Mount, Right	1
1b	114001	Rogue Standard Tipper Mount, Left	1
2	116550	M5 x 0.8 x 30, SHCS CLS 10.9, PT, BLZ, DIN 912	2
3	100657	M5 Nylock Nut Blk Zn	2
4	114018	Standard Tipper Sleeve, Rogue	2
5	100533	Detent Button	2
6	119672	Anti Tip Extra Long 24-26 Rigid Catalyst Spark w/Reclining Back <i>Replaces 200524 (XL Anti tips) and 501509</i>	2
7	119670	Anti Tip Long 25/26 CAT 22 ROGUE 22-26 RG ALX/LW XP 26 ETHOS <i>Replaces 100228</i>	2
8	119669	Anti Tip Medium 22/24CAT 18/20 Rogue-Clik-LW 22/24/25ETHOS <i>Replaces 100226</i>	2
9	004306	Anti-Tip Anti Rattle Bushing	2