


Catalyst 5 Hemi Wheel Lock

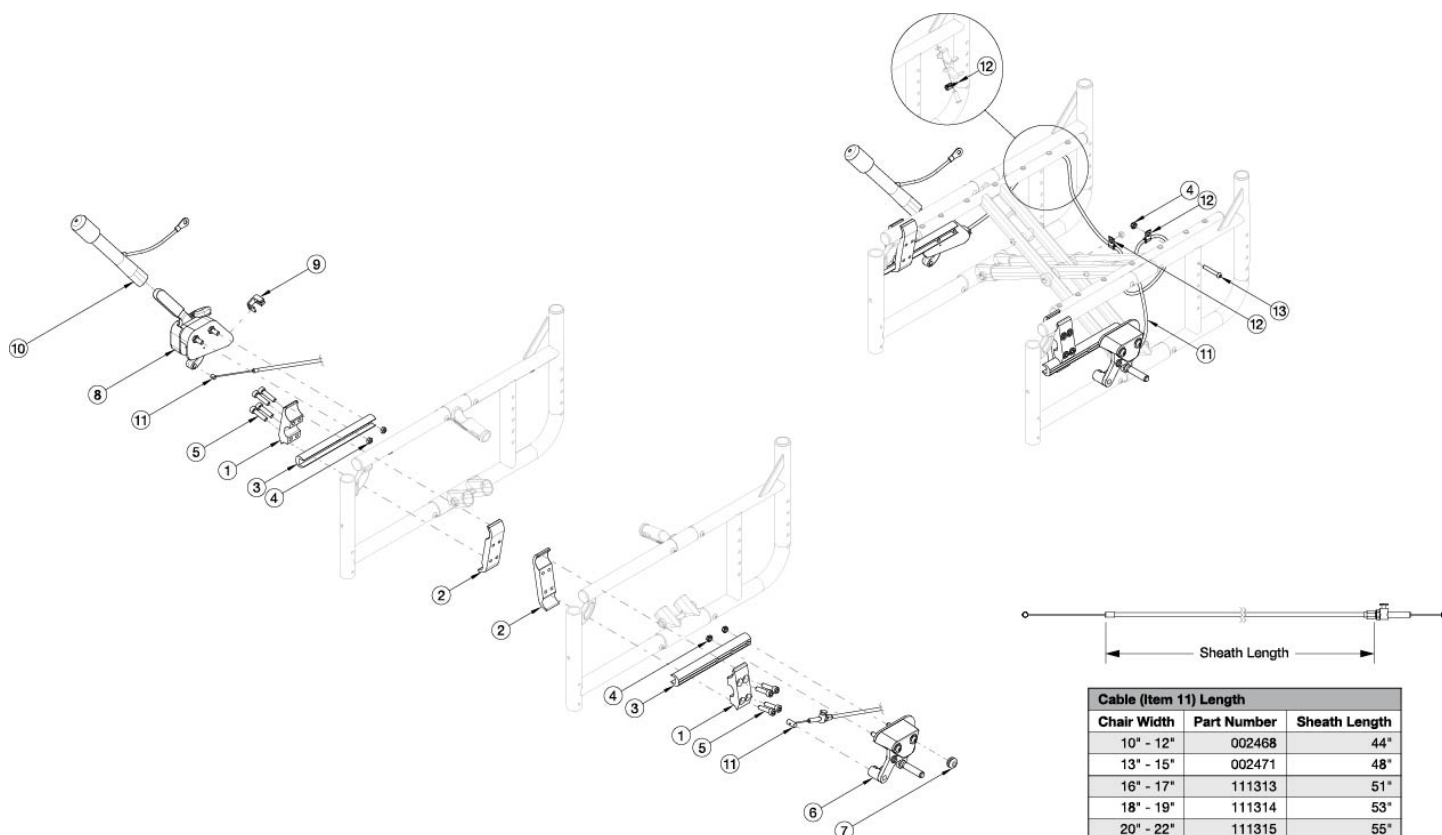

Item Number	Part Number	Description	Quantity
1-5, 6a, 7, 8a, 9, 10, 11a, 12, 13	122926	Hemi Wheel Lock Assembly, Width 10-17, Right Side Mounted, Catalyst 5	1
1-5, 6a, 7, 8a, 9, 10, 11b, 12, 13	122928	Hemi Wheel Lock Assembly, Width 18-22, Right Side Mounted, Catalyst 5	1
1-5, 6b, 7, 8b, 9, 10, 11a, 12, 13	122927	Hemi Wheel Lock Assembly, Width 10-17, Left Side Mounted, Catalyst 5	1
1-5, 6b, 7, 8b, 9, 10, 11b, 12, 13	122929	Hemi Wheel Lock Assembly, Width 18-22, Left Side Mounted, Catalyst 5	1
1	000143	Wheel Lock Clamp Extended Top CAT	1
2	000144	Wheel Lock Clamp Extended Bottom CAT	1
3	001321	Wheel Lock Mount Bar Extended	1
4	100658	M6 Nylock Nut Blk Zn	5
5	100780	M6 x 25 SHCS ZC	4
6a	111591	Wheel Lock Follower Assembly without Mount, Left	1

Item Number	Part Number	Description	Quantity
6b	111592	Wheel Lock Follower Assembly without Mount, Right	1
7	115743	Wheel Lock Retaining Grommet	1
8a	111593	Hemi Wheel Lock, Master Assembly, Right without Mount	1
8b	111594	Hemi Wheel Lock, Master Assembly, Left without Mount	1
9	122971	Hemi WL, Cable Retaining Clip - Molded	1
10	200369	Wheel Lock Extension	1
11a	122726	Drum Brake / Hemi Wheel Lock Short Cable <i>Used with Seat Widths 10 - 17"</i>	1
11b	122727	Drum Brake / Hemi Wheel Lock Medium Cable <i>Used with Seat Widths 18-22"</i>	1
12	001239	P-Clip 3/16"	3
13	100797	M6 x 40 BHCS BLK ZC	1