


Rogue ALX Mx Tower Index Mount (Formerly Tsunami)

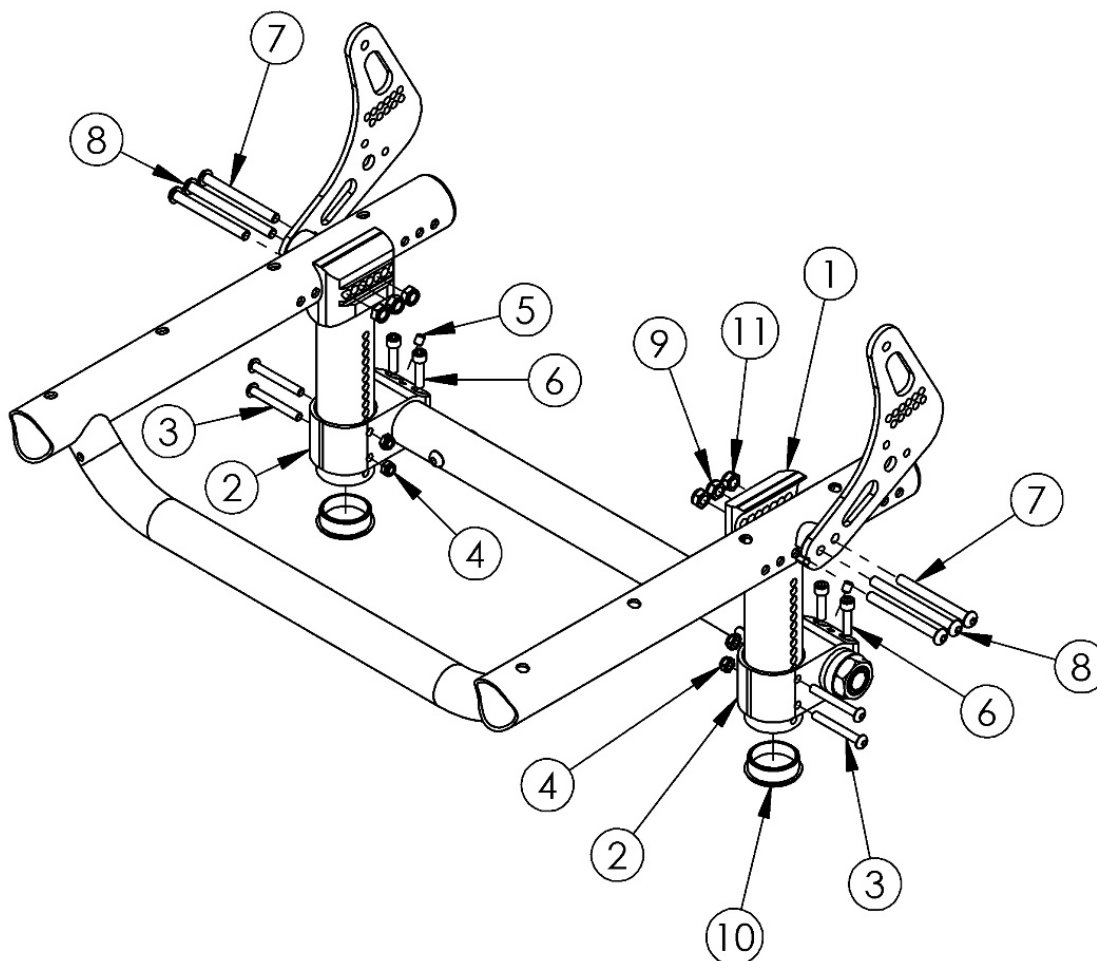

Replacement MX Towers may be quoted, but must be ordered through Customer Service

Item Number	Part Number	Description	Quantity
1	000493	MX Tower, RG ALX	1
2	000462	Camber Mount Offset, RG ALX/LW XP	1
3	101186	M5 X 45 CLS12.9 DIN912 SHCS P/THD BLZ	2
4	100657	M5 Nylock Nut Blk Zn	2
5	105015	M5 X 6 DIN914 SSS CONE PT PLN	1
6	101120	M5 x 16 SHCS BLZ	2
7	003232	M6 x 55 BHCS Blk Zc	2

Item Number	Part Number	Description	Quantity
8	107138	M6 x 65, BHCS, P/THD BLZ	1
9	101456	M6 Ovl Top L/N ZC	1
10	101113	Tube End Plug 1 3/8"	1
11	100658	M6 Nylock Nut Blk Zn	2