


Ethos AeroMesh Calf Strap

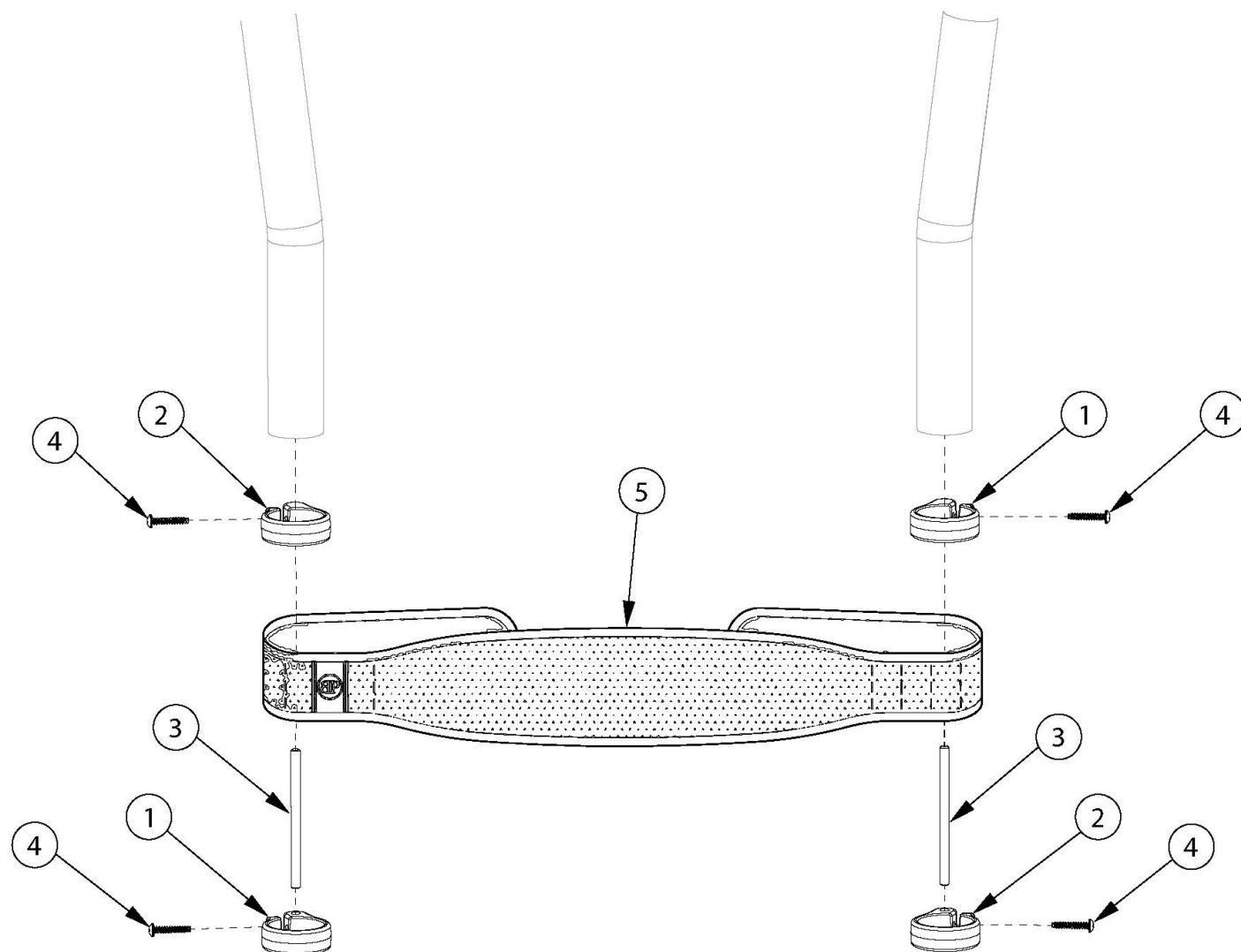

Sizes based on seat width

Item Number	Part Number	Description	Quantity
1-4, 5a	108231	Padded AeroMesh Calf Strap with Clamp Assembly, Short 10-14	1
1-4, 5b	108232	Padded AeroMesh Calf Strap with Clamp Assembly, Medium 15-17	1
1-4, 5c	108233	Padded AeroMesh Calf Strap with Clamp Assembly, Long 18-22	1
1	004462	Calf Strap Clamp A	2
2	004463	Calf Strap Clamp B	2
3	004464	Calf Strap Rod	2
4	103589	6-13 x ¾ Phillips Pan HD, Plastite 45, BLK ZN Screw	4

Item Number	Part Number	Description	Quantity
1-4	108224	Calf Strap Clamp Assembly, Ethos	1
5a	105082	Bodypoint Calf Strap Padded Aeromesh, Short 10-14	1
5b	105080	Bodypoint Calf Strap Padded Aeromesh, Medium 15-17	1
5c	105081	Bodypoint Calf Strap Padded Aeromesh, Long 18-24	1