


Catalyst 4 Flip Back Armrest

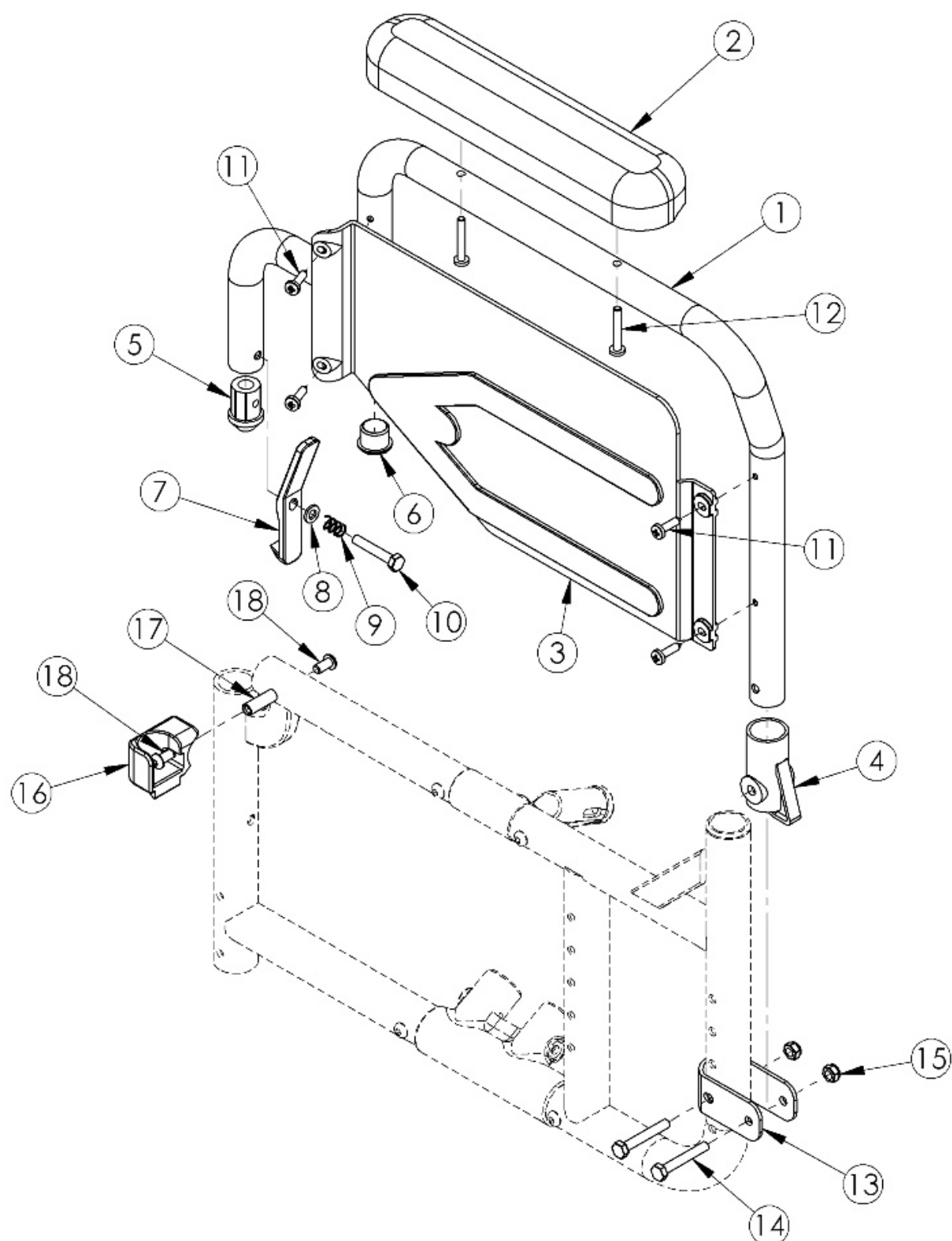

If retrofitting to Flip Back Armrests, please note the following: Receivers are needed New front frames are required Extended wheellock clamps may be needed

Item Number	Part Number	Description	Quantity
1a, 2a, 3b, 4 - 12, 13, 14, 15, 16a, 17, 18	502255	Armrest Flip Back w/ Receiver and Bracket Kit Left Desk CAT4	1
1c, 2a, 3a, 4 - 12, 13, 14, 15, 16b, 17, 18	502256	Armrest Flip Back w/ Receiver and Bracket Kit Right Desk CAT4	1
1b, 2b, 3b, 4 - 12, 13, 14, 15, 16a, 17, 18	502257	Armrest Flip Back w/ Receiver and Bracket Kit Left Full CAT4	1
1d, 2b, 3a, 4 - 12, 13, 14, 15, 16b, 17, 18	502258	Armrest Flip Back w/ Receiver and Bracket Kit Right Full CAT4	1
1a, 1c, 2a, 3a, 3b, 4 - 12, 13a, 14 - 18	502279	Armrest Flip Back w/ Receiver and Bracket Kit PAIR Desk CAT4	1
1a, 1c, 2b, 3a, 3b, 4 - 12, 13a, 14 - 18	502280	Armrest Flip Back w/ Receiver and Bracket Kit PAIR Full CAT4	1
1a, 2a, 3b, 4 - 12	500734	Armrest Assembly Std Flip Back Desk Left CAT	1
1c, 2a, 3a, 4 - 12	500735	Armrest Assembly Std Flip Back Desk Right CAT	1
1b, 2b, 3b, 4 - 12	500736	Armrest Assembly Std Flip Back Full Left CAT	1
1d, 2b, 3a, 4 - 12	500737	Armrest Assembly Std Flip Back Full Right CAT	1
13, 14, 15	502251	Armrest Flip Back Bracket Kit CAT4	1
16a, 17, 18	502253	Armrest Flip Back Receiver Kit Left CAT	1
16b, 17, 18	502254	Armrest Flip Back Receiver Kit Right CAT	1
2a	100501	Arm Pad Desk	1
2b	100500	Arm Pad Full	1
3a	100687	Armrest Flip Back Panel Fixed Right	1
3b	100688	Armrest Flip Back Panel Fixed Left	1
4	100691	Armrest Flip Back Rear Pivot Bracket	1
5	100690	Armrest Flip Back Front Tube Plug	1
6	100803	Tube End Plug 7/8"	1
7	100689	Armrest Flip Back Release Lever	1
8	102072	Washer Nylon .234 ID x .406 OD x .062 Thick	1
9	100693	Compression Spring 0.34" OD x 0.70 LG	1

Item Number	Part Number	Description	Quantity
10	100860	M6 X 35 Hhcs P/Thd Zc	1
11	100723	10 x 3/4 PH Pan HD T-A T/S ZC	4
12	100696	M5 x 30mm PPH Capscrew	2
13	000080	Armrest Flip Back Bracket CAT 4	1
14	100681	M6 x 40 CLS8.8 DIN931 HHCS ZC	2
15	100658	M6 Nylock Nut Blk Zn	2
16a	000081	Armrest Flip Back Receiver Left CAT	1
16b	000082	Armrest Flip Back Receiver Right CAT	1
17	000065	M6 x 23.68 Threaded Barrel	1
18	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2