


Flip Pediatric Height Adjustable T-Arm

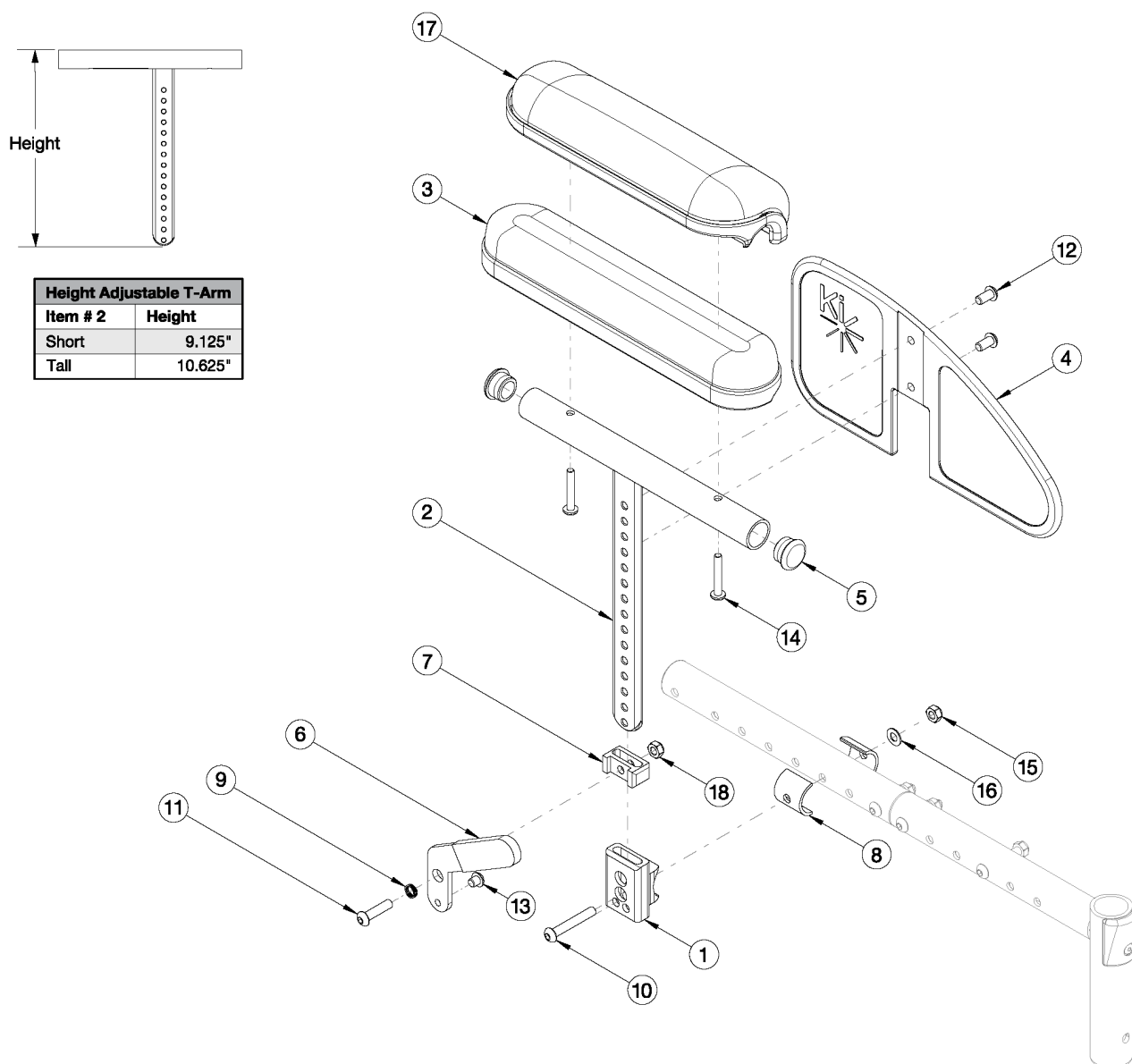

Height Adjustable T-Arm	
Item # 2	Height
Short	9.125"
Tall	10.625"

If retrofitting, Receivers are needed.

Item Number	Part Number	Description	Quantity
1, 2a, 3-14a, 15, 16, 18	502895	Height Adjustable Pediatric T-Arm Assm, Short Desk, w/ Receiver PAIR FLIP	1
1, 2a, 3-5, 6a, 7-14a, 15, 16, 18	503022	Height Adj Pediatric T-Arm Assm, Short Desk, w/ Receiver Left Flip	1
1, 2a, 3-5, 6b, 7-14a, 15, 16, 18	503023	Height Adj Pediatric T-Arm Assm, Short Desk, w/ Receiver Right Flip	1
1, 2b, 3-14a, 15, 16, 18	502896	Height Adjustable Pediatric T-Arm Assm, Tall Desk, w/ Receiver PAIR FLIP	1

Item Number	Part Number	Description	Quantity
1, 2b, 3-5, 6a, 7-14a, 15, 16, 18	503024	Height Adj Pediatric T-Arm Assm, Tall Desk, w/ Receiver Left Flip	1
1, 2b, 3-5, 6b, 7-14a, 15, 16, 18	503025	Height Adj Pediatric T-Arm Assm, Tall Desk, w/ Receiver Right Flip	1
1, 2a, 4-14b, 15-18	503047	Height Adj Pediatric T-Arm Assm, Short Waterfall Desk, w/ Receiver PAIR Flip	1
1, 2a, 4, 5, 6a, 7-14b, 15-18	503045	Height Adj Pediatric T-Arm Assm, Short Waterfall Desk, w/ Receiver Left Flip	1
1, 2a, 4, 5, 6b, 7-14b, 15-18	503046	Height Adj Pediatric T-Arm Assm, Short Waterfall Desk, w/ Receiver Right Flip	1
1, 2b, 4-14b, 15-18	503050	Height Adj Pediatric T-Arm Assm, Tall Waterfall Desk, w/ Receiver PAIR Flip	1
1, 2b, 4, 5, 6a, 7-14b, 15-18	503048	Height Adj Pediatric T-Arm Assm, Tall Waterfall Desk, w/ Receiver Left Flip	1
1, 2b, 4, 5, 6b, 7-14b, 15-18	503049	Height Adj Pediatric T-Arm Assm, Tall Waterfall Desk, w/ Receiver Right Flip	1
2a, 3-7, 9, 11-13, 14a, 18	505940	Height Adjustable Pediatric T-Arm Assm, Short Desk, PAIR FLIP	1
2a, 3-5, 6a, 7, 9, 11-13, 14a, 18	505942	Height Adj Pediatric T-Arm Assm, Short Desk, Left Flip	1
2a, 3-5, 6b, 7, 9, 11-13, 14a, 18	505943	Height Adj Pediatric T-Arm Assm, Short Desk, Right Flip	1
2b, 3-7, 9, 11-13, 14a, 18	505941	Height Adjustable Pediatric T-Arm Assm, Tall Desk, PAIR FLIP	1
2b, 3-5, 6a, 7, 9, 11-13, 14a, 18	505944	Height Adj Pediatric T-Arm Assm, Tall Desk, Left Flip	1
2b, 3-5, 6b, 7, 9, 11-13, 14a, 18	505945	Height Adj Pediatric T-Arm Assm, Tall Desk, Right Flip	1
2a, 4-7, 9, 11-13, 14b, 17, 18	505948	Height Adj Pediatric T-Arm Assm, Short Waterfall Desk, PAIR Flip	1
2a, 4-6a, 7, 9, 11-13, 14b, 17, 18	505946	Height Adj Pediatric T-Arm Assm, Short Waterfall Desk, Left Flip	1
2a, 4-6b, 7, 9, 11-13, 14b, 17, 18	505947	Height Adj Pediatric T-Arm Assm, Short Waterfall Desk, Right Flip	1
2b, 4-7, 9, 11-13, 14b, 17, 18	505951	Height Adj Pediatric T-Arm Assm, Tall Waterfall Desk, PAIR Flip	1
2b, 4-6a, 7, 9, 11-13, 14b, 17, 18	505949	Height Adj Pediatric T-Arm Assm, Tall Waterfall Desk, Left Flip	1
2b, 4-6b, 7, 9, 11-13, 14b, 17, 18	505950	Height Adj Pediatric T-Arm Assm, Tall Waterfall Desk, Right Flip	1
1, 8, 10, 15, 16	503026	Height Adj Pediatric T-Arm Receiver Assm, Flip	1
1	002037	Ped T-Arm / Side Guard Receiver FLIP <i>This receiver design effective October 23rd, 2020</i>	1

Item Number	Part Number	Description	Quantity
2a, 5	002045	Post Weldment, T-Arm, Ped, Short	1
2b, 5	002046	Post Weldment, T-Arm, Ped, Tall	1
3	100501	Arm Pad Desk	1
4	503881	Pediatric T Arm Panel w/Ki Logo	1
5	100803	Tube End Plug 7/8"	2
6a	501396	Pediatric T Arm Lever Left	1
6b	501397	Pediatric T Arm Lever Right	1
7	501395	Pediatric T Arm Bracket	1
8	114443	Tube Spacer, Black, 1" OD x .083" x .80 LG	2
9	101088	Wave Spring - 1/4 id x 3/8 od	1
10	100797	M6 x 40 BHCS BLK ZC	1
11	101593	M6 x 25 BTN SHCS BLKZC	1
12	101184	M6 x 10 BTN SHCS BLKZC w/Patch	2
13	101702	M6 x 6 BTN SHCS ZC	1
14a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pad</i>	2
14b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pad</i>	2
15	100658	M6 Nylock Nut Blk Zn	1
16	102803	Washer Flat .250 X .500 X .027 Hx Blz	1
17	102521	Waterfall Arm Pad Desk	1
18	101456	M6 Ovl Top L/N ZC	1