


Focus CR Wheel Lock

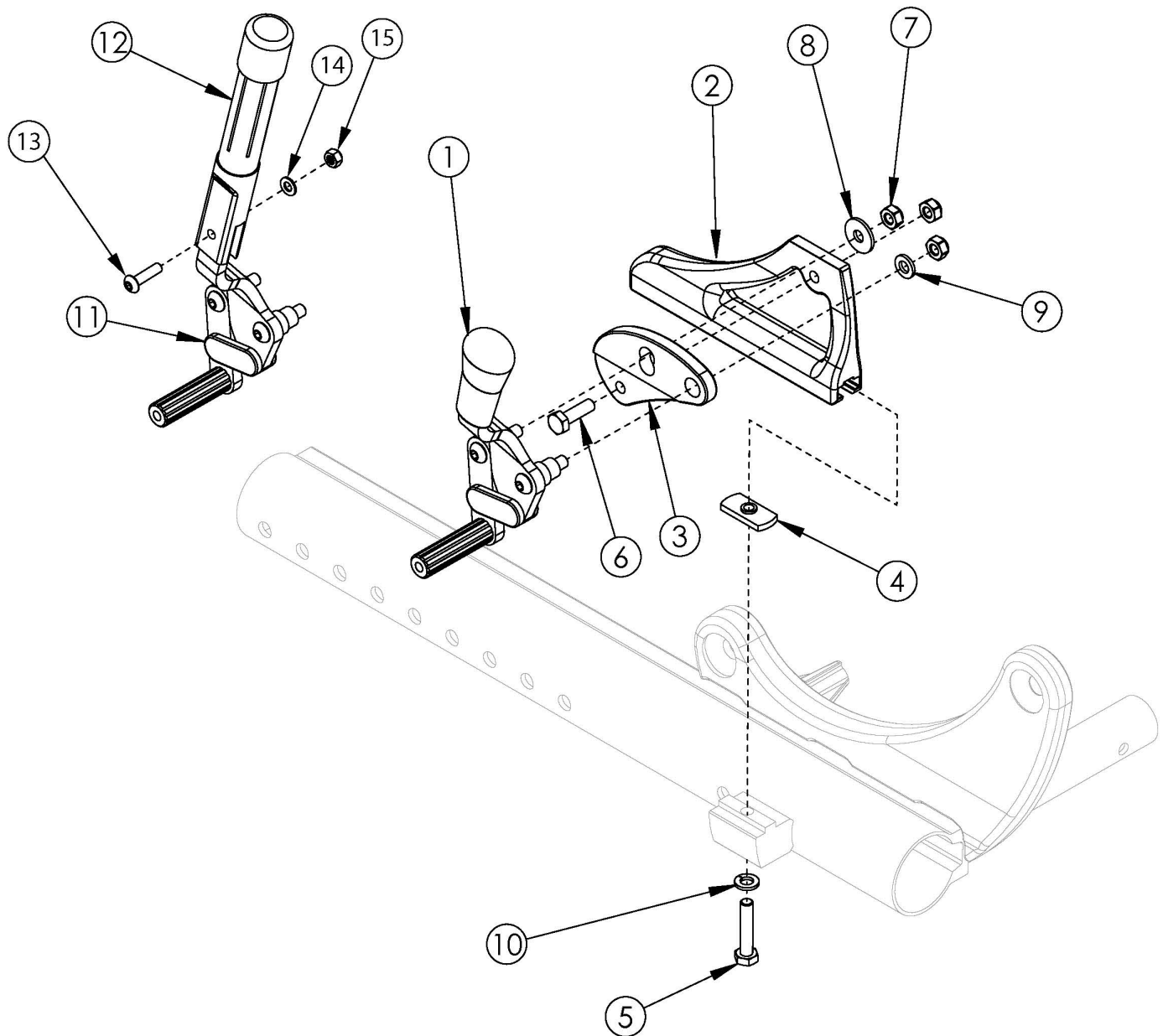

Does the chair have a combination wheel lock? If yes, please see the combination wheel lock parts manual page.

Item Number	Part Number	Description	Quantity
1a, 2b, 3a, 4 - 10	501486	Wheel Lock Assembly Push Left Focus	1
1b, 2a, 3a, 4 - 10	501487	Wheel Lock Assembly Push Right Focus	1
1a, 1b, 2a, 2b, 3a, 4 - 10	501113	Wheel Lock Assembly Push PAIR Focus	1
2b, 3a, 4 - 10, 11a, 12-15	501488	Wheel Lock Assembly Push w/ Ext Left Focus	1

Item Number	Part Number	Description	Quantity
2a, 3a, 4 - 10, 11b, 12-15	501489	Wheel Lock Assembly Push w/ Ext Right Focus	1
2a, 2b, 3a, 4 - 10, 11a, 11b, 12-15	501114	Wheel Lock Assembly Push w/ Ext PAIR Focus	1
1c, 2b, 3b, 4 - 10	501490	Wheel Lock Assembly Pull Left Focus	1
1d, 2a, 3b, 4 - 10	501491	Wheel Lock Assembly Pull Right Focus	1
1c, 1d, 2a, 2b, 3b, 4 - 10	501115	Wheel Lock Assembly Pull PAIR Focus	1
2b, 3b, 4 - 10, 11c, 12-15	501492	Wheel Lock Assembly Pull w/ Ext Left Focus	1
2a, 3b, 4 - 10, 11d, 12-15	501493	Wheel Lock Assembly Pull w/ Ext Right Focus	1
2a, 2b, 3b, 4 - 10, 11c, 11d, 12-15	501116	Wheel Lock Assembly Pull w/ Ext PAIR Focus	1
2a	001138	Plate Wheel Lock Right Tilt	1
2b	001139	Plate Wheel Lock Left Tilt	1
3a	003769	Wheel Lock Mount Push Tilt <i>Replaced 001140</i>	1
3b	003770	Wheel Lock Mount Pull Tilt <i>Replaced 001238</i>	1
4	001141	Weld Nut NDM 06028 Zinc	1
5	101824	M6 X 30 Hhcs Blk Zn	1
6	101823	M6 X 20 Hhcs Blk Zn	1
7	100658	M6 Nylock Nut Blk Zn	3
8	101853	M6 Washer Blk Zn	1
9	100746	M6 Flat Washer Blk Zn	1
10	101835	M6 Lock Washer Blk Zn	1
12-15	100854	Brake Extension Replacement	1
13	101117	M5 x 22 BTN SHCS BLK ZC	1
14	101856	Flat Washer, M5, Blk Zn	1
15	100657	M5 Nylock Nut Blk Zn	1

Inverted axle plates are not used with wheels 20" or larger.

Wheel	Tire	Caster	Standard Axle Plate Seat Heights	Inverted Axle Plate Seat Heights
12" Mag	Pneumatic	5"	13.5" - 14.5"	17.5" - 19.5"
12" Mag	Pneumatic	6"	14.5"	17.5" - 19.5"
12" Mag	Pneumatic	6x2"	14.5"	17.5" - 19.5"
12" Mag	Pneumatic	7"	n/a	17.5" - 19.5"
12" Mag	Pneumatic	8"	n/a	17" - 19.5"
12" Mag	Pneumatic	8"x1.5"	n/a	17.5" - 19.5"
12" Mag	Pneumatic	8x2"	n/a	17.5" - 19.5"
12" Mag	Full Poly	5"	n/a	16.5" - 19"
12" Mag	Full Poly	6"	n/a	16.5" - 19"
12" Mag	Full Poly	6x2"	n/a	16.5" - 19"
12" Mag	Full Poly	7"	n/a	16.5" - 19"
12" Mag	Full Poly	8"	n/a	16." - 18.5"
12" Mag	Full Poly	8"x1.5"	n/a	16.5" - 19"
12" Mag	Full Poly	8x2"	n/a	17" - 18.5"
12" Mag w/ Drum Brake	Pneumatic	5"	13" - 14.5"	17.5" - 20"
12" Mag w/ Drum Brake	Pneumatic	6"	14" - 14.5"	17.5" - 19.5"
12" Mag w/ Drum Brake	Pneumatic	6x2"	14" - 14.5"	17.5" - 20"
12" Mag w/ Drum Brake	Pneumatic	7"	n/a	17.5" - 20"
12" Mag w/ Drum Brake	Pneumatic	8"	n/a	17.5" - 20"
12" Mag w/ Drum Brake	Pneumatic	8"x1.5"	n/a	17.5" - 19.5"
12" Mag w/ Drum Brake	Pneumatic	8x2"	n/a	17.5" - 19.5"
12" Mag w/ Drum Brake	Full Poly	5"	13" - 14.5"	17.5" - 19.5"
12" Mag w/ Drum Brake	Full Poly	6"	14" - 14.5"	17.5" - 19.5"
12" Mag w/ Drum Brake	Full Poly	6x2"	14" - 14.5"	17.5" - 19.5"
12" Mag w/ Drum Brake	Full Poly	7"	n/a	17.5" - 19.5"
12" Mag w/ Drum Brake	Full Poly	8"	n/a	17.5" - 19.5"
12" Mag w/ Drum Brake	Full Poly	8"x1.5"	n/a	17.5" - 19.5"
12" Mag w/ Drum Brake	Full Poly	8x2"	n/a	17.5" - 19.5"

Inverted axle plates are not used with wheels 20" or larger.

Wheel	Tire	Caster	Standard Axle Plate Seat Heights	Inverted Axle Plate Seat Heights
16" Mag	Pneumatic	5"	13" - 16"	19" - 20.5"
16" Mag	Pneumatic	6"	14" - 16"	19" - 20.5"
16" Mag	Pneumatic	6x2"	14" - 16"	19" - 20.5"
16" Mag	Pneumatic	7"	15" - 16"	19" - 20.5"
16" Mag	Pneumatic	8"	16"	19" - 20.5"
16" Mag	Pneumatic	8"x1.5"	15.5" - 16"	19" - 20.5"
16" Mag	Pneumatic	8x2"	16"	19" - 20.5"
16" Mag	Full Poly	5"	13" - 16.5"	19" - 20.5"
16" Mag	Full Poly	6"	14" - 16.5"	19" - 20.5"
16" Mag	Full Poly	6x2"	14" - 16.5"	19" - 20.5"
16" Mag	Full Poly	7"	15" - 16.5"	19.5" - 20.5"
16" Mag	Full Poly	8"	16"	19" - 20.5"
16" Mag	Full Poly	8"x1.5"	15.5" - 16.5"	19" - 20.5"
16" Mag	Full Poly	8x2"	16" - 16.5"	19" - 20.5"
16" Mag	Low Poly	5"	13" - 15"	18" - 20"
16" Mag	Low Poly	6"	14" - 15"	18" - 20"
16" Mag	Low Poly	6x2"	14" - 15"	18" - 20.5"
16" Mag	Low Poly	7"	n/a	18" - 20.5"
16" Mag	Low Poly	8"	n/a	18" - 20.5"
16" Mag	Low Poly	8"x1.5"	n/a	18" - 20"
16" Mag	Low Poly	8x2"	n/a	18" - 20"
16" Mag w/ Drum Brake	All	5"	13.5" - 16"	19" - 20.5"
16" Mag w/ Drum Brake	All	6"	14" - 16"	18.5" - 20.5"
16" Mag w/ Drum Brake	All	6x2"	14" - 16"	18.5" - 20.5"
16" Mag w/ Drum Brake	All	7"	15" - 15.5"	18.5" - 20.5"
16" Mag w/ Drum Brake	All	8"	n/a	18.5" - 20.5"
16" Mag w/ Drum Brake	All	8"x1.5"	15.5" - 16"	18.5" - 20.5"
16" Mag w/ Drum Brake	All	8x2"	16"	18.5" - 20.5"