


# CR45 Transit

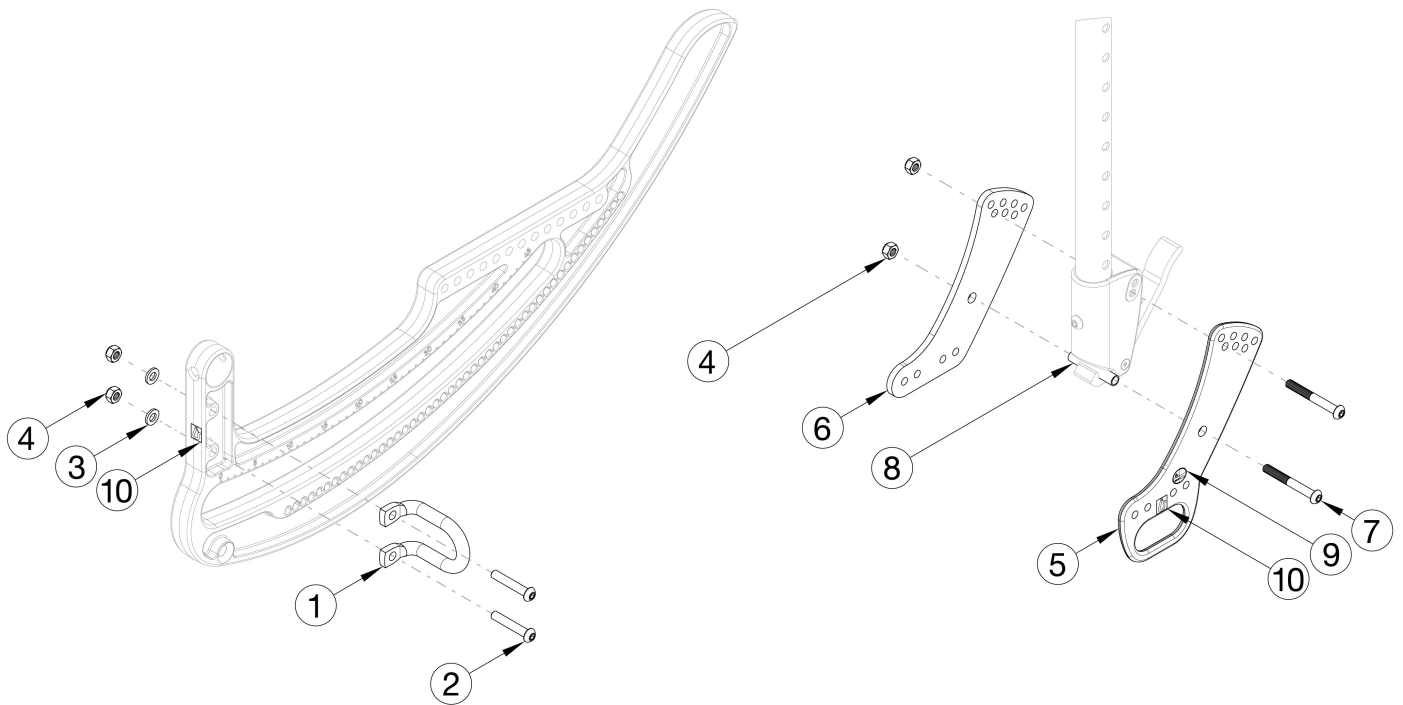

Item Number	Part Number	Description	Quantity
1-10	116299	Transit Retrofit Kit	1
1	001078	Transit Bracket Front TILT	2
2	101811	M6 x 35 BHCS Blk Zn	4
3	100746	M6 Flat Washer Blk Zn	12
4	100658	M6 Nylock Nut Blk Zn	14
5	112489	Transit Back Plate, CR45/Flip XPA	2
6	112488	Standard Back Plate, CR45/Flip XPA	2
7	101781	M6 x 50mm Partial Thread BHCS Blk ZC	4
8	002738	Spacer .313 OD x .243 ID x 1.68 long	2