


Focus CR Seat Pan

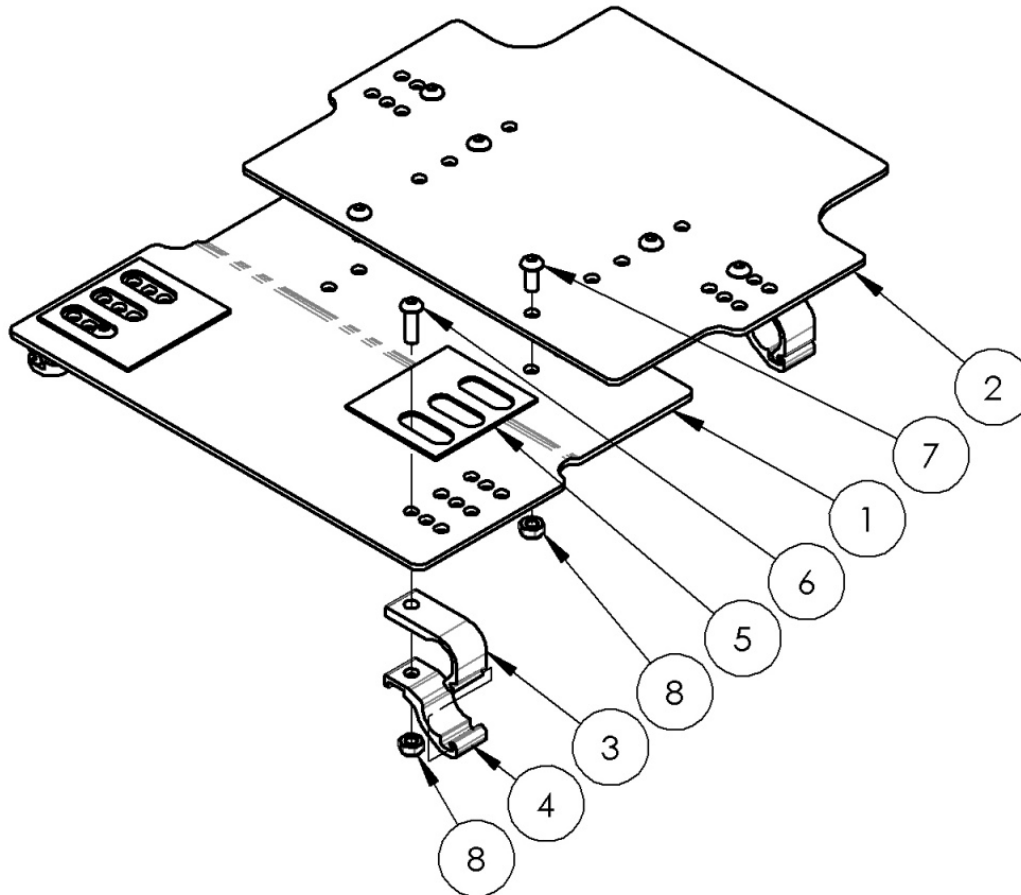

Grows two inches in width from even widths and one inch from odd widths. Max width is 24". <p>Depth growth is matched to frame depth range and can be adjusted to any depth in the range.

Item Number	Part Number	Description	Quantity	Retail
1a, 2a, 3 - 8	501144	Seat Pan Assm Short 14-16" Wide Focus	1	\$410.06
1b, 2b, 3 - 8	501145	Seat Pan Assm Short 16-18" Wide Focus	1	\$410.06
1c, 2c, 3 - 8	501146	Seat Pan Assm Short 18-20" Wide Focus	1	\$410.06
1d, 2d, 3 - 8	501147	Seat Pan Assm Short 20-22" Wide Focus	1	\$410.06
1e, 2e, 3 - 8	501218	Seat Pan Assm Short 22" Wide Focus	1	\$692.66
1f, 2a, 3 - 8	501148	Seat Pan Assm Medium 14-16" Wide Focus	1	\$410.06

Item Number	Part Number	Description	Quantity	Retail
1g, 2b, 3 - 8	501149	Seat Pan Assm Medium 16-18" Wide Focus	1	\$410.06
1h, 2c, 3 - 8	501150	Seat Pan Assm Medium 18-20" Wide Focus	1	\$410.06
1i, 2d, 3 - 8	501151	Seat Pan Assm Medium 20-22" Wide Focus	1	\$410.06
1j, 2e, 3 - 8	501219	Seat Pan Assm Medium 22" Wide Focus	1	\$692.66
1k, 2a, 3 - 8	501152	Seat Pan Assm Long 14-16" Wide Focus	1	\$410.06
1l, 2b, 3 - 8	501153	Seat Pan Assm Long 16-18" Wide Focus	1	\$410.06
1m, 2c, 3 - 8	501154	Seat Pan Assm Long 18-20" Wide Focus	1	\$410.06
1n, 2d, 3 - 8	501155	Seat Pan Assm Long 20-22" Wide Focus	1	\$410.06
1o, 2e, 3 - 8	501220	Seat Pan Assm Long 22-24" Wide Focus	1	\$692.66
1a	001184	Seat Pan Front Short 14-16" Wide TILT	1	\$195.19
1b	001185	Seat Pan Front Short 16-18" Wide TILT	1	\$195.19
1c	001186	Seat Pan Front Short 18-20" Wide TILT	1	\$195.19
1d	001187	Seat Pan Front Short 20-22" Wide TILT	1	\$195.19
1e	001188	Seat Pan Front Short 22" Wide TILT	1	\$195.19
1f	001189	Seat Pan Front Medium 14-16" Wide TILT	1	\$195.19
1g	001190	Seat Pan Front Medium 16-18" Wide TILT	1	\$195.19
1h	001191	Seat Pan Front Medium 18-20" Wide TILT	1	\$195.19
1i	001192	Seat Pan Front Medium 20-22" Wide TILT	1	\$195.19
1j	001193	Seat Pan Front Medium 22" Wide TILT	1	\$195.19
1k	001194	Seat Pan Front Long 14-16" Wide TILT	1	\$195.19
1l	001195	Seat Pan Front Long 16-18" Wide TILT	1	\$195.19
1m	001196	Seat Pan Front Long 18-20" Wide TILT	1	\$195.19
1n	001197	Seat Pan Front Long 20-22" Wide TILT	1	\$195.19
1o	001198	Seat Pan Front Long 22-24" Wide TILT	1	\$195.19
2a	001199	Seat Pan Rear 14-16" Wide TILT	1	\$195.19
2b	001200	Seat Pan Rear 16-18" Wide TILT	1	\$195.19
2c	001201	Seat Pan Rear 18-20" Wide TILT	1	\$195.19
2d	001202	Seat Pan Rear 20-22" Wide TILT	1	\$195.19

Item Number	Part Number	Description	Quantity	Retail
2e	001203	Seat Pan Rear 22-24" Wide TILT	1	\$195.19
3	001204	Seat Pan/Base Bracket TILT	4	\$34.86
4	001205	Seat Pan/Base Clamp TILT	4	\$36.00
5	001206	AB Tape 3x3 Die Cut Tilt	2	\$2.52
6	101814	M8 x 25 BHCS Blk Zn	4	\$1.05
7	101813	M8 x 18 BHCS Blk Zn	4	\$2.10
8	100659	M8 Nylock Nut Blk Zn	8	\$1.05