


# Catalyst Backrest - Straight Push Handle with Rigidizer Bar

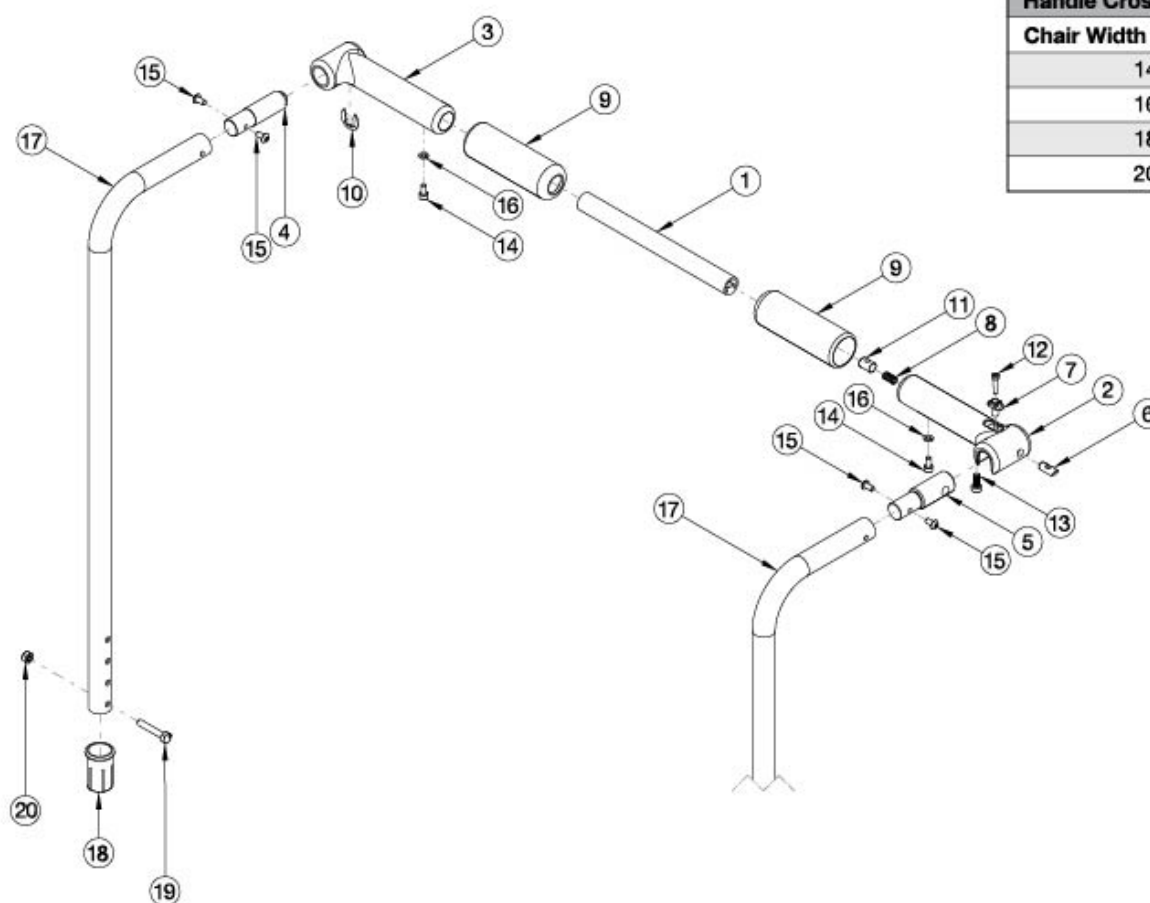

| Handle Crosstube (Item 1) |          |
|---------------------------|----------|
| Chair Width Range         | Part No. |
| 14" - 16"                 | 110264   |
| 16" - 18"                 | 110265   |
| 18" - 20"                 | 110266   |
| 20" - 22"                 | 110267   |

If Anti-Rollback is present, see Back Tubes for Anti-Rollback System

| Item Number   | Part Number | Description                                              | Quantity |
|---------------|-------------|----------------------------------------------------------|----------|
| 1a, 2-16, 17a | 116400      | Straight Backtube, Folding Handle, Short, 14"-16"W, CAT  | 1        |
| 1b, 2-16, 17a | 116401      | Straight Backtube, Folding Handle, Short, 16"-18"W, CAT  | 1        |
| 1c, 2-16, 17a | 116402      | Straight Backtube, Folding Handle, Short, 18"-20"W, CAT  | 1        |
| 1d, 2-16, 17a | 116403      | Straight Backtube, Folding Handle, Short, 20"-22"W, CAT  | 1        |
| 1a, 2-16, 17b | 116404      | Straight Backtube, Folding Handle, Medium, 14"-16"W, CAT | 1        |
| 1b, 2-16, 17b | 116405      | Straight Backtube, Folding Handle, Medium, 16"-18"W, CAT | 1        |
| 1c, 2-16, 17b | 116406      | Straight Backtube, Folding Handle, Medium, 18"-20"W, CAT | 1        |
| 1d, 2-16, 17b | 116407      | Straight Backtube, Folding Handle, Medium, 20"-22"W, CAT | 1        |

| Item Number   | Part Number | Description                                                         | Quantity |
|---------------|-------------|---------------------------------------------------------------------|----------|
| 1a, 2-16, 17c | 116408      | Straight Backtube, Folding Handle, Tall, 14"-16"W, CAT              | 1        |
| 1b, 2-16, 17c | 116409      | Straight Backtube, Folding Handle, Tall, 16"-18"W, CAT              | 1        |
| 1c, 2-16, 17c | 116410      | Straight Backtube, Folding Handle, Tall, 18"-20"W, CAT              | 1        |
| 1d, 2-16, 17c | 116411      | Straight Backtube, Folding Handle, Tall, 20"-22"W, CAT              | 1        |
| 1a, 2-16, 17d | 125245      | Straight Backtube, Folding Handle, Extra Tall, 14"-16"W, CAT        | 1        |
| 1b, 2-16, 17d | 125246      | Straight Backtube, Folding Handle, Extra Tall, 16"-18"W, CAT        | 1        |
| 1c, 2-16, 17d | 125247      | Straight Backtube, Folding Handle, Extra Tall, 18"-20"W, CAT        | 1        |
| 1d, 2-16, 17d | 125248      | Straight Backtube, Folding Handle, Extra Tall, 20"-22"W, CAT        | 1        |
| 1a, 2-16      | 116416      | Folding Handle, 14"-16"W, CAT                                       | 1        |
| 1b, 2-16      | 116417      | Folding Handle, 16"-18"W, CAT                                       | 1        |
| 1c, 2-16      | 116418      | Folding Handle, 18"-20"W, CAT                                       | 1        |
| 1d, 2-16      | 116419      | Folding Handle, 20"-22"W, CAT                                       | 1        |
| 1a            | 110264      | Cross Tube, Rigidizer Handle, Small/Large                           | 1        |
| 1b            | 110265      | Cross Tube, Rigidizer Handle, Medium / X-Large                      | 1        |
| 1c            | 110266      | Cross Tube, Rigidizer Handle, 18-20 Wide                            | 1        |
| 1d            | 110267      | Cross Tube, Rigidizer Handle, 20-22 Wide<br><i>Replaces 116463.</i> | 1        |
| 2             | 110261      | Latch, Rigidizer Handle                                             | 1        |
| 3             | 110260      | Pivot, Rigidizer Handle                                             | 1        |
| 4             | 110258      | Adapter, Pivot, Rigidizer Handle                                    | 1        |
| 5             | 110259      | Adapter, Latch, Rigidizer Handle                                    | 1        |
| 6             | 110263      | Pin, Latch, Rigidizer Handle                                        | 1        |
| 7             | 110262      | Button, Latch, Rigidizer Handle                                     | 1        |
| 10            | 110291      | E-RING, M13-M18, DIN6799, ZC                                        | 1        |
| 11            | 102768      | M6 Barrel Nut                                                       | 1        |
| 12            | 110290      | SHCS, M4 x 0.7 x 18, CLS12.9, BLZ, NYL, DIN912                      | 1        |
| 13            | 101185      | M6 x 14 CLS8.8 DIN7984 Low SHCS BLKZC                               | 1        |
| 14            | 101766      | M5 X 10 Cls12.9 Din912 Shcs Pln                                     | 1        |

| Item Number | Part Number | Description                                          | Quantity |
|-------------|-------------|------------------------------------------------------|----------|
| 15          | 100669      | M5 x 10 BTN SHCS BLKZC w/Patch                       | 1        |
| 16          | 102614      | M5 Split Lock Washer BLK ZC                          | 1        |
| 17a         | 116158      | Straight Backtube Push Handle, Short, Rigidizer      | 1        |
| 17b         | 116159      | Straight Backtube Push Handle, Rigidizer, Medium     | 1        |
| 17c         | 116160      | Straight Backtube Push Handle, Tall, Rigidizer       | 1        |
| 17d         | 125249      | Straight Backtube Push Handle, Extra Tall, Rigidizer | 2        |
| 18          | 111965      | Frame Sleeve, Angle Adjustable Backrest              | 1        |
| 19          | 100860      | M6 X 35 Hhcs P/Thd Zc                                | 1        |
| 20          | 100658      | M6 Nylock Nut Blk Zn                                 | 1        |