


(Discontinued) Focus CR One Arm Drive Hex Style

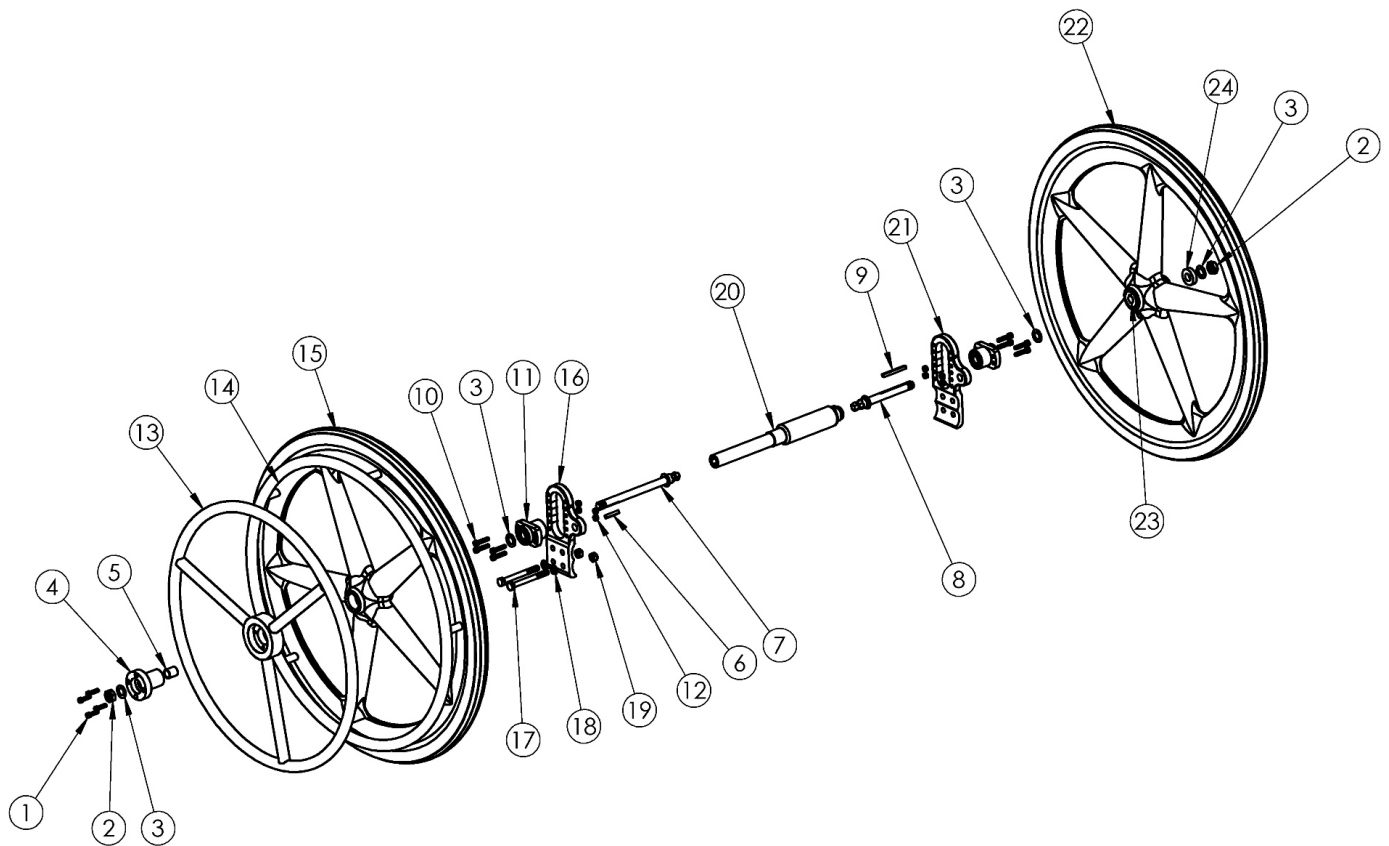

One Arm Drive Kits do not include rear tires. If changing wheel size and the Attendant Foot Lock is present, a new linkage may be required; reference the corresponding Focus CR Attendant Foot Lock page

Item Number	Part Number	Description	Quantity
11, 16, 21	003212	One Arm Drive Focus Axle Plate Assembly PAIR	1
10	101843	M5 x 20 SHCS Blk Zn	8
12	100657	M5 Nylock Nut Blk Zn	8
13a	003198	One Arm Drive Handrim Aluminum 24"	1
17a	101831	M8 x 80 HHCS Blk Zn	4
17b	101832	M8 x 90 HHCS Blk Zn	4
17c	102384	M8 x 100 HHCS Blk Zc Part THD	2
18	101858	Washer Flat M8 Blk Zn	8
19	100659	M8 Nylock Nut Blk Zn	4

Item Number	Part Number	Description	Quantity
20f	504348	One Arm Drive - Drive Shaft 17.69", Focus <i>Used on 19" wide chairs. This kit contains 102646 Drive Shaft and Installation Instructions.</i>	1
20i	504351	One Arm Drive - Drive Shaft 20.69", Focus <i>Used on 22" wide chairs. This kit contains 102752 Drive Shaft and Installation Instructions.</i>	1