


(Discontinued) Rogue Rigid Frame with XP Option

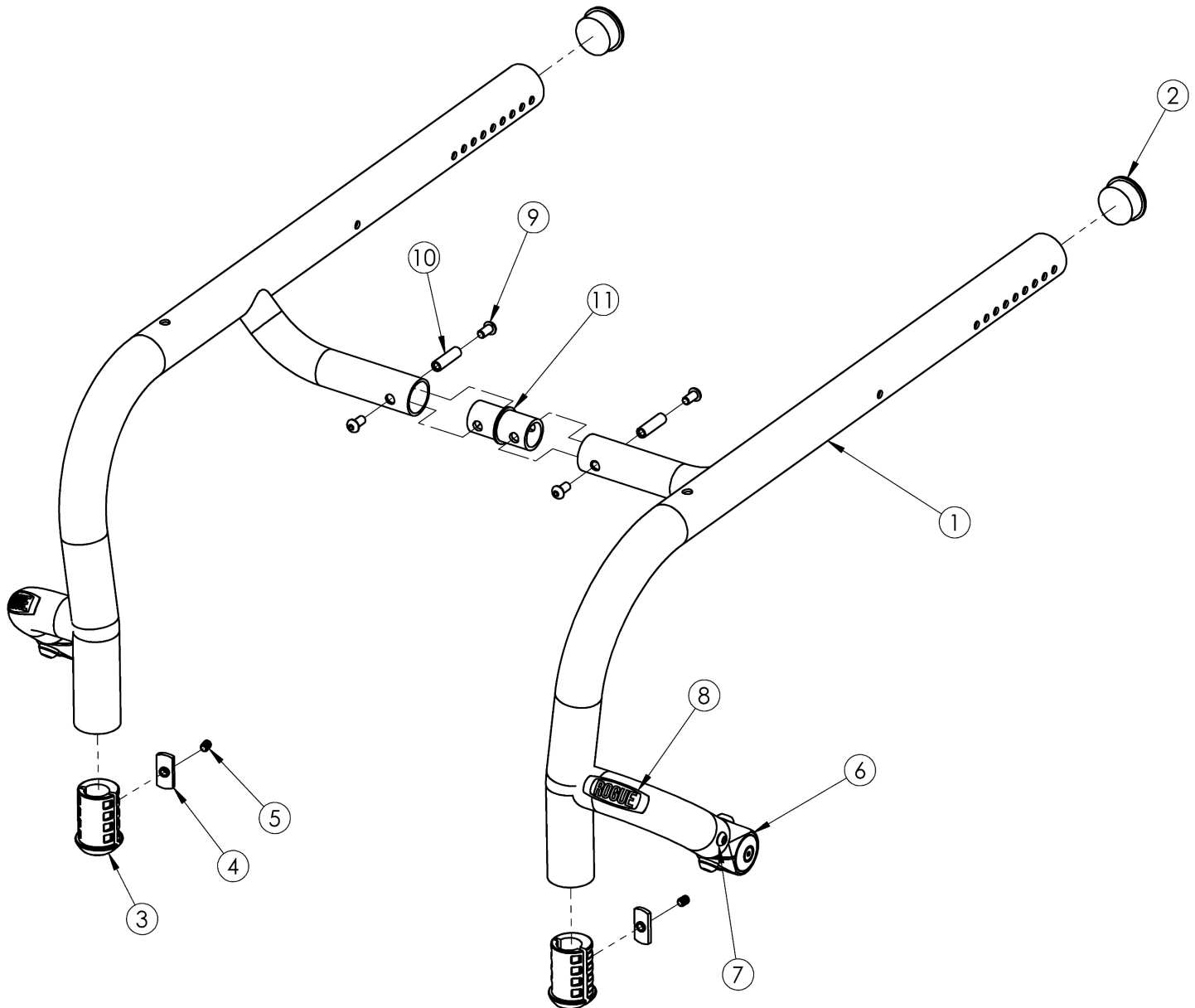

*This Rogue Rigid Frame with XP Option is discontinued as of July 25th, 2016 <p>
 Maximum Growable Seat Width: 19" <p>
 Replacement rigid frames may be quoted, but must be ordered through Customer Service*

| Item Number | Part Number | Description | Quantity | Retail |
|----------------------------------|-------------|--|----------|------------|
| 1a, 2-4a, 5, 6a, 7, 8, 9, 10, 11 | RFRGXP | Replacement Rogue XP Frame <i>Item 11 dependent on initial seat width. 11a is included on seat widths 10" - 16", 11b is included on 17" seat width, 11c is included on 18" seat width and 19" seat width will not be growable or receive a frame connector.</i> | 1 | \$1,071.00 |
| 1b, 2-4a, 5, 6a, 7, 8, 9, 10, 11 | RFRGXPGIT | Replacement Rogue XP Frame w/ Graphic Image Transfer | 1 | \$1,666.00 |

| Item Number | Part Number | Description | Quantity | Retail |
|----------------------------------|----------------|---|----------|------------|
| | | <i>Item 11 dependent on initial seat width. 11a is included on seat widths 10" - 16", 11b is included on 17" seat width, 11c is included on 18" seat width and 19" seat width will not be growable or receive a frame connector.</i> | | |
| 1c, 2-4a, 5, 6a, 7, 8, 9, 10, 11 | RFRGXPTRANS | Replacement Rogue XP Transit Frame <i>This replacement frame does not include the rear transit brackets. Item 11 dependent on initial seat width. 11a is included on seat widths 10" - 16", 11b is included on 17" seat width, 11c is included on 18" seat width and 19" seat width will not be growable or receive a frame connector.</i> | 1 | \$1,308.00 |
| 1d, 2-4a, 5, 6b, 7, 8, 9, 10, 11 | RFRGXPCOL | Replacement Rogue XP Frame w/ Color Anodized Package <i>Item 11 dependent on initial seat width. 11a is included on seat widths 10" - 16", 11b is included on 17" seat width, 11c is included on 18" seat width and 19" seat width will not be growable or receive a frame connector.</i> | 1 | \$1,098.50 |
| 1e, 2-4a, 5, 6b, 7, 8, 9, 10, 11 | RFRGXPGITCOL | Replacement Rogue XP Frame w/ Graphic Image Transfer and Color Anodized Package <i>Item 11 dependent on initial seat width. 11a is included on seat widths 10" - 16", 11b is included on 17" seat width, 11c is included on 18" seat width and 19" seat width will not be growable or receive a frame connector.</i> | 1 | \$1,693.50 |
| 1f, 2-4a, 5, 6b, 7, 8, 9, 10, 11 | RFRGXPTRANSCOL | Replacement Rogue XP Transit Frame w/ Color Anodized Package <i>This replacement frame does not include the rear transit brackets. Item 11 dependent on initial seat width. 11a is included on seat widths 10" - 16", 11b is included on 17" seat width, 11c is included on 18" seat width and 19" seat width will not be growable or receive a frame connector.</i> | 1 | \$1,335.50 |
| 9, 10, 11b | 501331 | Connector Assembly Frame 1" w/ Hardware | 1 | \$47.62 |
| 9, 10, 11c | 501332 | Connector Assembly Frame 2" w/ Hardware | 1 | \$47.62 |
| 9, 10, 11d | 501333 | Connector Assembly Frame 3" w/ Hardware | 1 | \$47.62 |
| 2 | 101113 | Tube End Plug 1 3/8" <i>Use quantity 4 with high-mounted footrests. Extra plugs are inserted into front of frame.</i> | 2 | \$3.09 |
| 3 | 001346 | Footrest Sleeve 3/4" to 1 3/8" <i>Used only with frame-inserted footrests. Use item number 2 with high mounted footrests.</i> | 2 | \$5.89 |
| 4a | 001141 | Weld Nut NDM 06028 Zinc <i>Used only with frame-inserted footrests. Use item number 2 with high mounted footrests.</i> | 2 | \$2.58 |
| 4b | 102092 | M6 Weld Nut Offset <i>Used only as needed with frame-inserted footrests. Use item number 2 with high mounted footrests.</i> | 2 | \$3.09 |
| 5 | 102090 | M6 Set Screw Flat Point <i>Used only with frame-inserted footrests. Use item number 2 with high mounted footrests.</i> | 2 | \$2.58 |
| 8 | 002473 | Aluminum Wing Badge Rogue | 2 | \$2.63 |
| 9 | 100667 | M6 x 12 BTN SHCS BLKZC w/Patch <i>Not included on 19" wide frame</i> | 4 | \$1.55 |

| Item Number | Part Number | Description | Quantity | Retail |
|--------------------|--------------------|---|-----------------|---------------|
| 10 | 000980 | M6 x 27.69 Threaded Barrel <i>Not included on 19" wide frame</i> | 2 | \$5.25 |
| 11a | 000585 | Frame Connector 0" LW/RG | 1 | \$32.40 |
| 11b | 000586 | Frame Connector 1" LW/RG | 1 | \$32.40 |
| 11c | 000587 | Frame Connector 2" LW/RG | 1 | \$32.40 |
| 11d | 000588 | Frame Connector 3" LW/RG | 1 | \$32.40 |

Color Anodizing Samples

