


(Discontinued 1) Rogue Fixed Height Backrest Frame with Non-Adjustable Rigidizer Bar

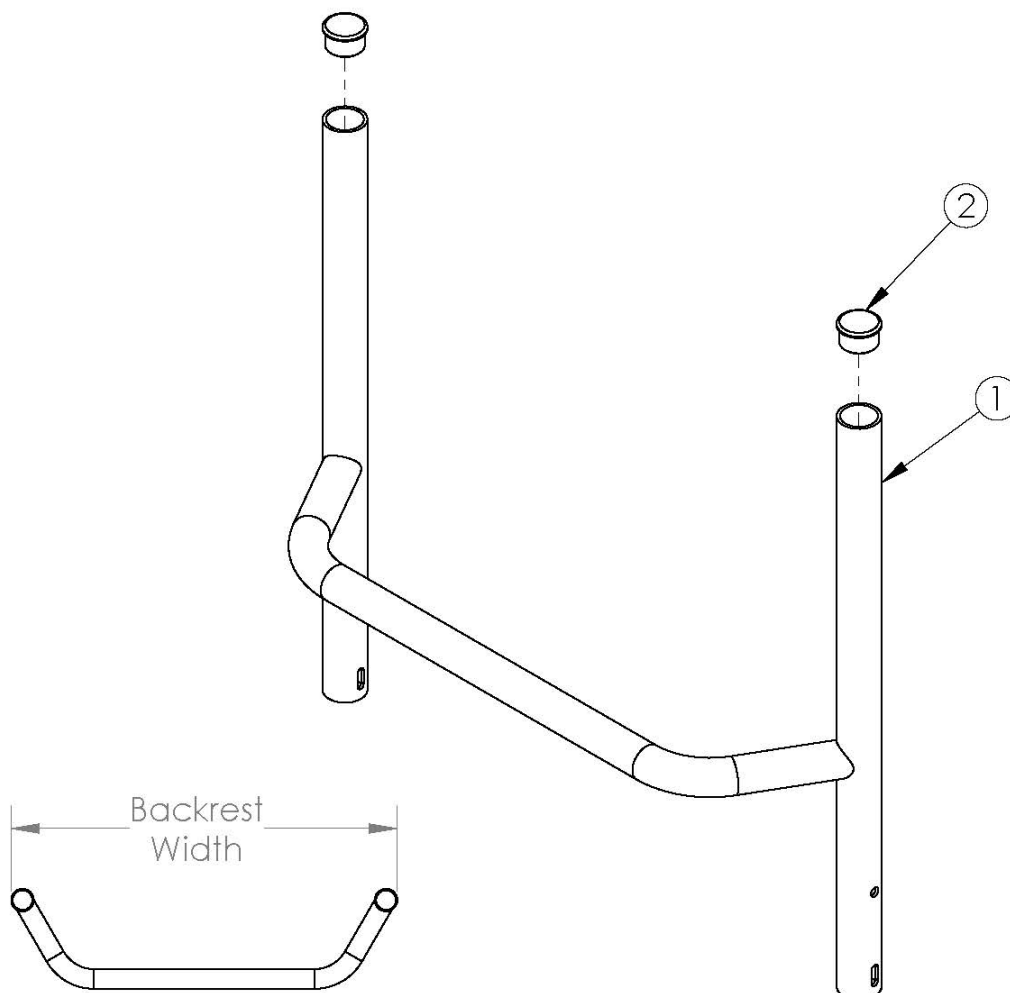

Backrest 10"	10.13"
Backrest 11"	11.13"
Backrest 12"	12.13"
Backrest 13"	13.13"
Backrest 14"	14.13"
Backrest 15"	15.13"
Backrest 16"	16.13"
Backrest 17"	17.13"
Backrest 18"	18.13"
Backrest 19"	19.13"
Backrest 20"	20.13"
Backrest 21"	21.13"
Backrest 22"	22.13"

This Rogue Fixed Height Backrest Frame is discontinued and was used on serial numbers prior to RG0016612.

Item Number	Part Number	Description	Quantity
1a	000574	12" Extended Rigid Backrest	1
1b	000575	13" Extended Rigid Backrest	1
1c	000576	14" Extended Rigid Backrest	1
1d	000577	15" Extended Rigid Backrest	1
1e	000578	16" Extended Rigid Backrest	1
1f	000579	17" Extended Rigid Backrest	1
1g	000580	18" Extended Rigid Backrest	1
1h	000581	19" Extended Rigid Backrest	1
1i	000582	20" Extended Rigid Backrest	1
2	100804	Tube End Plug 1"	2