




Rogue2 Tipping Lever

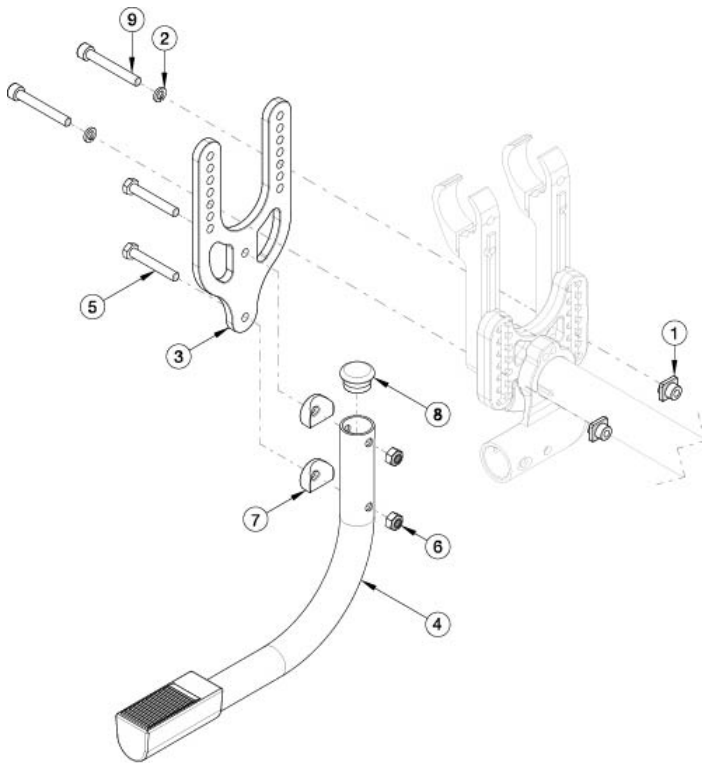


Standard

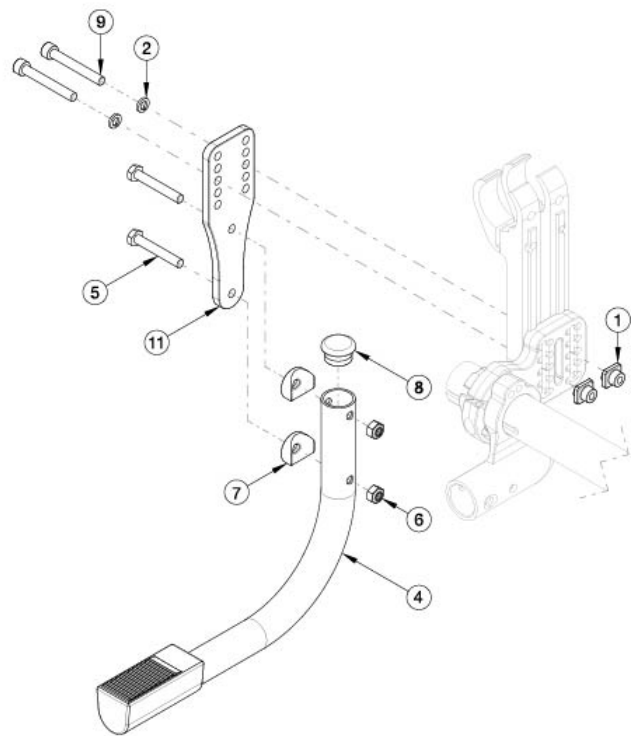

Amputee

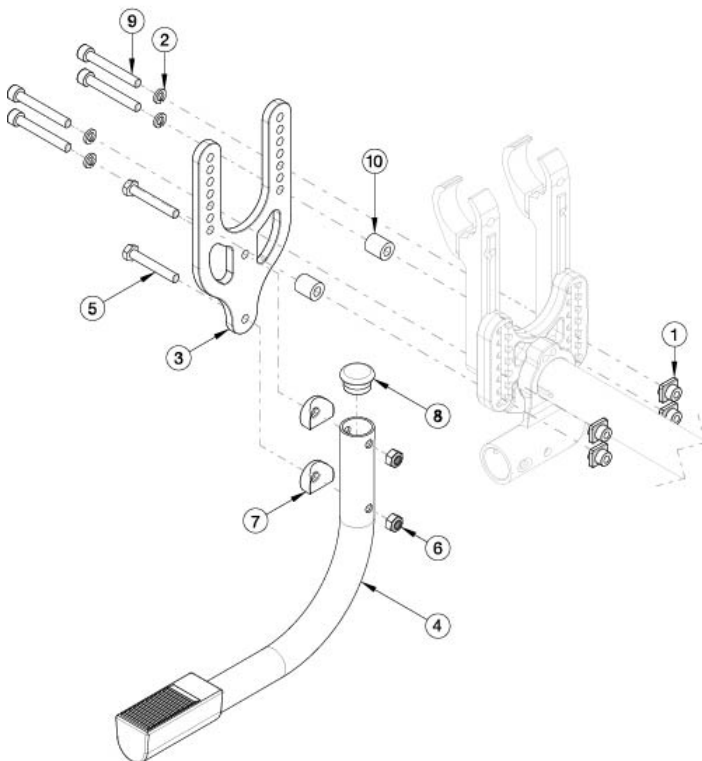

Standard with Spacers

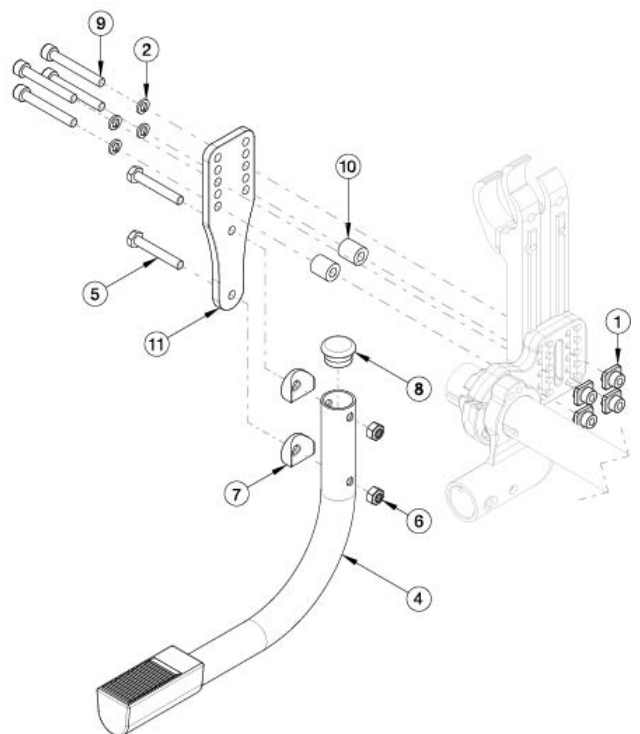

Amputee with Spacers

Not available if right and left User Activated Anti Tips are present

Item Number	Part Number	Description	Quantity
1 - 4a, 5 - 9	119636	Tipping Lever Standard Camber Mount Kit Rogue2 <i>Used with 20" or 22" wheels</i>	1
1 - 4b, 5 - 9	120813	Tipping Lever Standard Camber Mount Short Kit Rogue <i>Used with 24"-26" wheels</i>	1
1, 2, 4a, 5-9, 11	119635	Tipping Lever Amputee Camber Mount Kit Rogue2 <i>Used with 20" or 22" wheels</i>	1
1, 2, 4b, 5-9, 11	120812	Tipping Lever Amputee Camber Mount Short Kit Rogue2 / Rogue XP <i>Used with 24"-26" wheels</i>	1
1	114014	Rear Tower Square Nut	2
2	102810	M6 Lockwasher, Hi-Collar, ZC	2
3	119608	Rogue2 Tipping Lever Adapter Plate (Standard)	1
4a, 8	121382	Tipping Lever Tube Rogue2 <i>Used with 20" or 22" wheels</i>	1
4b, 8	121383	Tipping Lever Tube Short Rogue2 / Rogue XP <i>Used with 24"-26" wheels</i>	1
5	100681	M6 x 40 CLS8.8 DIN931 HHCS ZC	2
6	100658	M6 Nylock Nut Blk Zn	2
7	000054	Saddle 7/8"	2
9	101472	M6 x 45 SHCS BLK ZC	2
10	119613	Spacer, ½" OD, 9/16" Long, 2.57" ID, Black	2
11	119607	Rogue2 Tipping Lever Adapter Plate (Amputee)	1