


## CR45 Handrims - Aluminum Anodized

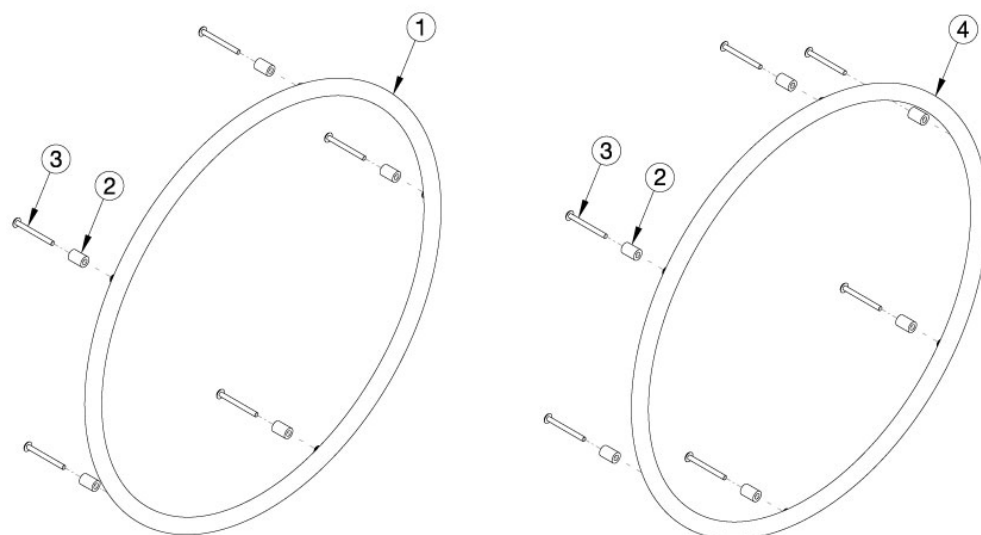

Aluminum Handrim with Micro Grip



| Item Number | Part Number | Description                                                                                                                      | Quantity |
|-------------|-------------|----------------------------------------------------------------------------------------------------------------------------------|----------|
| 2a, 3a      | 502171      | Handrim Hardware Kit, 5 Rivet Nut, EACH<br><i>Used with X Core Mag wheels with standard handrim mount option</i>                 | 1        |
| 2b, 3b      | 113070      | Handrim Hardware Kit, Close Mount, 5 Rivet Nut, EACH<br><i>Used with X Core Mag wheels with close mount handrim mount option</i> | 1        |
| 1a          | 101897      | Handrim Aluminum Anodized 5 Rivet Nut 20"<br><i>Used with 20" 5-Spoke X Core Mag wheels</i>                                      | 1        |
| 2a, 3b      | 117377      | Handrim Hardware Kit, Standard Mount, MAXX Mag, EACH<br><i>Used with MAXX Mag wheels with standard handrim mount option</i>      | 1        |
| 2b, 3c      | 117378      | Handrim Hardware Kit, Close Mount, MAXX Mag, EACH<br><i>Used with MAXX Mag wheels with close mount handrim mount option</i>      | 1        |
| 4a          | 200536      | Handrim Aluminum Anodized 6 Rivet Nut 20"<br><i>Used with 20" MAXX Mag wheels</i>                                                | 1        |
| 4a          | 502246      | Handrim Aluminum Anodized 6 Rivet Nut 20" w/ Micro Grip<br><i>Used with 20" MAXX Mag wheels</i>                                  | 1        |
| 4b          | 100560      | Handrim Aluminum Anodized 6 Rivet Nut 22"<br><i>Used with 22" MAXX Mag wheels</i>                                                | 1        |
| 4b          | 502247      | Handrim Aluminum Anodized 6 Rivet Nut 22" w/ Micro Grip<br><i>Used with 22" MAXX Mag wheels</i>                                  | 1        |
| 4c          | 105137      | Handrim Aluminum Anodized 6 Rivet Nut 24"                                                                                        | 1        |

| Item Number | Part Number | Description                                                                                                                                  | Quantity |
|-------------|-------------|----------------------------------------------------------------------------------------------------------------------------------------------|----------|
|             |             | <i>Used with 24" MAXX Mag wheels</i>                                                                                                         |          |
| 4c          | 117104      | Handrim Aluminum Anodized 6 Rivet Nut 24" w/<br>Micro Grip<br><i>Used with 24" MAXX Mag wheels</i>                                           | 1        |
| 2a          | 100629      | Handrim Spacer Alum Ano Riv<br><br><i>Used with X Core Mag (Qty 5 needed) and MAXX Mag (Qty 6 needed) with standard handrim mount option</i> | 1        |
| 3a          | 100724      | M5 x 55 PH TRS HD M/S ZC<br><br><i>Used with X Core Mag with standard handrim mount option</i>                                               | 5        |
| 3b          | 122064      | PHMS, M5x0.8x50, CLS4.8, PHL, BLZ, NYL,<br>DIN7985, Custom<br><i>Used with MAXX Mag with standard handrim mount option</i>                   | 6        |

| Handrim Hardware Chart |                |                    |                                                              |                                              |                                                                  |                   |                |                  |                  |                    |                  |                      |                  |          |  |
|------------------------|----------------|--------------------|--------------------------------------------------------------|----------------------------------------------|------------------------------------------------------------------|-------------------|----------------|------------------|------------------|--------------------|------------------|----------------------|------------------|----------|--|
| Wheel                  |                | Handrim Connection |                                                              |                                              |                                                                  | Handrim           |                |                  |                  |                    |                  |                      |                  |          |  |
| Wheel                  | Wheel Part No. | Connection Points  | Rivet/Tab                                                    | Spacer*                                      | Screw*                                                           | Aluminum Anodized | Superlight     | Plastic Coated   | Projection       | Ergonomic Standard | Ergonomic LT     | Natural Fit Standard | Natural Fit LT   | Flex Rim |  |
| 22" Spoke              | 116730         | 5 or 6             | 116732 (Used with 5 Rivet)<br><br>100698 (Used with 6 Rivet) | 100653 (Not used w/ Superlight Handrim)      | Aluminum, Plastic Ctd, Projection<br>Ergonomic: 122063           | 101898 (5 Rivet)  |                | 101964 (5 Rivet) | 112824 (6 Rivet) | 113082 (6 Rivet)   | 113085 (6 Rivet) | 200538 (6 Rivet)     | 200201 (6 Rivet) |          |  |
| 24" Spoke              | 116728         | 5 or 6             | Not used w/ Superlight Handrims                              |                                              | Natural Fit: 100835<br><br>Superlight: 100669<br><br>Nut: 100657 | 100975 (5 Rivet)  | 101161 (6 Tab) | 100976 (5 Rivet) | 112825 (6 Rivet) | 113083 (6 Rivet)   | 113086 (6 Rivet) | 100793 (6 Rivet)     | 200202 (6 Rivet) |          |  |
| 18" MAXX Spoke         | 109285         | 3                  | 100698 (Not used w/ Superlight Handrim)                      | 100653 (Not used w/ Superlight Handrim)      | Aluminum, Plastic Ctd, Projection, Ergonomic: 122063             | 100206            |                |                  |                  |                    |                  |                      |                  |          |  |
| 20" MAXX Spoke         | 108244         | 6                  |                                                              |                                              | Natural Fit: 100835                                              | 200536            |                | 112819           |                  |                    |                  |                      |                  |          |  |
| 22" MAXX Spoke         | 105135         |                    |                                                              |                                              | Superlight: 100669                                               | 100560            |                | 112820           | 112824           | 113082             | 113085           | 200538               | 200201           |          |  |
| 24" MAXX Spoke         | 105136         |                    |                                                              |                                              | Nut: 100657                                                      | 200349            | 101161         | 112821           | 112825           | 113083             | 113086           | 100793               | 200202           |          |  |
| 25" MAXX Spoke         | 107436         |                    |                                                              |                                              | 200350                                                           |                   | 112822         | 200548           | 113084           | 113087             | 200539           | 200540               |                  |          |  |
| 26" MAXX Spoke         | 107437         |                    |                                                              |                                              | 200351                                                           |                   | 112823         | 200549           |                  |                    | 100907           | 101454               |                  |          |  |
| 20" MAXX Mag           | 111853         | 6                  |                                                              | Aluminum, Plastic Coated, Projection: 100629 | Aluminum, Plastic Coated, Projection, Ergonomic: 122064          | 200536            |                | 112819           |                  |                    |                  |                      |                  |          |  |
| 22" MAXX Mag           | 111854         |                    |                                                              | Natural Fit & Ergonomic: 101756              | Natural Fit: 101893                                              | 100560            |                | 112820           | 112824           | 113082             | 113085           | 200538               | 200201           |          |  |
| 24" MAXX Mag           | 111855         |                    |                                                              |                                              |                                                                  | 105137            |                | 112821           | 112825           | 113083             | 113086           | 100793               | 200202           |          |  |
| 24" Superlight         | 101159         | 6                  |                                                              |                                              | 100536                                                           | 100754            | 101161         | 100836           |                  | 113080             | 113081           | 100830               | 100828           |          |  |
| 25" Superlight         | 101160         |                    |                                                              |                                              |                                                                  |                   |                | 101160           | 101091           |                    |                  |                      | 101464           | 101460   |  |

\* Spacer and and screw part numbers listed in the chart are for standard handrim mounting. If using close mount handrim mounting on aluminum anodized, plastic coated or projection handrims, use part number 100792 for the spacer, part number 122065 for the screw on Ki Spoke and MAXX Spoke wheels, part number 100666 for the screw on 5-Spoke X Core wheels, or part number 122063 for the screw on MAXX Mag wheels.

Effective 4/19/2024

| Handrim Hardware Chart (Continued) |                         |                    |           |         |                    |                   |            |                |            |                    |              |                      |                |                       |
|------------------------------------|-------------------------|--------------------|-----------|---------|--------------------|-------------------|------------|----------------|------------|--------------------|--------------|----------------------|----------------|-----------------------|
| Wheel                              |                         | Handrim Connection |           |         |                    | Handrim           |            |                |            |                    |              |                      |                |                       |
| Wheel                              | Wheel Part No.          | Connection Points  | Rivet/Tab | Spacer* | Screw*             | Aluminum Anodized | Superlight | Plastic Coated | Projection | Ergonomic Standard | Ergonomic LT | Natural Fit Standard | Natural Fit LT | Flex Rim              |
| 20" Spinerger Spox                 | See Spinerger Spox Page | 6                  |           |         | Ergonomic: 100536  | 103125            |            | 103179         |            |                    |              |                      |                |                       |
| 22" Spinerger Spox                 |                         |                    |           |         | All Others: 100669 | 100827            |            | 100808         |            |                    |              | 100889               | 100888         |                       |
| 24" Spinerger Spox                 |                         |                    |           |         |                    | 100766            | 101161     | 100615         |            |                    |              | 100830               | 100828         |                       |
| 25" Spinerger Spox                 |                         |                    |           |         | Nut: 100657        | 100767            | 101160     | 100765         |            |                    |              | 101464               | 101460         |                       |
| 26" Spinerger Spox                 |                         |                    |           |         |                    | 101477            |            | 101148         |            |                    |              | 200200               | 100950         |                       |
| 22" Spinerger LX                   | See Spinerger LX Page   | 6                  |           |         | Ergonomic: 100536  | 100827            |            | 100808         |            |                    |              | 100889               | 100888         |                       |
| 24" Spinerger LX                   |                         |                    |           |         | All Others: 100669 | 100766            | 101161     | 100615         |            | 113080             | 113081       | 100830               | 100828         | See Spinerger LX Page |
| 25" Spinerger LX                   |                         |                    |           |         |                    | 100767            | 101160     | 100765         |            |                    |              | 101464               | 101460         |                       |
| 26" Spinerger LX                   |                         |                    |           |         | Nut: 100657        | 101477            |            | 101148         |            |                    |              | 200200               | 100950         |                       |

| Handrim Hardware Chart (Discontinued Wheels) |                |                    |           |                                           |                                           |                     |            |                |            |                      |                |          |  |
|----------------------------------------------|----------------|--------------------|-----------|-------------------------------------------|-------------------------------------------|---------------------|------------|----------------|------------|----------------------|----------------|----------|--|
| Wheel                                        |                | Handrim Connection |           |                                           |                                           | Handrim             |            |                |            |                      |                |          |  |
| Wheel                                        | Wheel Part No. | Connection Points  | Rivet/Tab | Spacer*                                   | Screw*                                    | Aluminum Anodized   | Superlight | Plastic Coated | Projection | Natural Fit Standard | Natural Fit LT | Flex Rim |  |
| 18" Ki Spoke                                 | 200529         | 3                  | 100698    | 100653                                    | Aluminum, Plastic Ctd, Projection: 122063 | 100206              |            | 101106         |            |                      |                |          |  |
| 20" Ki Spoke                                 | 200530         | 6                  |           |                                           |                                           | Natural Fit: 100835 | 200536     |                | 200542     |                      |                |          |  |
| 22" Ki Spoke                                 | 200531         |                    |           |                                           |                                           |                     | 100560     |                | 100576     | 100569               | 200538         | 200201   |  |
| 24" Ki Spoke                                 | 200532         |                    |           |                                           |                                           |                     | 200349     |                | 100577     | 200547               | 100793         | 200202   |  |
| 25" Ki Spoke                                 | 200533         |                    |           |                                           | 200350                                    |                     |            | 101870         | 200548     | 200539               | 200540         |          |  |
| 26" Ki Spoke                                 | 200534         |                    |           |                                           | 200351                                    |                     |            | 100578         | 200549     | 100907               | 101454         |          |  |
| 20" 5-Spoke X Core                           | 101961         | 5                  |           | Aluminum, Projection, Natural Fit: 100629 | 100724                                    | 101897              |            | 101963         |            |                      |                |          |  |
| 22" 5-Spoke X Core                           | 101962         |                    |           |                                           |                                           | 101898              |            | 101964         |            |                      |                |          |  |
| 24" 5-Spoke X Core                           | 100960         |                    |           |                                           |                                           | 100975              |            | 100976         | 200546     |                      |                |          |  |
| 24" 5-Spoke X Core                           | 100960         |                    |           | Plastic Ctd: 101756                       | 101893                                    |                     |            |                |            | 100768               | 100769         |          |  |

\* Spacer and and screw part numbers listed in the chart are for standard handrim mounting. If using close mount handrim mounting on aluminum anodized, plastic coated or projection handrims, use part number 100792 for the spacer, part number 122065 for the screw on Ki Spoke and MAXX Spoke wheels, part number 100666 for the screw on 5-Spoke X Core wheels, or part number 122063 for the screw on MAXX Mag wheels.

Effective 4/19/2024