


# Spark Height Adjustable T-Arm

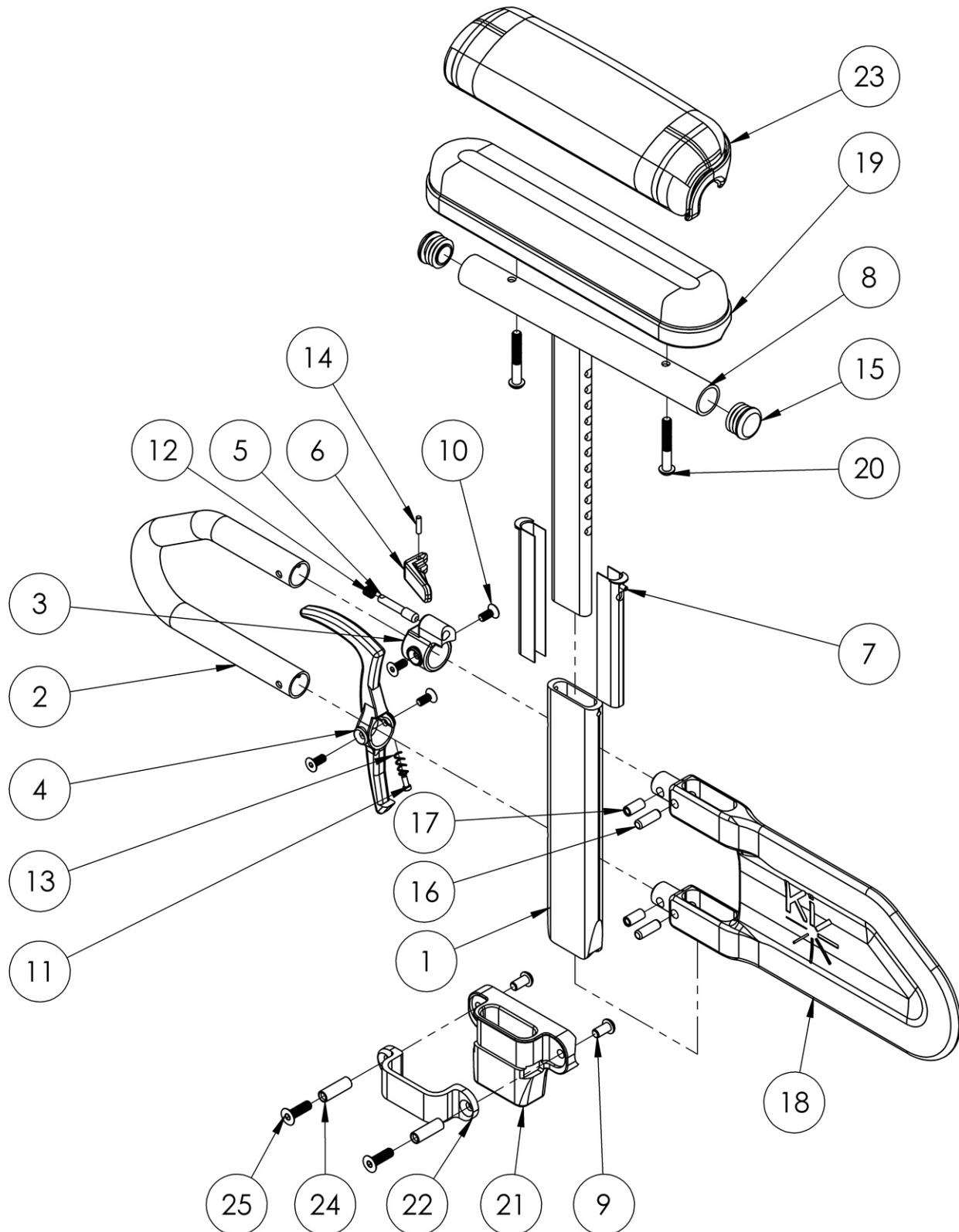

Short assemblies are used with 14-17 Seat Depths. Long assemblies are used with 18"-19" Seat Depths <p>

This design effective November 10, 2017 for long panels and December 6, 2017 for short panels <p>

If retrofitting to Height Adjustable T-Arms, new rear frames are required <p>

If retrofitting to Height Adjustable T-Arms and One Arm Drive is present, a new drive shaft may be required. In addition to a new drive shaft, order the corresponding hardware kit: 109439 for Blade Style One Arm Drive, 503433 for Hex Style One Arm Drive or 112898 for Quick Release Style One Arm Drive

Item Number	Part Number	Description	Quantity	Retail
1	000089	Height Adj T Armrest Post	1	\$25.10
2	000095	Height Adj T Armrest Transfer Tube	1	\$28.07
3, 5, 6, 10, 12, 14	505490	Height Adj T-Arm Upper Latch Assembly	1	\$40.56
7	000090	Height Adj T Armrest Shim	2	\$2.55
8a, 15	112906	Height Adj T Armrest Upper Desk w/ Tube End Plug	1	\$76.67
8b, 15	112907	Height Adj T Armrest Upper Full w/ Tube End Plug	1	\$76.67
9	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.58
10	003458	M5 X 12mm FHCS BLK ZN W/ Patch	4	\$1.05
11	000152	Height Adj T Armrest Release Spring Button	1	\$1.09
13	000135	Height Adj T Armrest Release Lever Spring	1	\$1.09
15	100803	Tube End Plug 7/8"	2	\$1.58
16	003641	.25" x .75" Dowel Pin	2	\$1.05
17	003457	M5 x 14mm Barrel Nut	2	\$1.05
19a	100500	Arm Pad Full	1	\$23.64
19b	100501	Arm Pad Desk	1	\$23.64
20a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2	\$1.58
20b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2	\$1.58
21	002723	Height Adjustable T-Arm Receiver Spark	1	\$15.25
22	000088	Height Adj T Armrest Bracket	1	\$27.42
23a	102452	Waterfall Arm Pad Full	1	\$32.80
23b	102521	Waterfall Arm Pad Desk	1	\$32.80
24	000065	M6 x 23.68 Threaded Barrel	2	\$4.13
25	100866	M6 x 20 CLS12.9 DIN7991 FSHCS Blk Zc w/Patch	2	\$2.63

Item Number	Part Number	Description	Quantity	Retail
4, 10, 11, 13	505507	Height Adj T-Arm Release Lever Assembly	1	\$8.49
16, 17, 18a	505508	Height Adj T-Arm Long Side Shield Assembly	1	\$39.18
16, 17, 18b	506082	Height Adj T-Arm Short Shield Assembly	1	\$16.44
9, 21, 22, 24, 25	505276	Height Adjustable T-Arm Receiver Assembly 1" Spark	1	\$55.66
1-7, 10-14, 16, 17, 18b	004160	Height Adjustable T-Arm Lower Assembly Short <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1	\$132.94
1-7, 10-14, 16, 17, 18a	004161	Height Adjustable T-Arm Lower Assembly Long <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1	\$132.94
1-7, 8a, 10-18a	003465	Height Adjustable T-Arm Long Desk <i>Does not include arm pad or arm pad screws</i>	1	\$207.98
1-7, 8b, 10-18a	003466	Height Adjustable T-Arm Long Full <i>Does not include arm pad or arm pad screws</i>	1	\$207.98
1-7, 8a, 10-18b	003463	Height Adjustable T-Arm Short Desk <i>Does not include arm pad or arm pad screws</i>	1	\$207.98
1-7, 8b, 10-18b	003464	Height Adjustable T-Arm Short Full <i>Does not include arm pad or arm pad screws</i>	1	\$207.98
1-7, 8a, 10-18a, 19a, 20a	505491	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full	1	\$234.78
1-7, 8a, 10-18a, 19a, 20a	505492	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full, PAIR	1	\$469.57
1-7, 8a, 10-18b, 19a, 20a	505644	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full	1	\$234.78
1-7, 8a, 10-18b, 19a, 20a	505645	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, PAIR	1	\$469.57
1-7, 8a, 10-18a, 19b, 20a	505493	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk	1	\$234.78
1-7, 8a, 10-18a, 19b, 20a	505494	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk, PAIR	1	\$469.57
1-7, 8a, 10-18b, 19b, 20a	505646	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk	1	\$234.78
1-7, 8a, 10-18b, 19b, 20a	505647	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, PAIR	1	\$469.57
1-7, 8a, 10-18a, 20b, 23a	505495	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full	1	\$243.94
1-7, 8a, 10-18a, 20b, 23a	505496	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full, PAIR	1	\$487.89
1-7, 8a, 10-18b, 20b, 23a	505648	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full	1	\$243.94
1-7, 8a, 10-18b, 20b, 23a	505649	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, PAIR	1	\$487.89
1-7, 8a, 10-18a, 20b, 23b	505497	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk	1	\$243.94
1-7, 8a, 10-18a, 20b, 23b	505498	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk, PAIR	1	\$487.89

Item Number	Part Number	Description	Quantity	Retail
1-7, 8a, 10-18b, 20b, 23b	505650	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk	1	\$243.94
1-7, 8a, 10-18b, 20b, 23b	505651	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, PAIR	1	\$487.89
1-7, 8a, 9-18a, 19a, 20a, 21, 22, 24, 25	505509	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full, 1" Receiver Spark	1	\$270.39
1-7, 8a, 9-18a, 19a, 20a, 21, 22, 24, 25	505510	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full, 1" Receiver, PAIR Spark	1	\$595.36
1-7, 8a, 9-18b, 19a, 20a, 21, 22, 24, 25	506057	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, 1" Receiver Spark	1	\$270.39
1-7, 8a, 9-18b, 19a, 20a, 21, 22, 24, 25	506058	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, 1" Receiver, PAIR Spark	1	\$595.36
1-7, 8a, 9-18a, 19b, 20a, 21, 22, 24, 25	505511	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk, 1" Receiver Spark	1	\$270.39
1-7, 8a, 9-18a, 19b, 20a, 21, 22, 24, 25	505512	Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk, 1" Receiver, PAIR Spark	1	\$595.36
1-7, 8a, 9-18b, 19b, 20a, 21, 22, 24, 25	506059	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, 1" Receiver Spark	1	\$270.39
1-7, 8a, 9-18b, 19b, 20a, 21, 22, 24, 25	506060	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, 1" Receiver, PAIR Spark	1	\$595.36
1-7, 8a, 9-18a, 20b, 21, 22, 23a, 24, 25	505513	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full, 1" Receiver Spark	1	\$280.02
1-7, 8a, 9-18a, 20b, 21, 22, 23a, 24, 25	505514	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full, 1" Receiver, PAIR Spark	1	\$614.25
1-7, 8a, 9-18b, 20b, 21, 22, 23a, 24, 25	506061	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, 1" Receiver Spark	1	\$280.02
1-7, 8a, 9-18b, 20b, 21, 22, 23a, 24, 25	506062	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, 1" Receiver, PAIR Spark	1	\$614.25
1-7, 8a, 9-18a, 20b, 21, 22, 23b, 24, 25	505515	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk, 1" Receiver Spark	1	\$280.02
1-7, 8a, 9-18a, 20b, 21, 22, 23b, 24, 25	505516	Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk, 1" Receiver, PAIR Spark	1	\$614.25
1-7, 8a, 9-18b, 20b, 21, 22, 23b, 24, 25	506063	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, 1" Receiver Spark	1	\$280.02
1-7, 8a, 9-18b, 20b, 21, 22, 23b, 24, 25	506064	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, 1" Receiver, PAIR Spark	1	\$614.25