


(Discontinued 1) Focus CR Height Adjustable Backrest

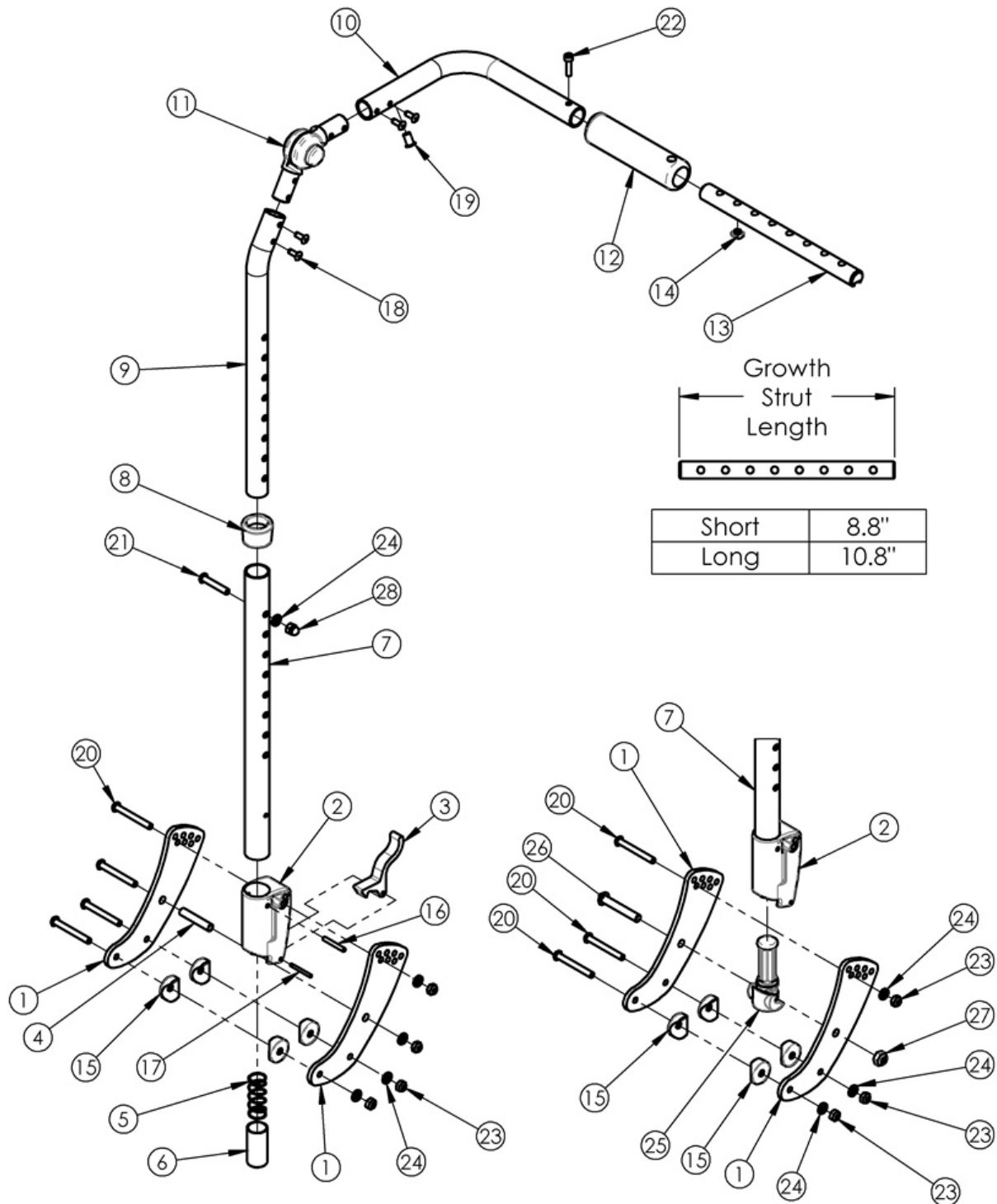

This Focus CR Height Adjustable Backrest was effective prior to October 2015.

Item Number	Part Number	Description	Quantity
1, 4, 15, 20, 23, 24	501094	Backrest Hardware Kit Std Focus	1
1, 15, 20, 23, 24, 26, 27	501406	Backrest Hardware Kit, Std Non Folding, Pair Focus	1
1	001071	Back Plate Std TILT	2
4	001075	Stop Pin Backrest Tilt	1
8	001083	Bushing Height Adjustable Back Tilt	1
15	000055	Saddle 1"	4
20	101781	M6 x 50mm Partial Thread BHCS Blk ZC	4
21	101811	M6 x 35 BHCS Blk Zn	1
22	101843	M5 x 20 SHCS Blk Zn	1
23	100658	M6 Nylock Nut Blk Zn	4
24	100746	M6 Flat Washer Blk Zn	4
26	101815	M8 x 50 BHCS Blk Zn	1
27	100659	M8 Nylock Nut Blk Zn	1
28	100558	M6 Nyl Ins Domed Cap L/N Zc	1