


Catalyst E Seat Upholstery

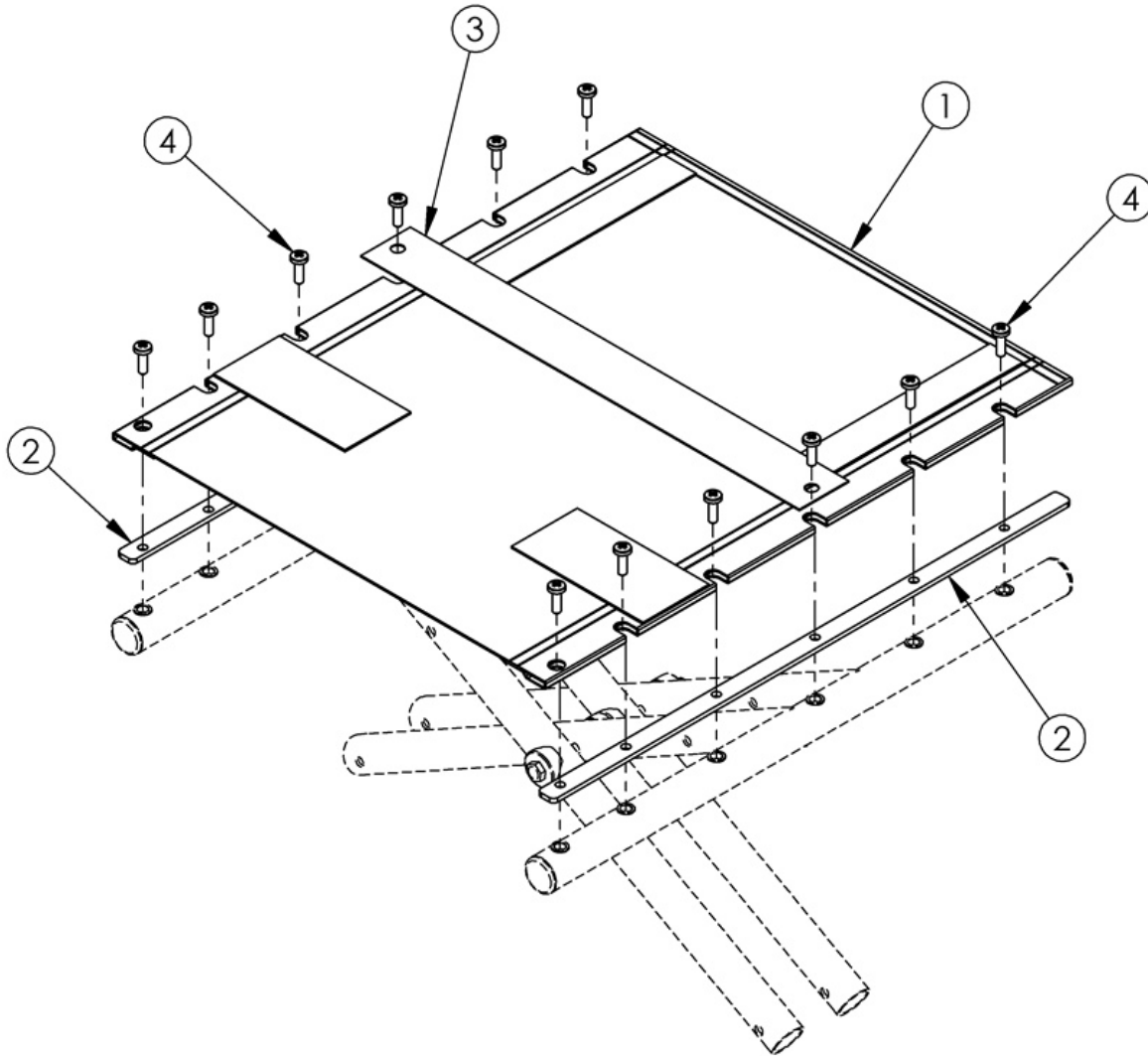

SCREW (ITEM 4) QTY PER CHAIR	
SEAT DEPTH	SCREW QTY
14" - 15"	10
16" - 17"	12
18" - 19"	14
20"	16

Seat upholstery width measures approximately 2.5" less than the chair size. Seat upholstery depth measures true to the chair size.

Item Number	Part Number	Description	Quantity
1a	100095	Seat Upholstery Cat 1416	1

Item Number	Part Number	Description	Quantity
1b	100097	Seat Upholstery Cat 1418	1
1c	100100	Seat Upholstery Cat 1516	1
1d	100102	Seat Upholstery Cat 1518	1
1e	100104	Seat Upholstery Cat 1520	1
1f	100107	Seat Upholstery Cat 1616	1
1g	100109	Seat Upholstery Cat 1618	1
1h	100111	Seat Upholstery Cat 1620	1
1i	100114	Seat Upholstery Cat 1716	1
1j	100116	Seat Upholstery Cat 1718	1
1k	100118	Seat Upholstery Cat 1720	1
1l	100121	Seat Upholstery Cat 1816	1
1m	100123	Seat Upholstery Cat 1818	1
1n	100125	Seat Upholstery Cat 1820	1
1o	100126	Seat Upholstery Cat 1916	1
1p	100128	Seat Upholstery Cat 1918	1
1q	100130	Seat Upholstery Cat 1920	1
1r	100131	Seat Upholstery Cat 2016	1
1s	100133	Seat Upholstery Cat 2018	1
1t	100135	Seat Upholstery Cat 2020	1
1u	101450	Seat Upholstery Cat 2116	1
1v	100855	Seat Upholstery Cat 2118	1
1w	100857	Seat Upholstery Cat 2120	1
1x	101556	Seat Upholstery Cat 2216	1
1y	100247	Seat Upholstery Cat 2218	1
1z	100261	Seat Upholstery Cat 2220	1
2a	100232	Seat Rail 16" Deep CAT	2
2b	100234	Seat Rail 18" Deep CAT	2
2c	100236	Seat Rail 20" Deep CAT	2

Item Number	Part Number	Description	Quantity
3a	000059	Lifting Strap 14" Wide CAT	1
3b	000060	Lifting Strap 15" Wide CAT	1
3c	000061	Lifting Strap 16" Wide CAT	1
3d	000062	Lifting Strap 17" Wide CAT	1
3e	000063	Lifting Strap 18" Wide CAT	1
3f	000064	Lifting Strap 19" Wide CAT	1
3g	000058	Lifting Strap 20" Wide CAT	1
3h	100248	Lifting Strap 21" Wide CAT	1
3i	100875	Lifting Strap 22" Wide CAT	1
4	100734	M5 x 16 PHMS ZC	16