


Arc Anti-Tips

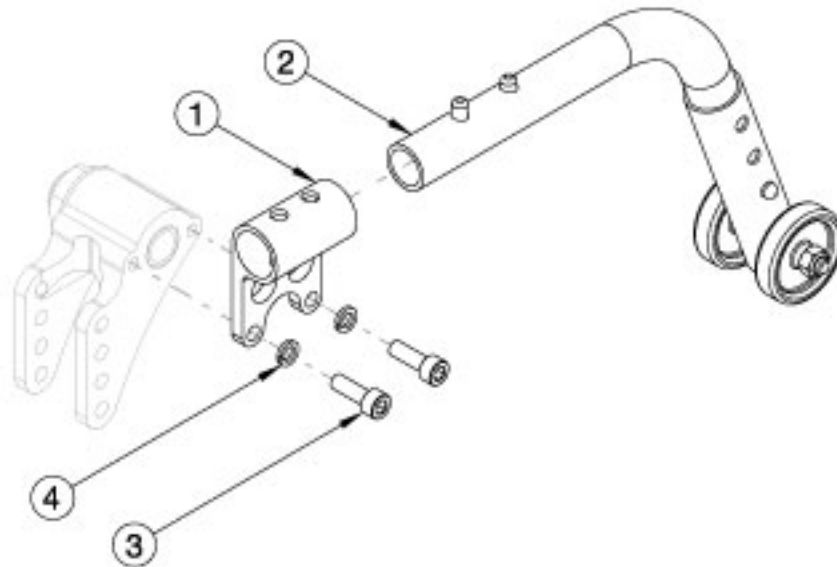

Item Number	Part Number	Description	Quantity
1, 2a, 3, 4	113189	Anti-Tip Assembly, 12", Arc	1
1, 2b, 3, 4	113190	Anti-Tip Assembly, 16", Arc	1
1, 2c, 3, 4	113191	Anti-Tip Assembly, 20", Arc	1
1	110189	Anti-Tip Receiver	1
2b	119679	Anti-Tip Assembly, 22-24", Liberty, Arc, Cr45	2
3	101844	M6 x 18 SHCS Blk Zn	2
4	102810	M6 Lockwasher, Hi-Collar, ZC	2