


Discontinued Focus CR Attendant Foot Lock

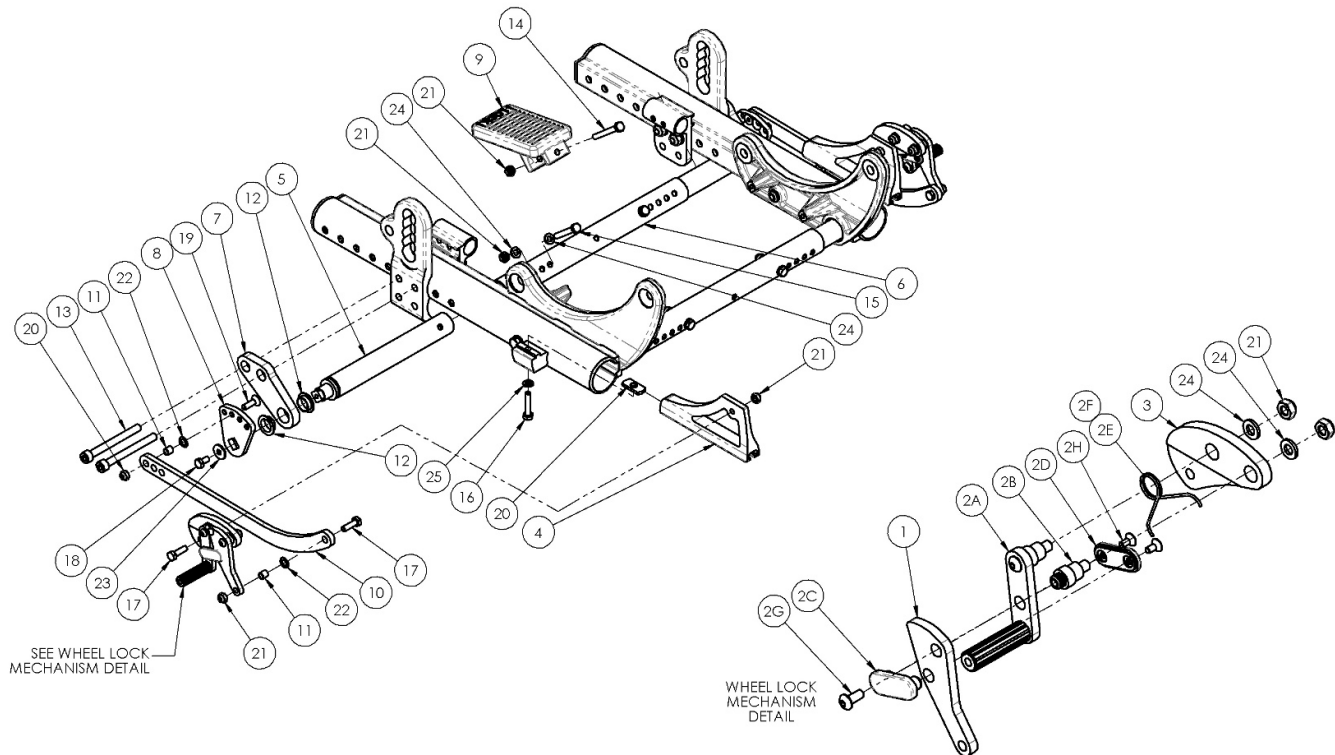

This Focus CR attendant Foot Lock is discontinued. Please contact Customer Service for more information.

| Item Number | Part Number | Description | Quantity | Retail |
|----------------|-------------|---|----------|---------|
| 7a, 12 | 501121 | Foot Lock Mount Assembly w/ Bearing Right Focus | 1 | \$76.38 |
| 7b, 12 | 501122 | Foot Lock Mount Assembly w/ Bearing Left Focus | 1 | \$76.37 |
| 6a, 15, 21, 24 | 501123 | Base/Axle/Foot Lock Strut Assembly Small Focus | 1 | \$24.78 |
| 6b, 15, 21, 24 | 501124 | Base/Axle/Foot Lock Strut Assembly Medium Focus | 1 | \$24.78 |
| 6c, 15, 21, 24 | 501125 | Base/Axle/Foot Lock Strut Assembly Large Focus | 1 | \$24.78 |
| 1a, 2A-2H | 001142 | Wheel Lock Foot Right Tilt | 1 | \$83.53 |
| 1b, 2A - 2H | 001143 | Wheel Lock Foot Left Tilt | 1 | \$83.53 |
| 4a | 001138 | Plate Wheel Lock Right Tilt | 1 | \$16.32 |
| 4b | 001139 | Plate Wheel Lock Left Tilt | 1 | \$16.32 |
| 5 | 001145 | Strut Foot Lock Tilt | 2 | \$75.33 |

| Item Number | Part Number | Description | Quantity | Retail |
|-------------|-------------|---|----------|---------|
| 6a | 001011 | Growth Strut, Base, Small Tilt | 1 | \$18.91 |
| 6b | 001012 | Growth Strut, Base, Medium Tilt | 1 | \$18.91 |
| 6c | 001013 | Growth Strut, Base, Large Tilt | 1 | \$18.91 |
| 8 | 001150 | Index Plate Foot Lock Tilt | 2 | \$33.05 |
| 11 | 001156 | Pivot Bushing Tilt | 4 | \$5.62 |
| 12 | 001157 | IGUS Bushing GFM-1618-06 | 4 | \$15.76 |
| 13a | 101851 | M8 x 80 SHCS Blk Zn | 4 | \$2.10 |
| 13b | 101852 | M8 x 90 SHCS Blk Zn | 4 | \$4.20 |
| 14 | 101826 | M6 x 50 HHCS Blk Zn | 1 | \$2.10 |
| 15 | 101825 | M6 x 40 HHCS Blk Zn | 2 | \$1.05 |
| 16 | 101824 | M6 x 30 HHCS Blk Zn | 2 | \$2.10 |
| 17 | 101823 | M6 x 20 HHCS Blk Zn | 6 | \$1.05 |
| 18 | 101822 | M6 x 12 HHCS BLKZC w/Patch | 2 | \$1.05 |
| 19 | 100866 | M6 x 20 CLS12.9 DIN7991 FSHCS Blk Zc w/Patch | 2 | \$2.63 |
| 20 | 001141 | Weld Nut NDM 06028 Zinc | 2 | \$2.63 |
| 21 | 100658 | M6 Nylock Nut Blk Zn | 13 | \$1.05 |
| 22 | 101861 | Nylon Washer 5/16 x 1/2 x .03 | 4 | \$0.53 |
| 23 | 101853 | M6 Washer Blk Zn | 2 | \$0.53 |
| 24 | 100746 | M6 Flat Washer Blk Zn | 8 | \$1.58 |
| 25 | 101835 | M6 Lock Washer Blk Zn | 2 | \$0.53 |