


Catalyst Height Adjustable Low T-Arm

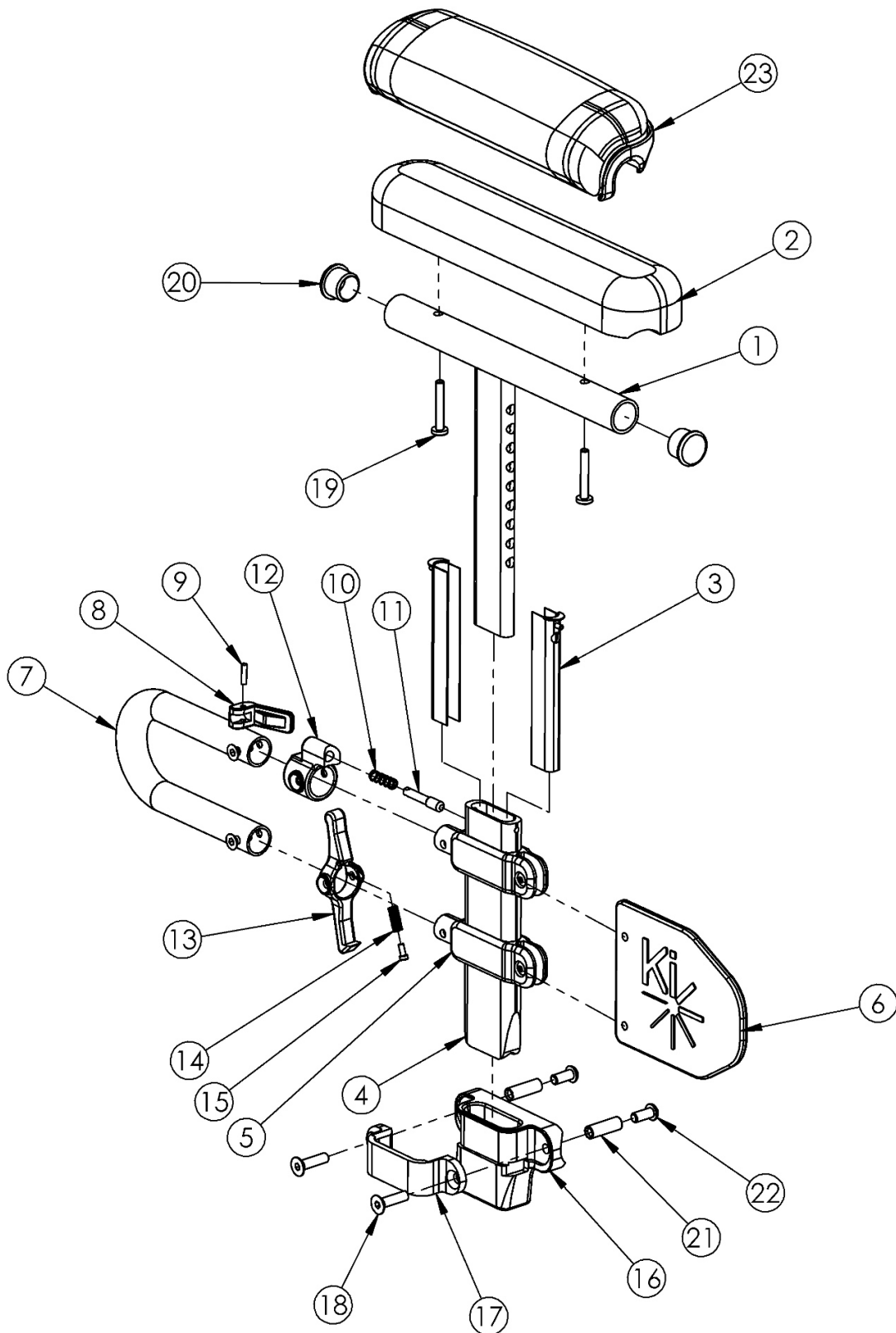

If retrofitting to Height Adjustable Low T-Arms and One Arm Drive is present, a new drive shaft may be required. In addition to a new drive shaft, order the corresponding hardware kit: 109438 for Blade Style One Arm Drive, 503433 for Hex Style One Arm Drive or 112898 for Quick Release Style One Arm Drive

Item Number	Part Number	Description	Quantity	Retail
1a, 2b, 3-18, 19a, 20-22	113452	Height Adj T-Arm Assm Low, Desk w/ Std Pad and Receiver PAIR	1	\$440.32
1a, 2b, 3-18, 19a, 20-22	113347	Height Adj T-Arm Assm Low, Desk w/ Std Pad and Receiver Right	1	\$220.16
1a, 2b, 3-18, 19a, 20-22	113348	Height Adj T-Arm Assm Low, Desk w/ Std Pad and Receiver Left	1	\$220.16
1a, 2b, 3-15, 19a, 20	503167	Height Adj T-Arm Assm Low, Desk w/ Std Pad Right	1	\$160.81
1a, 2b, 3-15, 19a, 20	503168	Height Adj T-Arm Assm Low, Desk w/ Std Pad Left	1	\$160.81
1b, 2a, 3-18, 19a, 20-22	113453	Height Adj T-Arm Assm Low, Full w/ Std Pad and Receiver PAIR	1	\$440.32
1b, 2a, 3-18, 19a, 20-22	113349	Height Adj T-Arm Assm Low, Full w/ Std Pad and Receiver Right	1	\$220.16
1b, 2a, 3-18, 19a, 20-22	113350	Height Adj T-Arm Assm Low, Full w/ Std Pad and Receiver Left	1	\$220.16
1b, 2a, 3-15, 19a, 20	503169	Height Adj T-Arm Assm Low, Full w/ Std Pad Right	1	\$160.81
1b, 2a, 3-15, 19a, 20	503170	Height Adj T-Arm Assm Low, Full w/ Std Pad Left	1	\$160.81
1a, 3-18, 19b, 20-22, 23b	113454	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad and Receiver PAIR	1	\$458.64
1a, 3-18, 19b, 20-22, 23b	113351	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad and Receiver Right	1	\$229.32
1a, 3-18, 19b, 20-22, 23b	113352	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad and Receiver Left	1	\$229.32
1a, 3-15, 19b, 20, 23b	503183	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad Right	1	\$169.97
1a, 3-15, 19b, 20, 23b	503184	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad Left	1	\$169.97
1b, 3-18, 19b, 20-22, 23a	113455	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad and Receiver PAIR	1	\$458.64
1b, 3-18, 19b, 20-22, 23a	113353	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad and Receiver Right	1	\$229.32
1b, 3-18, 19b, 20-22, 23a	113354	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad and Receiver Left	1	\$229.32
1b, 3-15, 19b, 20, 23a	503185	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad Right	1	\$169.97
1b, 3-15, 19b, 20, 23a	503186	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad Left	1	\$169.97
1a, 3-15, 20	002681	Arm T Height Adj Low, Desk Right Assembly	1	\$134.01
1a, 3-15, 20	002682	Arm T Height Adj Low, Desk Left Assembly	1	\$134.01
1b, 3-15, 20	002683	Arm T Height Adj Low, Full Right Assembly	1	\$134.01
1b, 3-15, 20	002684	Arm T Height Adj Low, Full Left Assembly	1	\$134.01
16-18, 21, 22	501730	Height Adj T-Arm Receiver Assm 1" CAT	1	\$55.66

Item Number	Part Number	Description	Quantity	Retail
1a, 20	002648	Post Weldment, T-Arm, Low, Desk	1	\$67.08
1b, 20	002649	Post Weldment, T-Arm, Low, Full	1	\$69.35
2a	100500	Arm Pad Full	1	\$23.64
2b	100501	Arm Pad Desk	1	\$23.64
3	000090	Height Adj T Armrest Shim	2	\$2.55
4	002105	Tube, Outer, T-Arm, Low	1	\$29.62
5	000092	Height Adj T Armrest Clamp	2	\$8.97
6	002107	Sideguard, T-Arm, Low	1	\$8.10
7	002104	Transfer Tube, T-Arm, Low	1	\$16.81
8	000098	Height Adj Armrest Lever	1	\$3.58
9	200257	Dowel Pin 1/8 x 1/2	1	\$2.63
10	000134	Height Adj Armrest Lever Spring	1	\$1.11
11	000099	Height Adj Armrest Lever Pin	1	\$19.13
12	000096	Height Adj T Armrest Pin Lock Housing	1	\$14.71
8-12	500541	Arm Height Adj T Upper Lever Assembly	1	\$40.56
13	002106	Lever, T-Arm, Low	1	\$3.47
14	000135	Height Adj T Armrest Release Lever Spring	1	\$1.09
15	000152	Height Adj T Armrest Release Spring Button	1	\$1.09
16	000087	Height Adjustable T-Arm Receiver 1" CAT	1	\$15.25
17	000088	Height Adj T Armrest Bracket	1	\$27.42
18	100866	M6 x 20 CLS12.9 DIN7991 FSHCS Blk Zc w/Patch	2	\$2.63
19a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2	\$1.58
19b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2	\$1.58
20	100803	Tube End Plug 7/8"	2	\$1.58
21	000065	M6 x 23.68 Threaded Barrel	2	\$4.13
22	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.58

Item Number	Part Number	Description	Quantity	Retail
23a	102452	Waterfall Arm Pad Full	1	\$32.80
23b	102521	Waterfall Arm Pad Desk	1	\$32.80