


## CR45 Footplates - Composite Angle Adjustable

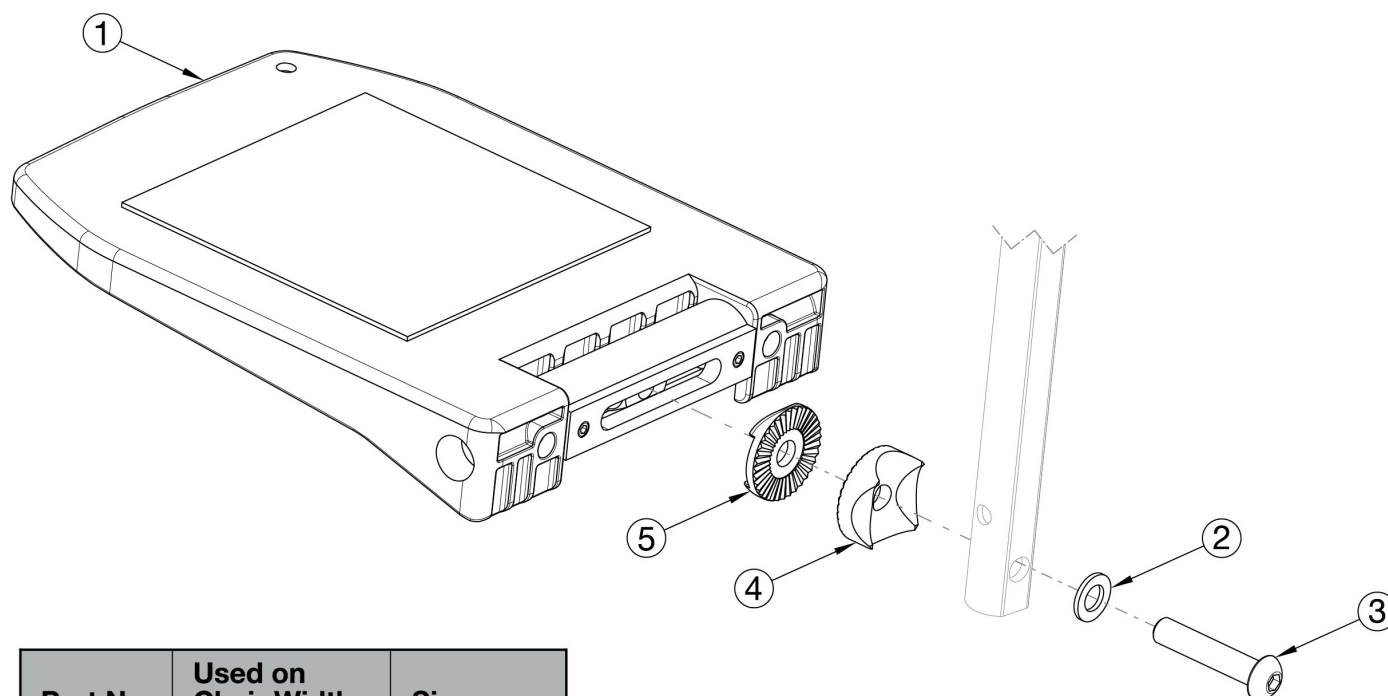

| Part No. | Used on Chair Widths | Size        |
|----------|----------------------|-------------|
| 003173   | 15" - 16"            | 6.3" x 6.3" |
| 003174   | 17" - 18"            | 7.1" x 6.3" |
| 003175   | 19" - 22"            | 8.1" x 6.3" |

If changing to Composite Angle Adjustable Footplates, new extension tubes are required. Sizes for footplates are based on seat width. Only compatible with extension tube hangers.

| Item Number | Part Number | Description                                                                                                                                                                       | Quantity |
|-------------|-------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------|
| 2 - 5       | 503726      | Extension Tube Hardware Assembly for Composite Angle Adjustable Footplate<br><i>Used only for Extension Tube Mount Hangers for Pro ELR assembly please go to Pro ELR PM page.</i> | 1        |
| 1a          | 003173      | Composite Angle Adjustable Footplate Small 15-16" Wide                                                                                                                            | 1        |
| 1b          | 003174      | Composite Angle Adjustable Footplate Medium 17-18" Wide                                                                                                                           | 1        |
| 1c          | 003175      | Composite Angle Adjustable Footplate Large 19-24" Wide                                                                                                                            | 1        |
| 2           | 003188      | Washer D8.4 x 1.6 MM                                                                                                                                                              | 1        |
| 3           | 003189      | Bolt M8 x 45MM, Galvanized Steel                                                                                                                                                  | 1        |
| 4           | 003187      | Locking Wheel w/Radius, R12.5 & R10, D30 x 11.3, Cast Steel, Black                                                                                                                | 1        |
| 5           | 003186      | Locking Wheel w/Hexagon, D30 x 6.8/6, Cast Steel, Black                                                                                                                           | 1        |