


# Catalyst 5 Curved Rear Frame

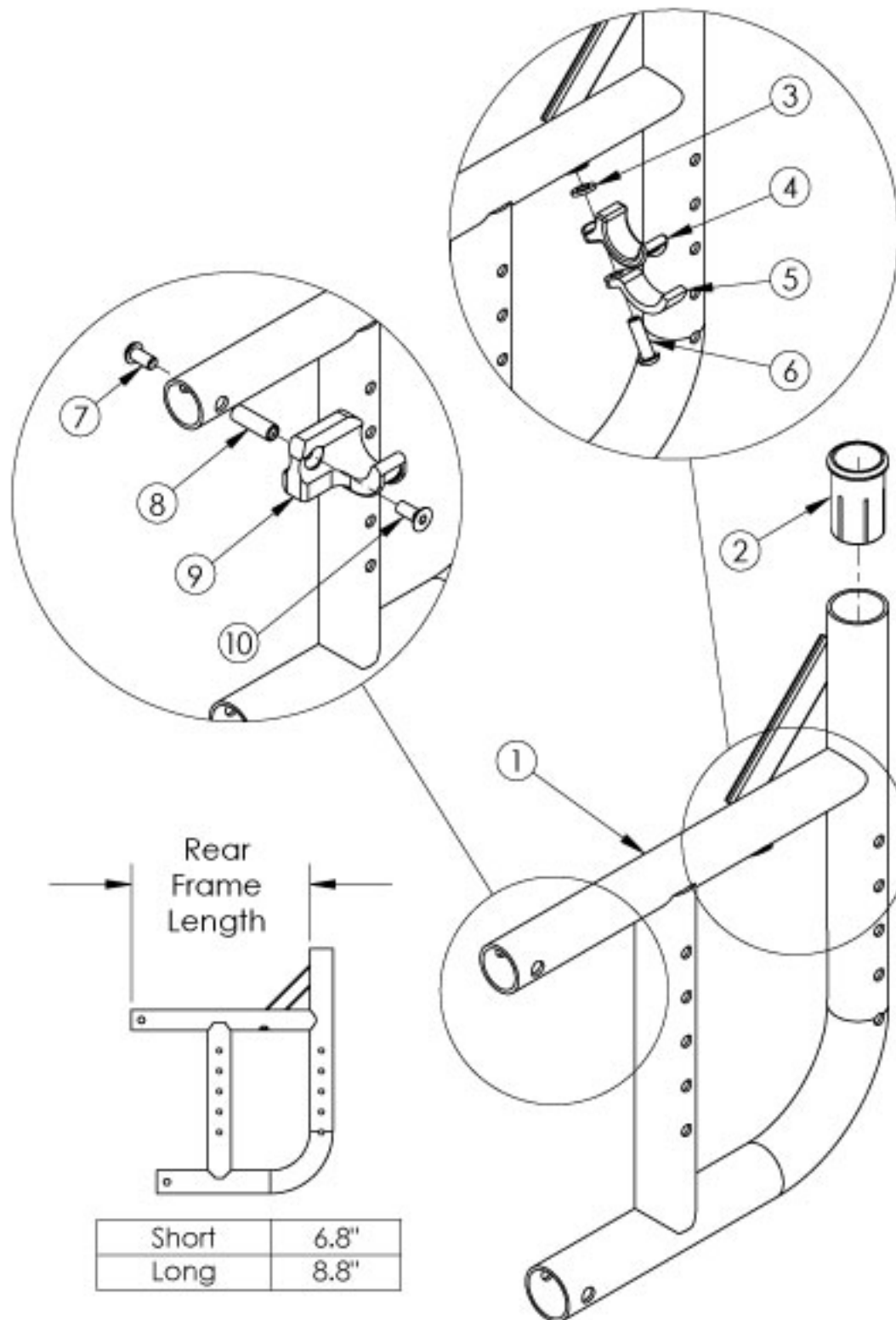

<p>

If chair has any type of direct mounted side guards, select a frame assembly with the Side Guard Modification.<p>

Use short frame for 14-17" Seat Depths. Use long frame for 18-20" Seat Depths<p>

If switching to a new Seat Depth, new Height Adjustable Flip Back T-Arms or Height Adjustable T-Arms may be required.

Item Number	Part Number	Description	Quantity	Retail
3, 4, 5, 6a	501005	Seat Hook and Support Hardware Kit, Standard, Catalyst <i>Used with Standard Weight Capacity. This kit contains quantity of 1 each</i>	1	\$17.40
3, 4, 5, 6b	114528	Seat Hook and Support Hardware Kit, Standard, for Heavy Duty Catalyst <i>Used with Heavy Duty / 350 lbs Weight Capacity. This kit contains quantity of 1 each</i>	1	\$17.93
7, 8, 9a, 10	111569	Seat Hook and Support Hardware Kit, Seating System, Right Side, Catalyst <i>This kit contains quantity of 1 each</i>	1	\$29.56
7, 8, 9b, 10	111570	Seat Hook and Support Hardware Kit, Seating System, Left Side, Catalyst <i>This kit contains quantity of 1 each</i>	1	\$29.56
1a, 2	100166_ _ _	Frame Assembly, Cat 5 Rear, Left, Short	1	\$219.90 - \$336.44
1c, 2	100136_ _ _	Frame Assembly, Cat 5 Rear, Right, Short	1	\$219.90 - \$336.44
1b, 2	100167_ _ _	Frame Assembly, Cat 5 Rear, Left, Long	1	\$219.90 - \$336.44
1d, 2	100137_ _ _	Frame Assembly, Cat 5 Rear, Right, Long	1	\$219.90 - \$336.44
1a, 2	114408_ _ _	Frame Assembly, Cat 5 Rear with Side Guard Mount Modification, Left, Short	1	\$219.90 - \$336.44
1c, 2	114406_ _ _	Frame Assembly, Cat 5 Rear with Side Guard Mount Modification, Right, Short	1	\$219.90 - \$336.44
1b, 2	114409_ _ _	Frame Assembly, Cat 5 Rear with Side Guard Mount Modification, Left, Long	1	\$219.90 - \$336.44
1d, 2	114407_ _ _	Frame Assembly, Cat 5 Rear with Side Guard Mount Modification, Right, Long	1	\$219.90 - \$336.44
2	000030	Backrest Sleeve Cat	1	\$3.52
3	100831	1/2" Washer OD .040" Thick	1	\$1.58
4	000003	Seat Hook, Catalyst	1	\$7.70
5	000004	Seat Hook Support, Catalyst	1	\$6.55
6a	100668	M6 x 20 CLS10.9 BTN SHCS ZC w/Patch <i>Used with Standard Weight Capacity</i>	1	\$1.58
6b	101180	M6 x 20 SHCS BLK ZC <i>Used with Heavy Duty / 350 lbs Weight Capacity</i>	1	\$2.10
7	100667	M6 x 12 BTN SHCS BLKZC w/Patch	1	\$1.58
8	000065	M6 x 23.68 Threaded Barrel	1	\$4.13
9a	000960R	Seat Hook Open Frame Right Front Frame / Left Rear Frame, Catalyst	1	\$21.75
9b	000960L	Seat Hook Open Frame Left Front Frame / Right Rear Frame, Catalyst	1	\$21.75
10	101594	M6 x 14 CLS10.9 DIN7991 FSHCS BLKZC w/Patch	1	\$2.10

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

\* Discontinued Color

\*\* Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia*
MOR	Metallic Orange
ORG	Orange*
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer