


Catalyst 4 Height Adjustable Flip Back T-Arm

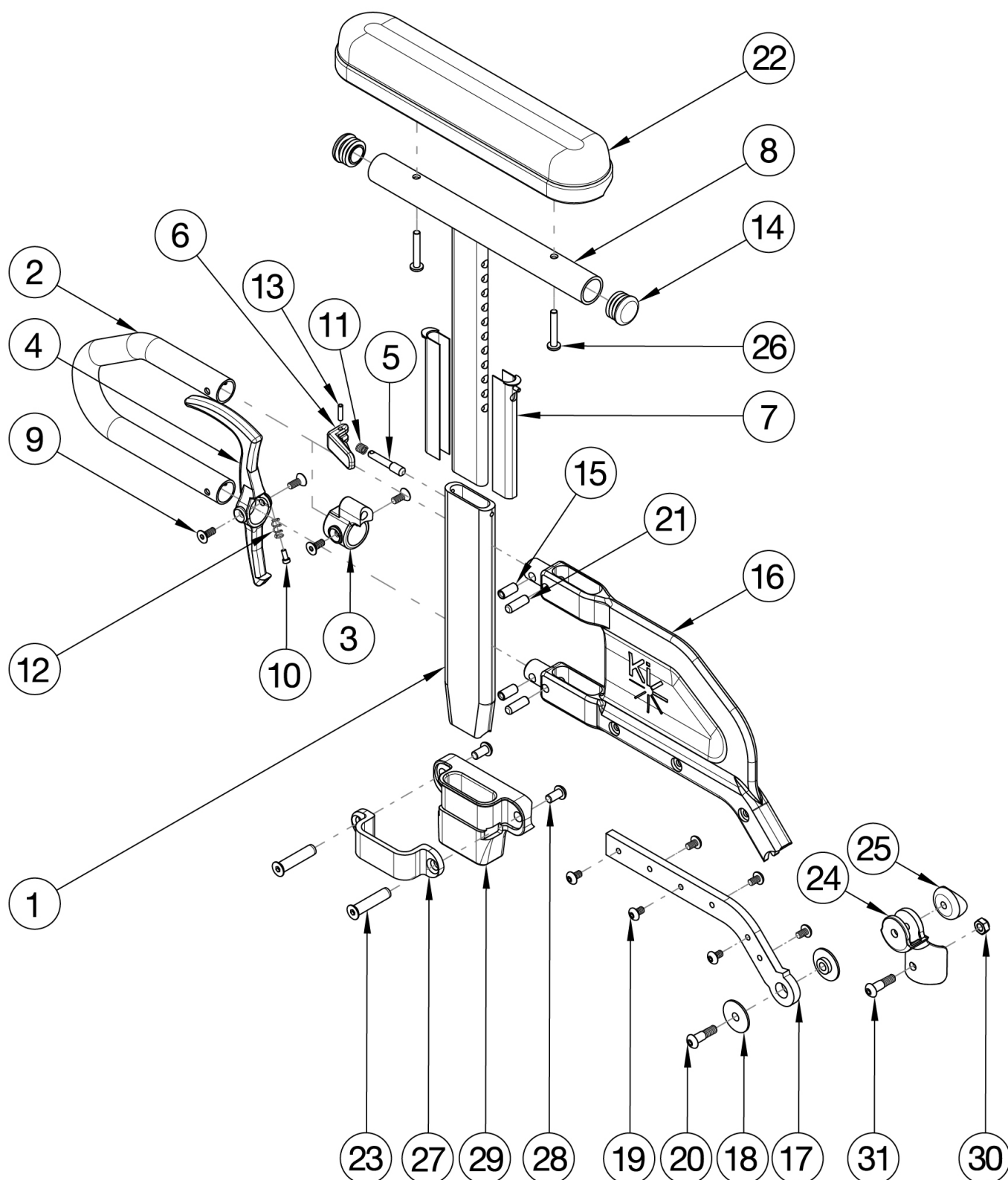

Receiver and Hardware update effective 11/25/24 Short Side Shield used with 14"-17" Seat Depths. Long Side Shield used with 18"-20" Seat Depths. If retrofiting, Receivers are needed.

Item Number	Part Number	Description	Quantity
1-7, 9-13, 15, 16a, 17a, 19, 21	004158	Height Adjustable Flip Back T-Arm Lower Assembly Short <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1
1-7, 9-13, 15, 16b, 17b, 19, 21	004159	Height Adjustable Flip Back T-Arm Lower Assembly Long <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1
1-7, 8a, 9-15, 16a, 17a, 19, 21	003459	Height Adjustable Flip Back T-Arm Short Desk <i>Does not include arm pad or arm pad screws</i>	1
1-7, 8b, 9-15, 16a, 17a, 19, 21	003460	Height Adjustable Flip Back T-Arm Short Full <i>Does not include arm pad or arm pad screws</i>	1
1-7, 8a, 9-15, 16b, 17b, 19, 21	003461	Height Adjustable Flip Back T-Arm Long Desk <i>Does not include arm pad or arm pad screws</i>	1
1-7, 8b, 9-15, 16b, 17b, 19, 21	003462	Height Adjustable Flip Back T-Arm Long Full <i>Does not include arm pad or arm pad screws</i>	1
1-7, 9-13, 15, 16a, 17a, 18-21, 24, 25	505617	Height Adjustable Flip Back T-Arm Short Retrofit Assembly, Catalyst 4 <i>This kit is used to retrofit from a Height Adjustable T-Arm to a Height Adjustable Flip Back T-Arm. This kit is to retrofit one side.</i>	1
1-7, 9-13, 15, 16b, 17b, 18-21, 24, 25	505618	Height Adjustable Flip Back T-Arm Long Retrofit Assembly, Catalyst 4 <i>This kit is used to retrofit from a Height Adjustable T-Arm to a Height Adjustable Flip Back T-Arm. This kit is to retrofit one side.</i>	1
1-7, 8a, 9-15, 16a, 17a, 18-21, 22b, 24-26, 30, 31	505266	Height Adjustable Flip Back T-Arm Short Desk with Standard Pad Assembly, Catalyst 4	1
1-7, 8b, 9-15, 16a, 17a, 18-21, 22a, 24-26, 30, 31	505267	Height Adjustable Flip Back T-Arm Short Full with Standard Pad Assembly, Catalyst 4	1
1-7, 8a, 9-15, 16b, 17b, 18-21, 22b, 24-26, 30, 31	505268	Height Adjustable Flip Back T-Arm Long Desk with Standard Pad Assembly, Catalyst 4	1
1-7, 8b, 9-15, 16b, 17b, 18-21, 22a, 24-26, 30, 31	505269	Height Adjustable Flip Back T-Arm Long Full with Standard Pad Assembly, Catalyst 4	1
1-7, 8a, 9-15, 16a, 17a, 18-21, 22b, 23-31	505270	Height Adjustable Flip Back T-Arm Short Desk with Standard Pad and Receiver Assembly, Catalyst 4	1

Item Number	Part Number	Description	Quantity
1-7, 8b, 9-15, 16a, 17a, 18-21, 22a, 23-31	505271	Height Adjustable Flip Back T-Arm Short Full with Standard Pad and Receiver Assembly, Catalyst 4	1
1-7, 8a, 9-15, 16b, 17b, 18-21, 22b, 23-31	505272	Height Adjustable Flip Back T-Arm Long Desk with Standard Pad and Receiver Assembly, Catalyst 4	1
1-7, 8b, 9-15, 16b, 17b, 18-21, 22a, 23-31	505273	Height Adjustable Flip Back T-Arm Long Full with Standard Pad and Receiver Assembly, Catalyst 4	1
18, 20, 24, 25	112957	Armrest Flip Back Height Adj T-Arm Hinge Assembly, PAIR, Catalyst 4 <i>This kit is for a pair of hinge assemblies. A pair is required as a replacement as of May 19th, 2021</i>	1
3, 5, 6, 9, 11, 13	504560	Height Adjustable Flip Back T-Arm Latch Assembly	1
4, 9, 10, 12	504561	Height Adjustable Flip Back T-Arm Release Lever Assembly	1
23, 27, 28, 29	501730	Height Adj T-Arm Receiver Assm 1" CAT	1
30, 31	504577	Height Adjustable Flip Back T-Arm Mounting Hardware Assembly <i>This kit contains various mounting hardware for multiple configurations</i>	1
1	003420	T-Arm Post	1
2	000095	Height Adjustable T Armrest Transfer Tube	1
3	000096	Height Adjustable T Armrest Pin Lock Housing	1
4	003419	Armrest Release Lever	1
5	000099	Height Adjustable Armrest Lever Pin	1
6	003418	Armrest Height Adjustment Lever	1
7	000090	Height Adjustable T Armrest Shim	2
8a	000091	Height Adj T Armrest Upper Desk	1
8b	000101	Height Adj T Armrest Upper Full	1
8a, 14	112906	Height Adjustable T Armrest Upper Desk w/ Tube End Plug	1
8b, 14	112907	Height Adjustable T Armrest Upper Full w/ Tube End Plug	1
9	003458	M5 X 12mm FHCS BLK ZN W/ Patch	2
10	000152	Height Adjustable T Armrest Release Spring Button	1
11	000134	Height Adjustable Armrest Lever Spring	1
12	000135	Height Adjustable T Armrest Release Lever Spring	1
13	200257	Dowel Pin 1/8 x 1/2	1

Item Number	Part Number	Description	Quantity
14	100803	Tube End Plug 7/8"	2
15, 16a, 17a, 19, 21	504562	Height Adjustable Flip Back T-Arm Side Shield and Metal Insert Short	1
15, 16b, 17b, 19, 21	504563	Height Adjustable Flip Back T-Arm Side Shield and Metal Insert Long	1
22a	100500	Arm Pad Full	1
22b	100501	Arm Pad Desk	1
23	122055	Binding Post, Flat Head, M6, .38"	2
26	100696	M5 x 30mm PPH Capscrew	2
27	122222	Height Adjustable T Armrest Bracket, CAT/SPARK	1
28	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2
29	122221	T-Arm Receiver 1", Cat	1
30	100658	M6 Nylock Nut Blk Zn	2
31	100544	M6 x 75 CLS 8.8 DIN931 HHCS ZC	2