


# CR45 Seat Frame

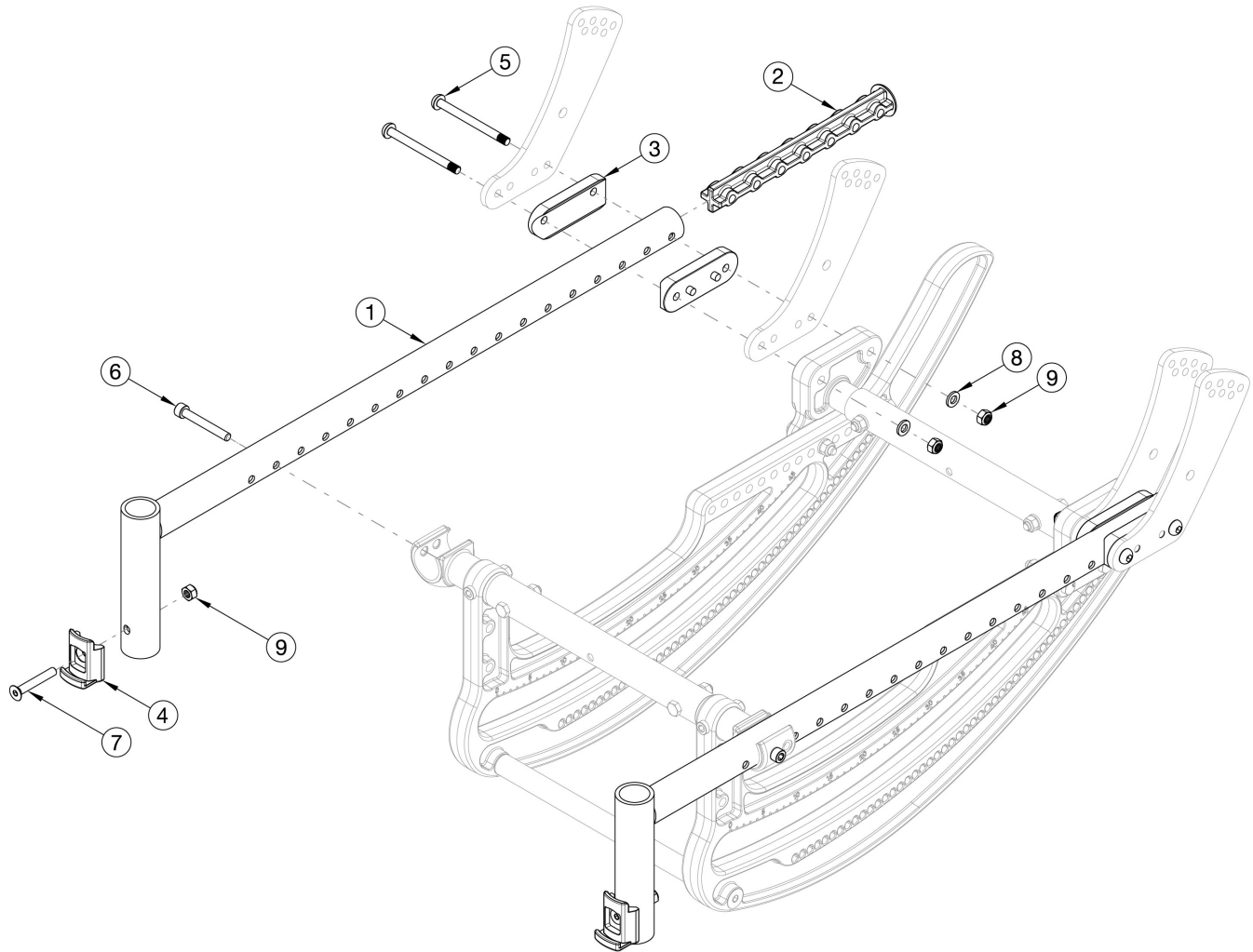

Item Number	Part Number	Description	Quantity
1a, 2, 4, 7, 9	116363_ _ _	Seat Tube Assy, Sht	1
1b, 2, 4, 7, 9	116364_ _ _	Seat Tube Assy, Med	1
1c, 2, 4, 7, 9	116365_ _ _	Seat Tube Assy, Lng	1
1a	112446	Short Seat Tube	2
1b	112447	Medium Seat Tube	2
1c	112448	Long Seat Tube	2
2	112449	Seat Tube Plug	2
3	112491	Saddle, Dual, 1" x 2.0SP	4

Item Number	Part Number	Description	Quantity
4	000336	Latch Block	2
5	107139	M6 x 70, BHCS, P/THD BLZ	4
6	101472	M6 x 45 SHCS BLK ZC	2
7	101819	M6 x 40 FHCS Blk Zn	2
8	100746	M6 Flat Washer Blk Zn	4
9	100658	M6 Nylock Nut Blk Zn	6