


## Catalyst E Armrests - Tall Height Adjustable Flip Back T-Arm

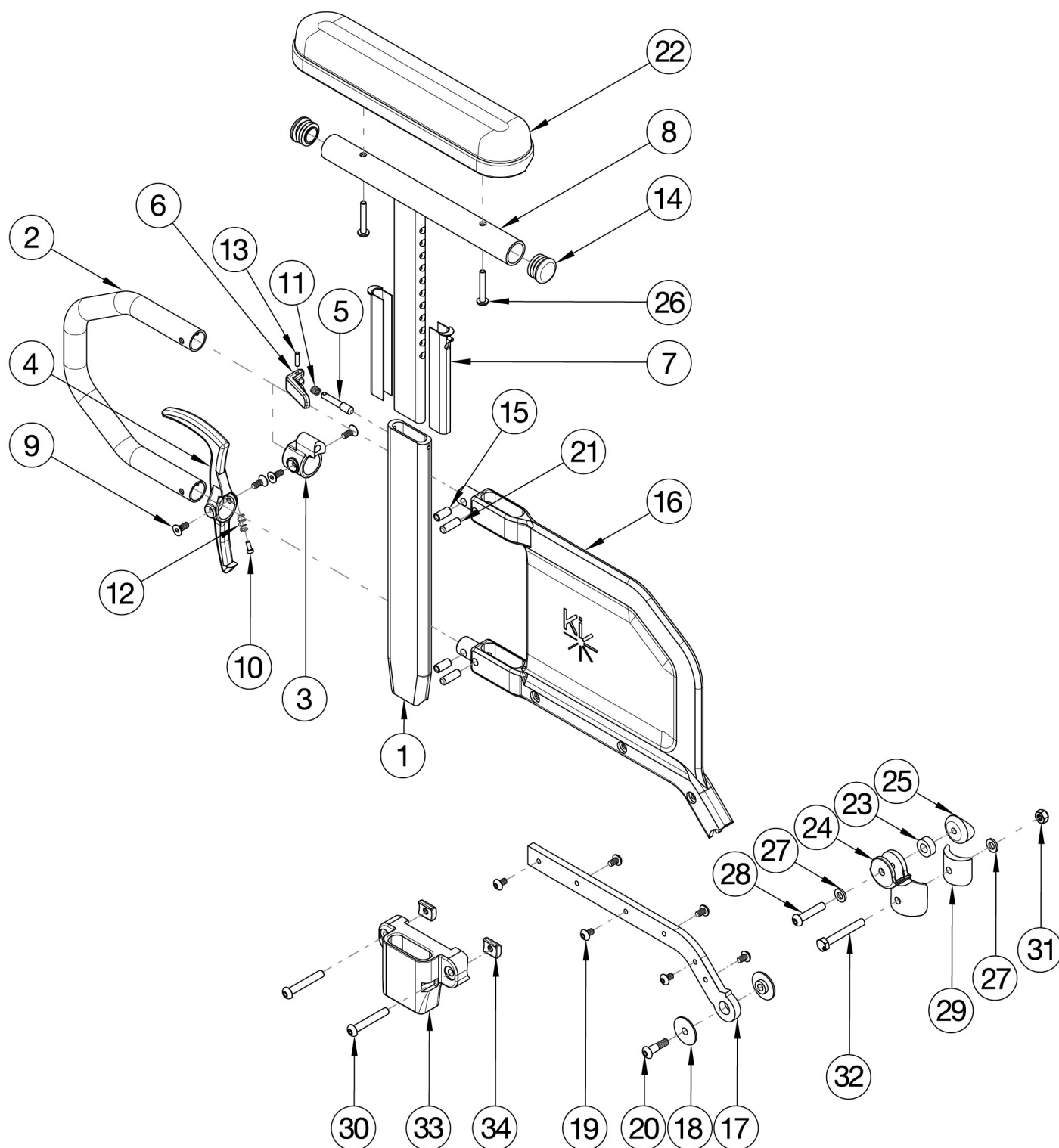

Short assemblies used with 14" - 16" Seat Depths. Long assemblies used with 17" - 20" Seat Depths.

| Item Number                                        | Part Number | Description                                                                                                                | Quantity |
|----------------------------------------------------|-------------|----------------------------------------------------------------------------------------------------------------------------|----------|
| 1-7, 8a, 9-15, 16a, 17a, 18-21, 22b, 24-26, 30-35  | 116560      | Height Adjustable Flip Back Tall T-Arm Short Desk with Standard Pad and Receiver Assembly, Catalyst E                      | 1        |
| 1-7, 8b, 9-15, 16a, 17a, 18-21, 22a, 24-26, 30-35  | 116561      | Height Adjustable Flip Back Tall T-Arm Short Full with Standard Pad and Receiver Assembly, Catalyst E                      | 1        |
| 1-7, 8a, 9-15, 16b, 17b, 18-21, 22b, 24-26, 30-35  | 116562      | Height Adjustable Flip Back Tall T-Arm Long Desk with Standard Pad and Receiver Assembly, Catalyst E                       | 1        |
| 1-7, 8b, 9-15, 16b, 17b, 18-21, 22a, 24-26, 30-35  | 116563      | Height Adjustable Flip Back Tall T-Arm Long Full with Standard Pad and Receiver Assembly, Catalyst E                       | 1        |
| 1-7, 8a, 9-15, 16a, 17a, 18-21, 22b, 24-26, 31, 32 | 116556      | Height Adjustable Flip Back Tall T-Arm Short Desk with Standard Pad Assembly, Catalyst E                                   | 1        |
| 1-7, 8b, 9-15, 16a, 17a, 18-21, 22a, 24-26, 31, 32 | 116557      | Height Adjustable Flip Back Tall T-Arm Short Full with Standard Pad Assembly, Catalyst E                                   | 1        |
| 1-7, 8a, 9-15, 16b, 17b, 18-21, 22b, 24-26, 31, 32 | 116558      | Height Adjustable Flip Back Tall T-Arm Long Desk with Standard Pad Assembly, Catalyst E                                    | 1        |
| 1-7, 8b, 9-15, 16b, 17b, 18-21, 22a, 24-26, 31, 32 | 116559      | Height Adjustable Flip Back Tall T-Arm Long Full with Standard Pad Assembly, Catalyst E                                    | 1        |
| 1-7, 9-13, 15, 16a, 17a, 19, 21                    | 114152      | Height Adjustable Flip Back T-Arm Short Lower Assy, Tall<br><i>Does not include t-arm upper, arm pad or arm pad screws</i> | 1        |
| 1-7, 9-13, 15, 16b, 17b, 19, 21                    | 114153      | Height Adjustable Flip Back T-Arm Long Lower Assy, Tall<br><i>Does not include t-arm upper, arm pad or arm pad screws</i>  | 1        |
| 1-7, 8a, 9-15, 16a, 17a, 19, 21                    | 114436      | Height Adjustable Flip Back T-Arm Desk, Short, Tall<br><i>Does not include arm pad or arm pad screws</i>                   | 1        |
| 1-7, 8b, 9-15, 16a, 17a, 19, 21                    | 114437      | Height Adjustable Flip Back T-Arm Short, Full, Tall<br><i>Does not include arm pad or arm pad screws</i>                   | 1        |
| 1-7, 8a, 9-15, 16b, 17b, 19, 21                    | 114438      | Height Adjustable Flip Back T-Arm Long, Desk, Tall<br><i>Does not include arm pad or arm pad screws</i>                    | 1        |
| 1-7, 8b, 9-15, 16b, 17b, 19, 21                    | 114439      | Height Adjustable Flip Back T-Arm Long, Full, Tall<br><i>Does not include arm pad or arm pad screws</i>                    | 1        |

| Item Number                                    | Part Number | Description                                                                                                                                                                                                                    | Quantity |
|------------------------------------------------|-------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------|
| 1-7, 9-13, 15, 16a, 17a, 18-21, 24, 25, 31, 32 | 504567      | Height Adjustable Flip Back T-Arm Short Retrofit Assembly, Catalyst 5<br><br><i>This kit is used to retrofit from a Height Adjustable T-Arm to a Height Adjustable Flip Back Tall T-Arm. This kit is to retrofit one side.</i> | 1        |
| 1-7, 9-13, 15, 16b, 17b, 18-21, 24, 25, 31, 32 | 504568      | Height Adjustable Flip Back T-Arm Long Retrofit Assembly, Catalyst 5<br><br><i>This kit is used to retro fit from a Height Adjustable T-Arm to a Height Adjustable Flip Back Tall T-Arm. This kit is to retrofit one side.</i> | 1        |
| 18, 20, 24, 25                                 | 112958      | Armrest Flip Back Height Adj T-Arm Hinge Assembly, PAIR, Catalyst 5<br><br><i>This kit is for a pair of hinge assemblies. A pair is required as a replacement as of May 19th, 2021</i>                                         | 1        |
| 30, 33-34                                      | 116058      | Height Adjustable Flip Back Receiver Assm Cat E                                                                                                                                                                                | 1        |
| 31, 32                                         | 504577      | Height Adjustable Flip Back T-Arm Mounting Hardware Assembly<br><br><i>This kit contains various mounting hardware for multiple configurations</i>                                                                             | 1        |
| 3, 5, 6, 9, 11, 13                             | 504560      | Height Adjustable Flip Back T-Arm Latch Assembly                                                                                                                                                                               | 1        |
| 4, 9, 10, 12                                   | 504561      | Height Adjustable Flip Back T-Arm Release Lever Assembly                                                                                                                                                                       | 1        |
| 1                                              | 115983      | Height Adjustable Flip Back T-Arm Post Tall                                                                                                                                                                                    | 1        |
| 2                                              | 113138      | Height Adjustable T-Arm Transfer Tube, Tall                                                                                                                                                                                    | 1        |
| 7                                              | 000090      | Height Adjustable T Armrest Shim                                                                                                                                                                                               | 2        |
| 8a, 14                                         | 116564      | Height Adj Tall T Armrest Upper Desk w/ Tube End Plug                                                                                                                                                                          | 1        |
| 8b, 14                                         | 116565      | Height Adj T Armrest Upper Full w/ Tube End Plug                                                                                                                                                                               | 1        |
| 14                                             | 100803      | Tube End Plug 7/8"                                                                                                                                                                                                             | 2        |
| 15, 16a, 21                                    | 113143      | Height Adjustable Flip Back T-Arm Short Shield, Tall                                                                                                                                                                           | 1        |
| 15, 16b, 21                                    | 113142      | Height Adjustable Flip Back T-Arm Long Side Shield, Tall                                                                                                                                                                       | 1        |
| 17a, 19                                        | 504564      | Height Adjustable Flip Back T-Arm Metal Insert Short                                                                                                                                                                           | 1        |
| 17b, 19                                        | 504565      | Height Adjustable Flip Back T-Arm Metal Insert Long                                                                                                                                                                            | 1        |
| 22a                                            | 100500      | Arm Pad Full                                                                                                                                                                                                                   | 1        |
| 22b                                            | 100501      | Arm Pad Desk                                                                                                                                                                                                                   | 1        |
| 23                                             | 003261      | Spacer, .62ODx.316IDx.250L                                                                                                                                                                                                     | 2        |
| 26                                             | 100696      | M5 x 30mm PPH Capscrew                                                                                                                                                                                                         | 2        |

| Item Number | Part Number | Description                        | Quantity |
|-------------|-------------|------------------------------------|----------|
| 27          | 100746      | M6 Flat Washer Blk Zn              | 2        |
| 28          | 102625      | M6 X 30 BHCS BLK ZC                | 1        |
| 29          | 111976      | Flip Back T-Arm Spacer, Catalyst E | 1        |
| 30          | 100797      | M6 x 40 BHCS BLK ZC                | 2        |
| 33          | 112010      | T-Arm Receiver                     | 1        |
| 34          | 111954      | Threaded Half Nut Plate            | 1        |
| 35          | 100831      | 1/2" Washer OD .040" Thick         | 1        |