


Focus CR / Liberty Removable Stroller Handle

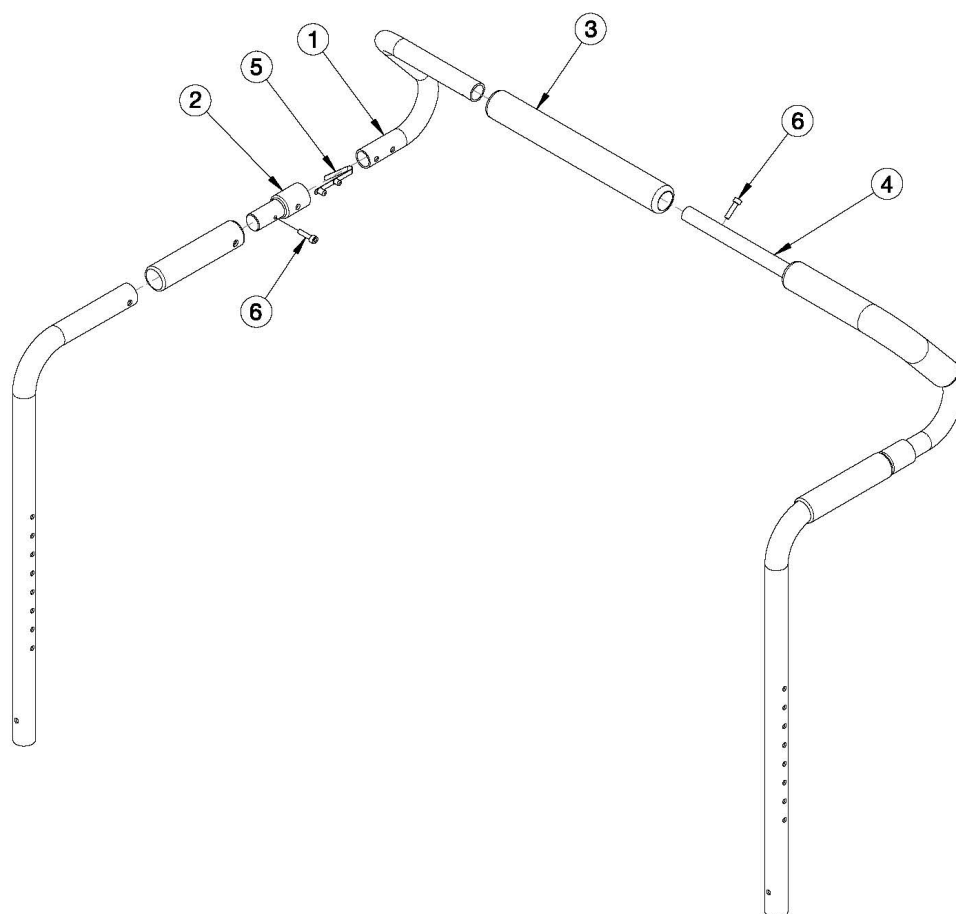

Part No.	Growth Range	Used on Widths	Length
002578	14"-20"	14"-17"	8.75"
002579	16"-22"	18"-22"	10.75"
112003	18"-24"	23"-24"	12.75"

Assemblies and Growth Strut based on Seat Width. If retrofitting, new back canes will be required. If changing from another style and Drum Brake present, Brake Lever may be needed

Item Number	Part Number	Description	Quantity
1a, 1b, 2, 3, 4a, 5, 6	503104	Removable Stroller Handle Assembly 14-18W Focus/CR45/CR45/Flip/Liberty	1
1a, 1b, 2, 3, 4b, 5, 6	503105	Removable Stroller Handle Assembly 18-22W Focus/CR45/CR45/Liberty	1
1a, 1b, 2, 3, 4c, 5, 6	114288	Removable Stroller Handle Assm 23-24W Focus	1
1a, 3	501090	Stroller Handle with Foam Grip, Right	1
1b, 3	501091	Stroller Handle with Foam Grip, Left	1
2	001243	Stroller Handle Adaptor	2
4a	002578	Growth Strut, Stroller, Short	1
4b	002579	Growth Strut, Stroller, Long	1

Item Number	Part Number	Description	Quantity
4c	112003	Growth Strut, Stroller, Extra Long	1
5	100533	Detent Button	2
6	101843	M5 x 20 SHCS Blk Zn	4