


Arc Folding Rigidizer Bar

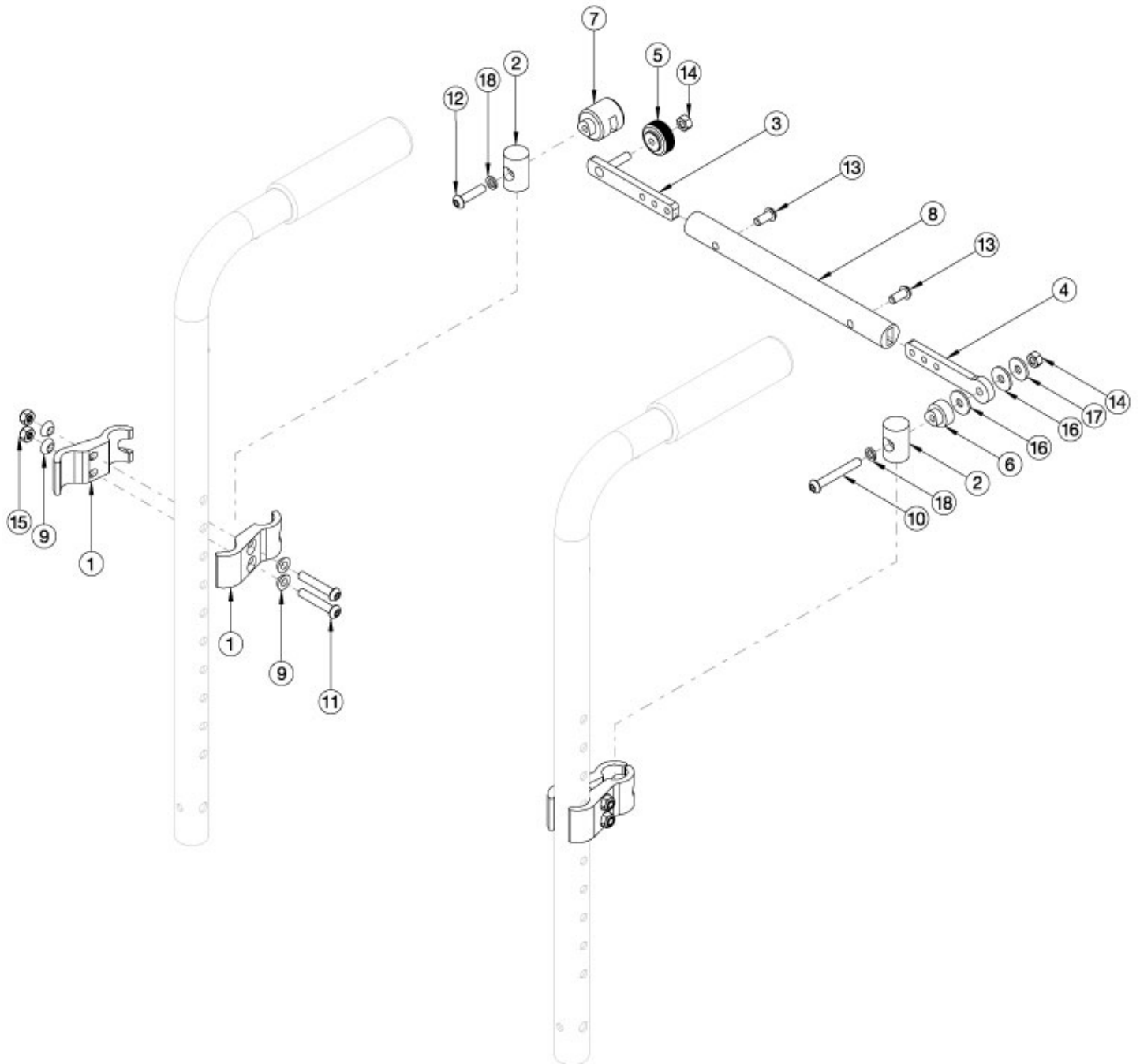

Item Number	Part Number	Description	Quantity
1-7, 8a, 9-18	113504	Folding Rigidizer Bar Assembly with Clamp, 10-12" Wide, Arc	1
1-7, 8b, 9-18	113505	Folding Rigidizer Bar Assembly with Clamp, 12-14" Wide, Arc	1
1-7, 8c, 9-18	113506	Folding Rigidizer Bar Assembly with Clamp, 14-16" Wide, Arc	1
1-7, 8d, 9-18	113507	Folding Rigidizer Bar Assembly with Clamp, 16-18" Wide, Arc	1

Item Number	Part Number	Description	Quantity
2-7, 8a, 10, 12-14, 16-18	113508	Folding Rigidizer Bar Assembly, 10-12" Wide, Arc	1
2-7, 8b, 10, 12-14, 16-18	113509	Folding Rigidizer Bar Assembly, 12-14" Wide, Arc	1
2-7, 8c, 10, 12-14, 16-18	113510	Folding Rigidizer Bar Assembly, 14-16" Wide, Arc	1
2-7, 8d, 10, 12-14, 16-18	113511	Folding Rigidizer Bar Assembly, 16-18" Wide, Arc	1
1, 9, 11, 15	113512	Folding Rigidizer Bar Clamp with Hardware, One Side, Arc	1
2, 5-7, 10, 12, 14, 16-18	113513	Folding Rigidizer Bar Hardware Kit, Arc	1
1	111589	Clamp, Mount, Accessory	2
2	111577	Barrel, Mount, Stabilizer Bar	2
3	111579	Latch, Stabilizer Bar	1
4	111578	Pivot, Stabilizer Bar	1
5	111586	Knob, Knurled, M6	1
6	111587	Spacer, Pivot, Stabilizer Bar	1
7	111588	Clevis, Latch, Stabilizer Bar	1
8a	111580	Cross Tube Folding Rigidizer, Stabilizer Bar, 10-12W	1
8b	111581	Cross Tube Folding Rigidizer, Stabilizer Bar, 12-14W	1
8c	111582	Cross Tube Folding Rigidizer, Stabilizer Bar, 14-16W	1
8d	111583	Cross Tube Folding Rigidizer, Stabilizer Bar, 16-18W	1
9	003810	Washer, Spher, 10Mm	8
10	100670	M6 x 45 BTN SHCS BLK ZC	1
11	101811	M6 x 35 BHCS Blk Zn	4
12	101593	M6 x 25 BTN SHCS BLKZC	1
13	101708	M6 x 14 BTN SHCS BLKZC w/Patch	2
14	101456	M6 Ovl Top L/N ZC	2
15	100658	M6 Nylock Nut Blk Zn	4
16	102629	Seastrom 5610-69-062 Washer Nylon Black	2
17	101853	M6 Washer Blk Zn	1
18	102810	M6 Lockwasher, Hi-Collar, ZC	2