


# Catalyst Straight Backrest with Push Handle

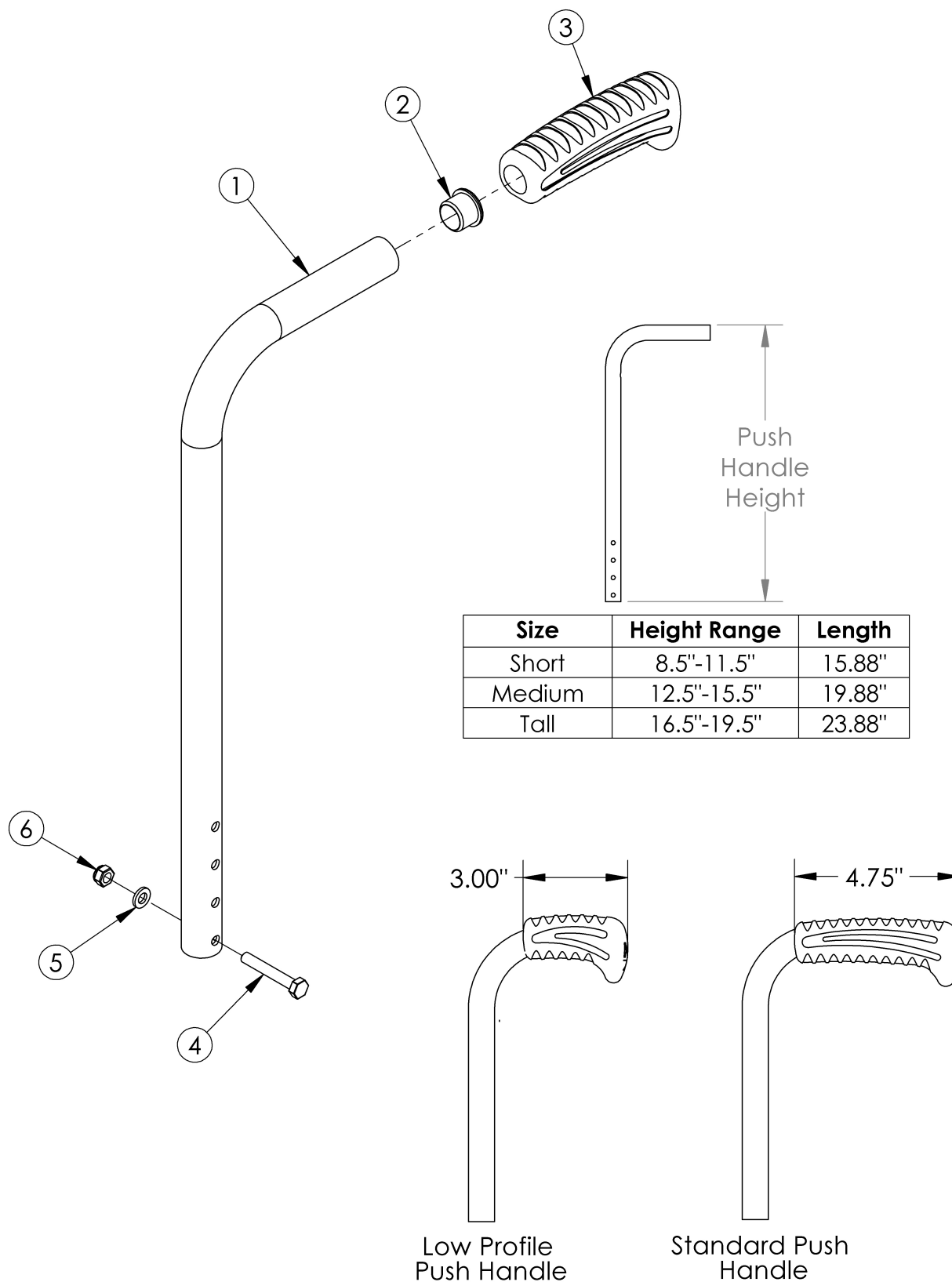

Low Profile Push Handles are not available with Heavy Duty Option or Drum Brake If changing from another style backrest and the Drum Brake is present, a new Drum Brake Cable Kit may be required

Item Number	Part Number	Description	Quantity
1a, 2, 3a	100715	Back Post Assembly Short Straight with Standard Push Handle <i>Used if drum brake is not present</i>	1
1b, 2, 3a	100716	Back Post Assembly Medium Straight with Standard Push Handle <i>Used if drum brake is not present</i>	1
1c, 2, 3a	100717	Back Post Assembly Tall Straight with Standard Push Handle <i>Used if drum brake is not present</i>	1
1d, 2, 3a	113299	Backtube Straight w/ Push Handle Short with Backcane Extension, Drum Brake, Catalyst 5/VX <i>Used if drum brake is present. To view the backcane extension included in this kit, see the Catalyst 5 or Catalyst 5VX Drum Brake page</i>	1
1e, 2, 3a	113300	Backtube Straight w/ Push Handle Medium with Backcane Extension, Drum Brake, Catalyst 5/VX <i>Used if drum brake is present. To view the backcane extension included in this kit, see the Catalyst 5 or Catalyst 5VX Drum Brake page</i>	1
1f, 2, 3a	113301	Backtube Straight w/ Push Handle Tall with Backcane Extension, Drum Brake, Catalyst 5/VX <i>Used if drum brake is present. To view the backcane extension included in this kit, see the Catalyst 5 or Catalyst 5VX Drum Brake page</i>	1
1a, 2, 3b	002752	Back Post Assembly Short Straight with Low Profile Push Handle	1
1b, 2, 3b	002753	Back Post Assembly Medium Straight with Low Profile Push Handle	1
1c, 2, 3b	002754	Back Post Assembly Tall Straight with Low Profile Push Handle	1
2	100803	Tube End Plug 7/8"	1
3a	109433	Ergo Push Handle Grip 7/8" Standard with Instruction	1
4	100681	M6 x 40 CLS8.8 DIN931 HHCS ZC	1
5	100746	M6 Flat Washer Blk Zn	1
6	100658	M6 Nylock Nut Blk Zn	1