


Spark Cross Braces

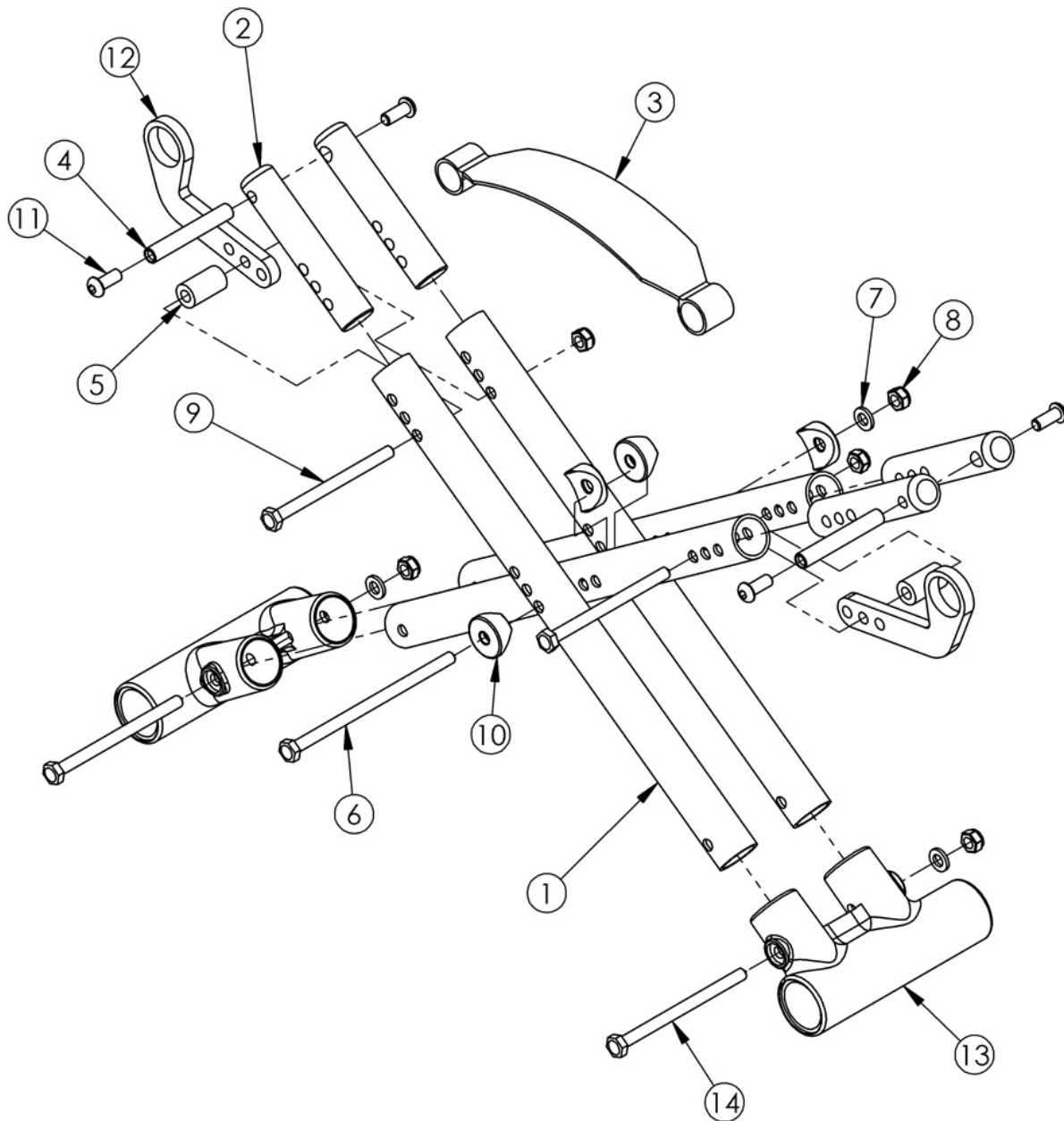

Item Number	Part Number	Description	Quantity	Retail
4-11, 14	501512	Cross Brace Hardware Kit SPARK	1	\$58.86
1a	000636	Cross Tube 10-11" Wide SPARK	4	\$37.14
1b	000637	Cross Tube 11-13" Wide SPARK	4	\$37.14
1c	000638	Cross Tube 12-14" Wide SPARK	4	\$37.14

Item Number	Part Number	Description	Quantity	Retail
1d	000639	Cross Tube 13-15" Wide SPARK	4	\$37.14
1e	000640	Cross Tube 14-16" Wide SPARK	4	\$37.14
2a	000630	Cross Tube Extension 10-11" Wide SPARK	4	\$24.38
2b	000631	Cross Tube Extension 11-13" Wide SPARK	4	\$24.38
2c	000632	Cross Tube Extension 12-14" Wide SPARK	4	\$24.38
2d	000633	Cross Tube Extension 13-15" Wide SPARK	4	\$24.38
2e	000634	Cross Tube Extension 14-16" Wide SPARK	4	\$24.38
3	000673	Cross Tube Lift Strap, Spark/Arc	1	\$7.50
4	000422	M6 x 56 Threaded Barrel	2	\$10.64
5	101099	Cross Tube Spacer SPARK	2	\$5.02
6	100678	M6 x 110 CLS8.8 DIN931 HHCS ZC	1	\$2.63
7	100746	M6 Flat Washer Blk Zn	5	\$1.58
8	100658	M6 Nylock Nut Blk Zn	5	\$1.05
9	100660	M6 x 80mm CLS8.8 DIN931 HHCS ZC	2	\$1.58
10	000054	Saddle 7/8"	4	\$1.58
11	100668	M6 x 20 CLS10.9 BTN SHCS ZC w/Patch	4	\$1.58
12a	000641	Cross Tube Hinge 10-11" Wide SPARK	2	\$44.68
12b	000642	Cross Tube Hinge 11-13" Wide SPARK	2	\$44.68
12c	000643	Cross Tube Hinge 12-14" Wide SPARK	2	\$44.68
12d	000644	Cross Tube Hinge 13-15" Wide SPARK	2	\$44.68
12e	000645	Cross Tube Hinge 14-16" Wide SPARK	2	\$44.68
13	000001	Lower Frame Pivot CAT	2	\$27.71
14	100704	M6 x 90 CLS8.8 DIN931 HHCS ZC	2	\$1.58