


## Flip for Leckey Push Wheel Lock 16" Wheels

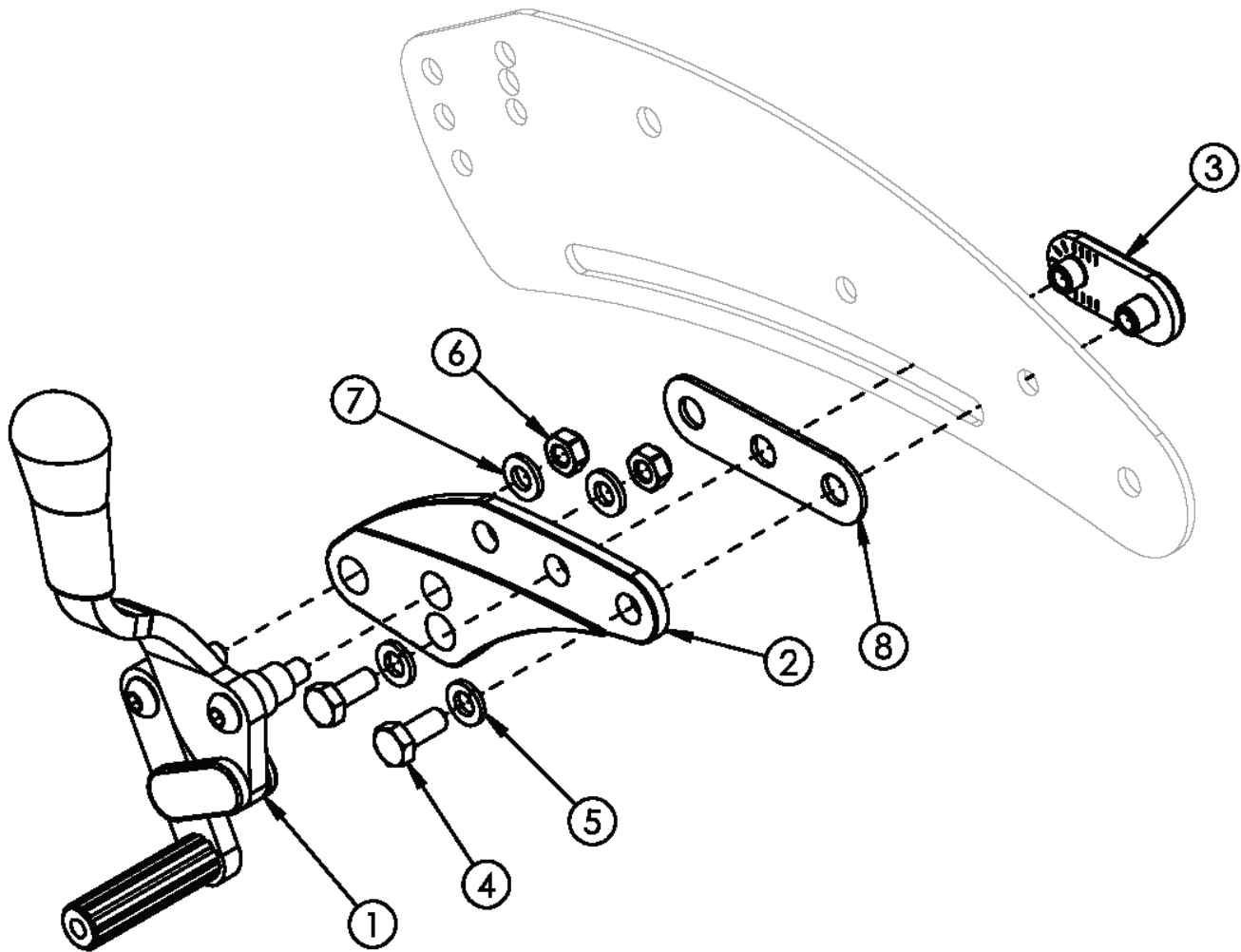

Item Number	Part Number	Description	Quantity
1a, 2 - 8	502990	Wheel Lock Assembly, Std Push, Right, Flip	1
1b, 2 - 8	502991	Wheel Lock Assembly, Std Push, Left, Flip	1
1a, 1b, 2 - 8	502836	Wheel Lock Assembly, Std Push, PAIR, Flip	1
3	001905	Nut, Slide, Flip	1
4	102794	M6 x 14, HHCS, Blk Zn	2
5	101835	M6 Lock Washer Blk Zn	2
6	100658	M6 Nylock Nut Blk Zn	2

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>
7	100746	M6 Flat Washer Blk Zn	2
8	002913	Wheel Lock Mount Spacer FLIP	1