


Rogue ALX / Little Wave XP Frog Legs Suspension Fork (Ultra-Sport Fork) (Formerly Tsunami)

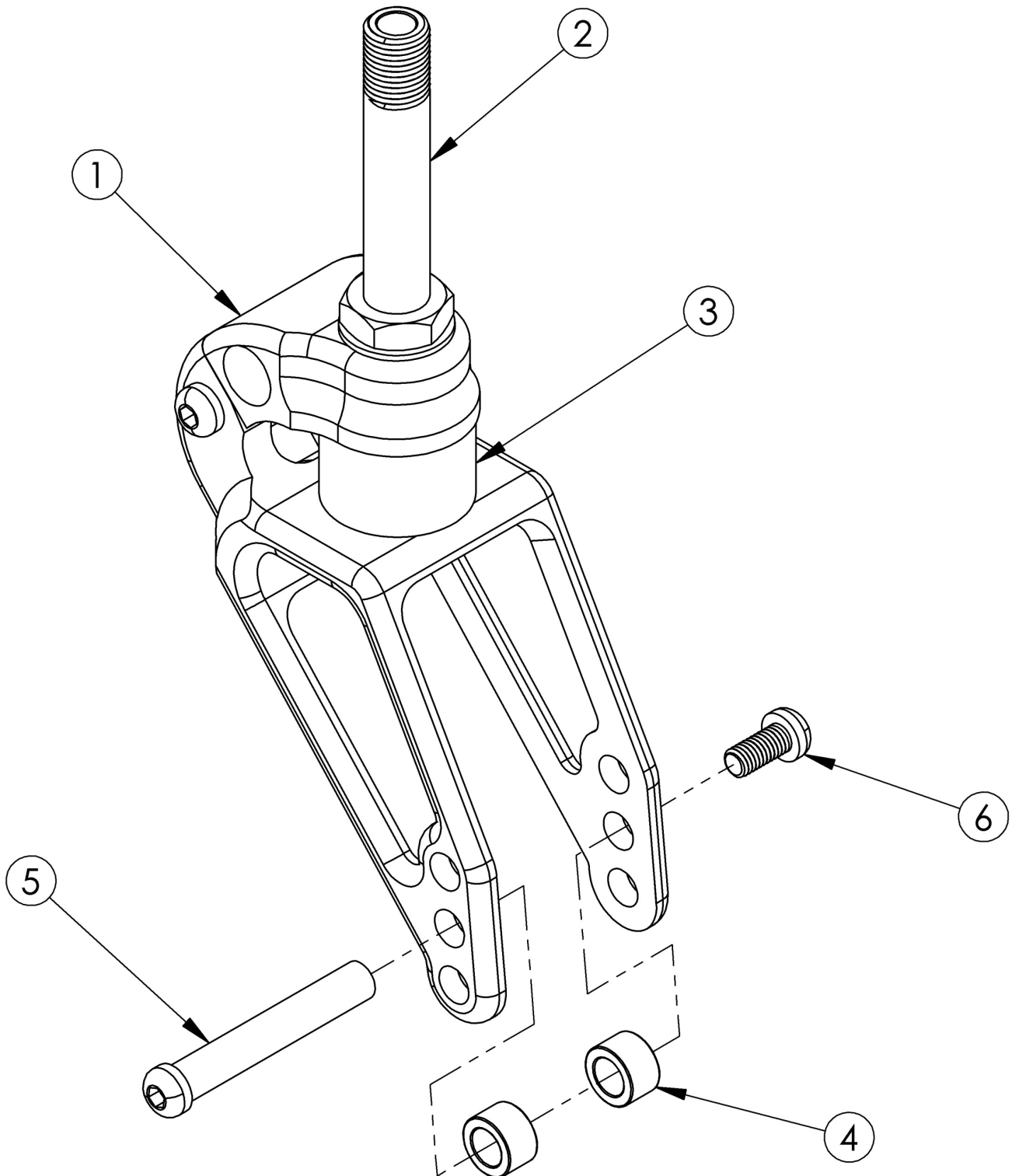

Not available with Heavy Duty Option

Item Number	Part Number	Description	Quantity
1, 2, 4-6	503746	Frog Legs Suspension Fork Assembly with 1/2 x 1.85" Stem, RG ALX/LW XP <i>This assembly does not include a polymer.</i>	1
1, 4 - 6	100549	Frog Legs Suspension Fork, Catalyst/RG ALX/LW XP <i>This assembly does not include a polymer.</i>	1
4-6	102723	Frog Legs Suspension Axle w/Hdwre PAIR <i>This hardware may incur a longer lead time</i>	1
2	105073	Frog Legs Fork Stem 1/2 x 1.85", Std	1
3a	105053	Frog Legs Polymer < 80lbs	1
3b	105054	Frog Legs Polymer 80 - 120lbs	1
3c	105055	Frog Legs Polymer 120 - 160lbs	1
3d	105056	Frog Legs Polymer 160 - 220lbs	1
3e	105057	Frog Legs Polymer 220 - 260lbs	1
3f	105058	Frog Legs Polymer 260 - 300lbs	1