


Catalyst Straight Backrest with Push Handle

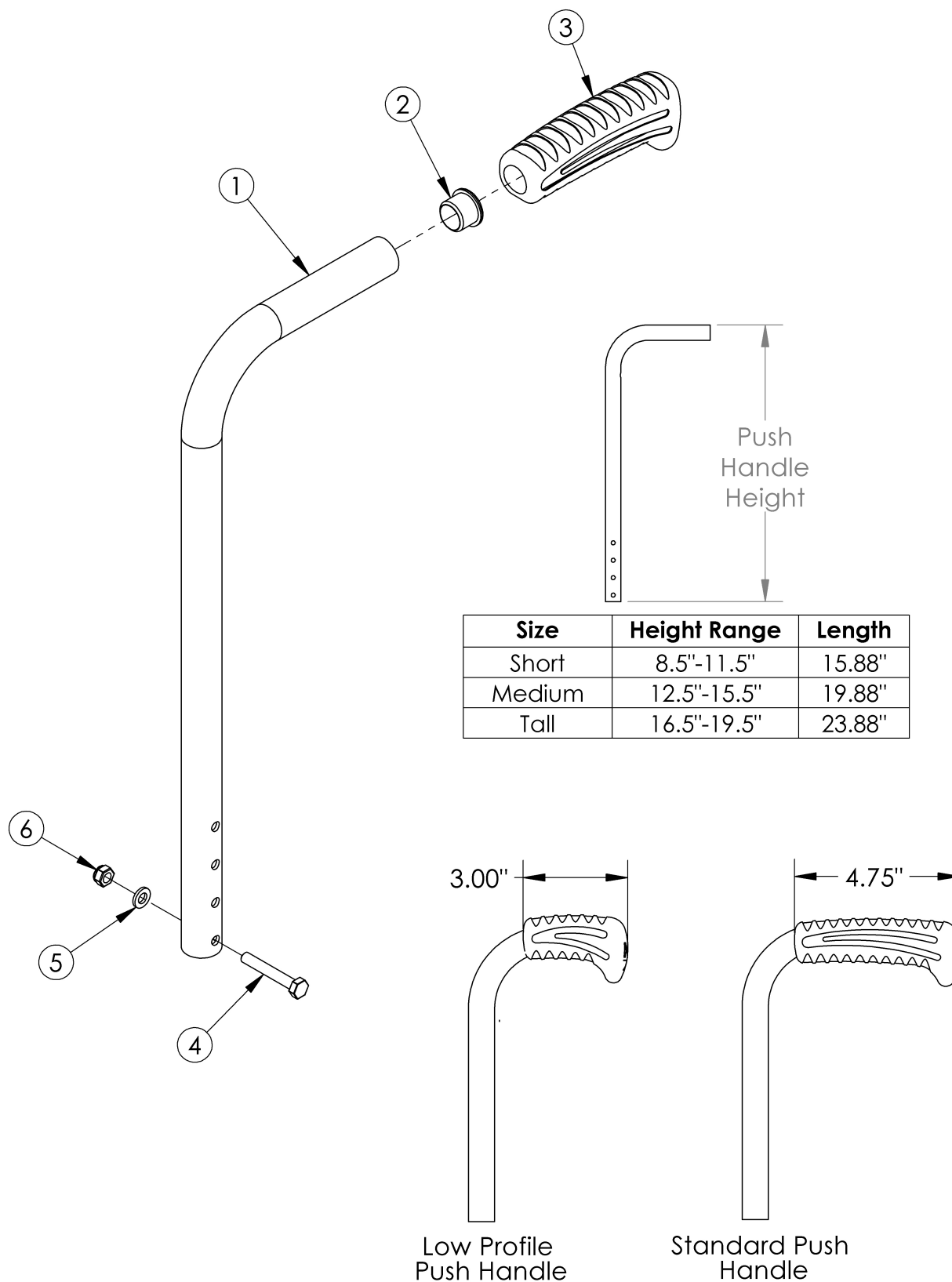

Low Profile Push Handles are not available with Heavy Duty Option or Drum Brake <p>

If changing from another style backrest and the Drum Brake is present, a new Drum Brake Cable Kit may be required

Item Number	Part Number	Description	Quantity	Retail
1a, 2, 3a	100715	Back Post Assembly Short Straight with Standard Push Handle <i>Used if drum brake is not present</i>	1	\$72.60
1b, 2, 3a	100716	Back Post Assembly Medium Straight with Standard Push Handle <i>Used if drum brake is not present</i>	1	\$72.60
1c, 2, 3a	100717	Back Post Assembly Tall Straight with Standard Push Handle <i>Used if drum brake is not present</i>	1	\$72.60
1d, 2, 3a	113299	Backtube Straight w/ Push Handle Short with Backcane Extension, Drum Brake, Catalyst 5/VX <i>Used if drum brake is present. To view the backcane extension included in this kit, see the Catalyst 5 or Catalyst 5VX Drum Brake page</i>	1	\$51.75
1e, 2, 3a	113300	Backtube Straight w/ Push Handle Medium with Backcane Extension, Drum Brake, Catalyst 5/VX <i>Used if drum brake is present. To view the backcane extension included in this kit, see the Catalyst 5 or Catalyst 5VX Drum Brake page</i>	1	\$51.75
1f, 2, 3a	113301	Backtube Straight w/ Push Handle Tall with Backcane Extension, Drum Brake, Catalyst 5/VX <i>Used if drum brake is present. To view the backcane extension included in this kit, see the Catalyst 5 or Catalyst 5VX Drum Brake page</i>	1	\$51.75
1a, 2, 3b	002752	Back Post Assembly Short Straight with Low Profile Push Handle	1	\$72.60
1b, 2, 3b	002753	Back Post Assembly Medium Straight with Low Profile Push Handle	1	\$72.60
1c, 2, 3b	002754	Back Post Assembly Tall Straight with Low Profile Push Handle	1	\$72.60
2	100803	Tube End Plug 7/8"	1	\$1.58
3a	109433	Ergo Push Handle Grip 7/8" Standard with Instruction	1	\$7.14
3b	109431	Ergo Push Handle Grip 7/8" Low Profile with Instruction	1	\$7.14
4	100681	M6 x 40 CLS8.8 DIN931 HHCS ZC	1	\$1.58
5	100746	M6 Flat Washer Blk Zn	1	\$1.58
6	100658	M6 Nylock Nut Blk Zn	1	\$1.05