


Focus CR Fixed Height with Adjustable Handle Backrest

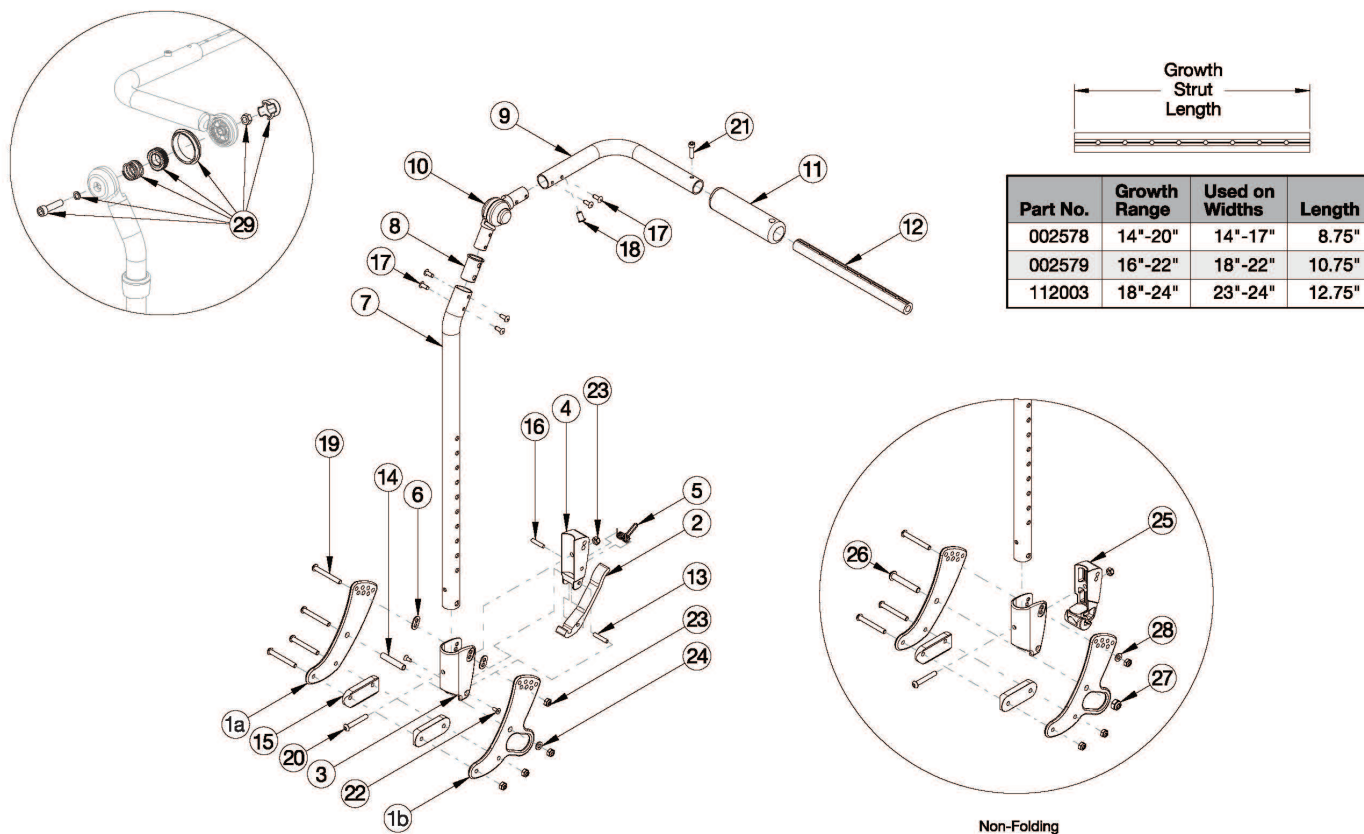

If chair was invoiced prior to 12/18/23, an upper back tube assembly is required. If chair was invoiced after to 12/18/23, parts 7-10 and 17 shown in the diagram above can now be purchased individually Used on chairs with serial numbers prior to FCR0026599 and FCHD002333, unless a foot tilt or power tilt is present. If foot tilt or power tilt is present, this design effective starting with serial numbers FCR0033747 and FCHD002467. If changing from another style backrest or changing back cane height and the hand tilt is present, a new Hand Tilt Cable/Trigger Kit may be required. If retrofitting from a foot tilt or power tilt to hand tilt and serial number is prior to FCR0033747 and FCHD002467, new back cane assemblies are required in addition to the hand tilt mechanism. If changing from another style and Drum Brake present, Brake Lever may be needed

Item Number	Part Number	Description	Quantity
1a, 14, 15, 19, 23, 24	502675	Backrest Hardware Assembly, Single Side, Std Focus	1
1a, 1b, 14, 15, 19, 23, 24	502676	Backrest Hardware Assembly, Single Side, Transit Focus	1
1a, 15, 19, 23, 26-28	503102	Backrest Hardware Assembly, Single Side, Std Non-Folding Focus	1
1a, 1b, 15, 19, 23, 26-28	503103	Backrest Hardware Assembly, Single Side, Transit Non-Folding Focus	1
7a, 8, 9b, 10, 11, 17, 18	506815	Backpost Assembly Fixed Height w/Adjustable Handle, Short, Left Focus	1

Item Number	Part Number	Description	Quantity
		<i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i>	
7a, 8, 9a, 10, 11, 17, 18	506816	Backpost Assembly Fixed Height w/Adjustable Handle, Short, Right Focus <i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i>	1
7b, 8, 9b, 10, 11, 17, 18	506817	Backpost Assembly Fixed Height w/Adjustable Handle, Tall, Left Focus <i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i>	1
7b, 8, 9a, 10, 11, 17, 18	506818	Backpost Assembly Fixed Height w/Adjustable Handle, Tall, Right Focus <i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i>	1
1a, 2-6, 7a, 8, 9b, 10, 11, 13-20, 22-24	506742	Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Short with Hardware, Left Focus <i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i>	1
1a, 2-6, 7a, 8, 9a, 10, 11, 13-20, 22-24	506743	Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Short with Hardware, Right Focus <i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i>	1
1a, 2-6, 7b, 8, 9b, 10, 11, 13-20, 22-24	506744	Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Tall with Hardware, Left Focus <i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i>	1
1a, 2-6, 7b, 8, 9a, 10, 11, 13-20, 22-24	506745	Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Tall with Hardware, Right Focus	1

Item Number	Part Number	Description	Quantity
		<i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i>	
1a, 3, 7a, 8, 9b, 10, 11, 15, 17-20, 23, 25-28	506746	<p>Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Short with Hardware, Non-Folding Left Focus</p> <p><i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i></p>	1
1a, 3, 7a, 8, 9a, 10, 11, 15, 17-20, 23, 25-28	506747	<p>Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Short with Hardware, Non-Folding Right Focus</p> <p><i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i></p>	1
1a, 3, 7b, 8, 9b, 10, 11, 15, 17-20, 23, 25-28	506748	<p>Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Tall with Hardware, Non-Folding Left Focus</p> <p><i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i></p>	1
1a, 3, 7b, 8, 9a, 10, 11, 15, 17-20, 23, 25-28	506749	<p>Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Tall with Hardware, Non-Folding Right Focus</p> <p><i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i></p>	1
2, 3-6, 7a, 8, 9b, 10, 11, 13, 16-18, 20, 22, 23	506750	<p>Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Short, Left Focus</p> <p><i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i></p>	1
2, 3-6, 7a, 8, 9a, 10, 11, 13, 16-18, 20, 22, 23	506751	<p>Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Short, Right Focus</p>	1

Item Number	Part Number	Description	Quantity
		<p><i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i></p>	
<p>2, 3-6, 7b, 8, 9b, 10, 11, 13, 16-18, 20, 22, 23</p>	<p>506752</p>	<p>Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Tall, Left Focus</p> <p><i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i></p>	<p>1</p>
<p>2, 3-6, 7b, 8, 9a, 10, 11, 13, 16-18, 20, 22, 23</p>	<p>506753</p>	<p>Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Tall, Right Focus</p> <p><i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i></p>	<p>1</p>
<p>3, 7a, 8, 9b, 10, 11, 17, 18, 20, 23, 25</p>	<p>506754</p>	<p>Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Short, Non-Folding Left Focus</p> <p><i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i></p>	<p>1</p>
<p>3, 7a, 8, 9a, 10, 11, 17, 18, 20, 23, 25</p>	<p>506755</p>	<p>Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Short, Non-Folding Right Focus</p> <p><i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i></p>	<p>1</p>
<p>3, 7b, 8, 9b, 10, 11, 17, 18, 20, 23, 25</p>	<p>506756</p>	<p>Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Tall, Non-Folding Left Focus</p> <p><i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i></p>	<p>1</p>
<p>3, 7b, 8, 9a, 10, 11, 17, 18, 20, 23, 25</p>	<p>506757</p>	<p>Back Assembly w/ Latch Insert, Fixed Height w/ Adjustable Handle Tall, Non-Folding Right Focus</p> <p><i>If changing to a new hand tilt mechanism and serial number is prior to FCR0026599 and FCHD002333, this kit is required in addition to the hand tilt mechanism. If replacing this side back tube assembly and foot tilt or power tilt is present and serial number is prior to FCR0033747 and FCHD002467, both left and right back tube assemblies will need to be replaced.</i></p>	<p>1</p>

Item Number	Part Number	Description	Quantity
2 - 3, 5 - 6, 13, 20	114490	Lower Backrest Latch Assembly, Fixed Height w/ Adjustable Handle, Focus	1
9b, 11, 17, 18, 21	122918	Push Handle Tube w/ Grip and Hardware, Left	1
9a, 11, 17, 18, 21	122920	Push Handle Tube w/ Grip and Hardware, Right	1
1a	001071	Back Plate Std TILT <i>Quantity of 4 used with non-transit. Quantity of 2 used with transit.</i>	2
1b	001421	Back Plate Transit Inner TILT <i>Quantity of 2 used with transit.</i>	1
2	001893	Latch Lever, Backrest	1
3, 6	503834	Cover Plate, Latch Insert Kit	1
4	001895	Latch Insert, Folding Backrest	1
5	001896	Spring, Backrest Latch	1
6	001897	Washer, Dual, PSA	2
7a	003495	Fixed Height Back Post for Adjustable Handle Short <i>Used on chairs starting 12/18/23</i>	1
7b	003496	Fixed Height Back Post for Adjustable Handle Tall <i>Used on chairs starting 12/18/23</i>	1
8	001266	Bushing .832 OD x .720 ID <i>Used on chairs starting 12/18/23</i>	1
10	116066	Locking Hinge Assembly <i>Used on chairs starting 12/18/23</i>	1
12a	002578	Growth Strut, Stroller, Short	1
12b	002579	Growth Strut, Stroller, Long	1
12c	112003	Growth Strut, Stroller, Extra Long	1
12a, 21	503107	Growth Strut Assembly Short, Focus/Flip <i>Used on chairs 14"-17" wide</i>	1
12b, 21	503108	Growth Strut Assembly Long, Flip/Focus <i>Used on chairs 18"-22" wide</i>	1
12c, 21	114289	Growth Strut Assembly Extra Long, Focus <i>Used on chairs 23"-24" wide</i>	1
13	002513	Threaded Barrel, 5mm OD x M4	1
14	002738	Spacer .313 OD x .243 ID x 1.68 long	1

Item Number	Part Number	Description	Quantity
15	002594	Saddle, Dual, 1" OD X 1.75 SP	2
16	102694	Dowel Pin, M5 X 24, Blk Zn	1
17	100669	M5 x 10 BTN SHCS BLKZC w/Patch <i>Used on chairs starting 12/18/23</i>	4
19	101781	M6 x 50mm Partial Thread BHCS Blk ZC	4
20	100797	M6 x 40 BHCS BLK ZC	1
21	101843	M5 x 20 SHCS Blk Zn	1
22	101818	M4 x 10 FHCS BLKZC w/Patch	2
23	100658	M6 Nylock Nut Blk Zn	5
24	101861	Nylon Washer 5/16 x 1/2 x .03	1
25	002035	Insert, Backrest, Non-Folding	1
26	101815	M8 x 50 BHCS Blk Zn	1
27	100659	M8 Nylock Nut Blk Zn	1
28	100746	M6 Flat Washer Blk Zn	1
29	110524	Locking Hinge Button Assembly with Instruction <i>This kit contains the contents inside callout # 10. This assembly may not be compatible if chair was invoiced prior to 6/22/16</i>	1