


# Arc Foot Tilt Mechanism

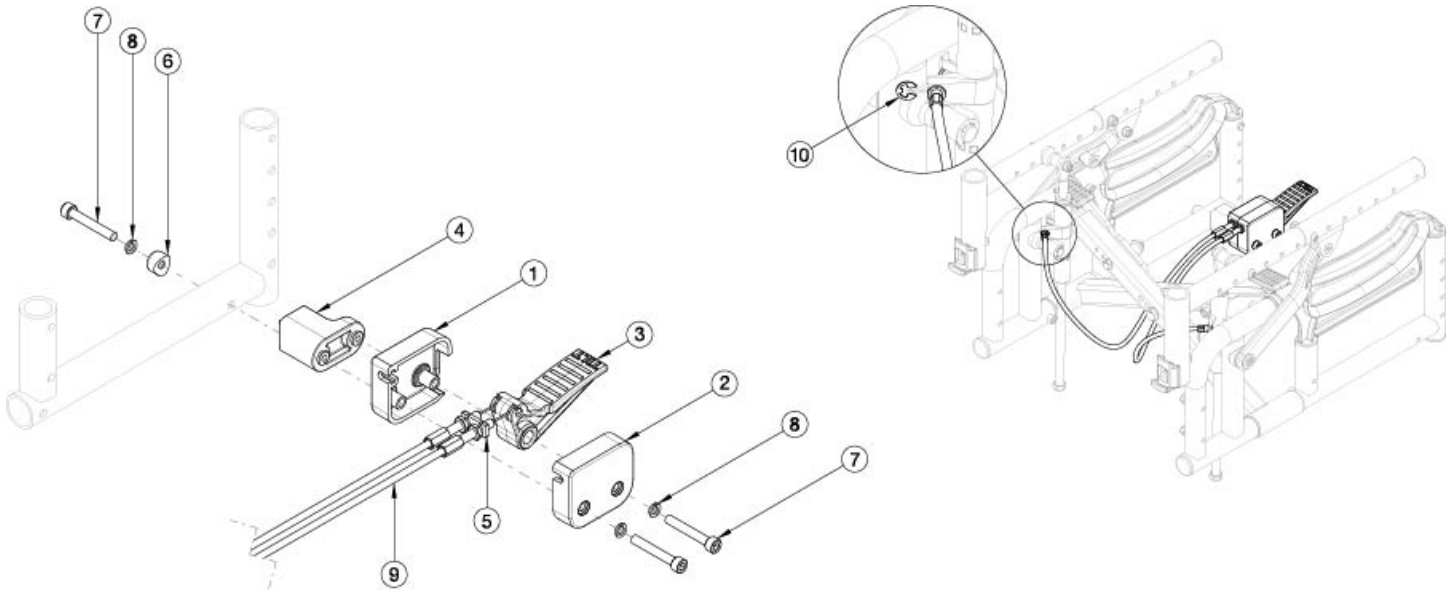

| Cables (Item 9) |                     |                      |               |                     |               |
|-----------------|---------------------|----------------------|---------------|---------------------|---------------|
| Frame Style     | Seat Width          | Extra Short (12-15D) |               | Short, Medium, Long |               |
|                 |                     | Same Side            | Opposite Side | Same Side           | Opposite Side |
| Standard Frame  | Small, 10-12"       | 113027               | 113028        | 113029              | 113029        |
|                 | Medium, 12-14"      | 113027               | 113029        | 113029              | 113030        |
|                 | Large, 14-16"       | 113027               | 113029        | 113029              | 113030        |
|                 | Extra Large, 16-18" | 113027               | 113030        | 113029              | 113031        |
| Extended Frame  | Small, 10-12"       | 113028               | 113029        | 113030              | 113030        |
|                 | Medium, 12-14"      | 113028               | 113030        | 113030              | 113031        |
|                 | Large, 14-16"       | 113028               | 113031        | 113030              | 113031        |
|                 | Extra Large, 16-18" | 113028               | 113030        | 113030              | 113031        |

Please reference the table above to determine which cables you need for your configuration. Cable kits contain one cable and one e-ring.

| Item Number | Part Number | Description                    | Quantity |
|-------------|-------------|--------------------------------|----------|
| 1-8         | 113026      | Foot Tilt Actuator Assembly    | 1        |
| 9a, 10      | 113027      | Cable, Tilt, Foot, Short, Arc  | 1        |
| 9b, 10      | 113028      | Cable, Tilt, Foot, Medium, Arc | 1        |
| 9c, 10      | 113029      | Cable, Tilt, Foot, Long, Arc   | 1        |
| 9d, 10      | 113030      | Cable, Tilt, Foot, XL, Arc     | 1        |
| 9e, 10      | 113031      | Cable, Tilt, Foot, XXL, Arc    | 1        |
| 10          | 102822      | E-Ring, M6, YZ                 | 2        |